



IDM-MULTI-9

Stealth's mo-Vis Series Multifunctional Wheelchair Joystick

- **All Round Joystick:** Regular size joystick, regular force
- **Multi Joystick:** Smaller size joystick, limited force
- **Micro Joystick:** Very small size joystick, minimal force

IDM-MULTI-9	Elegant Design	Controlled by finger, lip, or chin
	Size: 2.64in x 1.31in	
	Required Force: 50 grams	



Product Description

The Multi Joystick is recommended for those with tetraplegia, or neuro-muscular diseases such as ALS, MS, etc. A lighter and smaller version of a standard joystick, the Multi Joystick is ideal for clients with limited muscular force. It can be controlled by the finger, lip, tongue, etc.

The Multi Joystick is a purely proportional joystick with a reduced amplitude and force requirement of only 50 grams. The ball of the joystick is a compact design and is only 1.03in. x 1.19in. in size, giving the driver the capability for a more precise drive.

The joystick is built in a compact and discreet housing with two jack inputs for switch input. 1 or 2 switches, such as the Twister, can be immediately plugged in as an on/off or mode switch.

A separate interface with sophisticated electronics makes the joystick adaptable to individual capabilities and needs. The switches (on/off and mode) can be directly plugged into the jack inputs of the joystick, or the jack inputs of the interface box.

To adjust the parameters of the joystick, the free downloadable mo-Vis Configurator Software is available. The joystick connection to the pc is made by a USB cable directly from the interface to the computer (mini USB port). The Multi Joystick, with its SUB D9 pin connector, is compatible with R-Net and Q-Logic connections.

The Multi Joystick is compatible with the following Stealth mountings:

- The Swing-Away SM600 Series
- The Gatlin GAT400 Series
- The ARMS260 Series
- The Power Chin Boom

Thanks to its smaller shape and reduced force, the Multi Joystick is not only a more comfortable solution for wheelchair driving, but is also a very elegant design.

Please visit our website at www.stealthproducts.com to see more information about the mounting options available for the Multi Joystick.



About mo-Vis

The mo-Vis line is exclusive to Stealth Products. mo-Vis stands for 'move with Vision', as we like to keep moving forward and get people 'on the move'! All mo-Vis products follow this vision. Not only are functionality, flexibility and ergonomics important, but they are accompanied by discretion and elegance as well.

Contact



Questions about the Multi Joystick?

Contact us by e-mail:
info@stealthproducts.com
or by phone:
+1 (512) 715-9995
+1 (800) 965-9229

