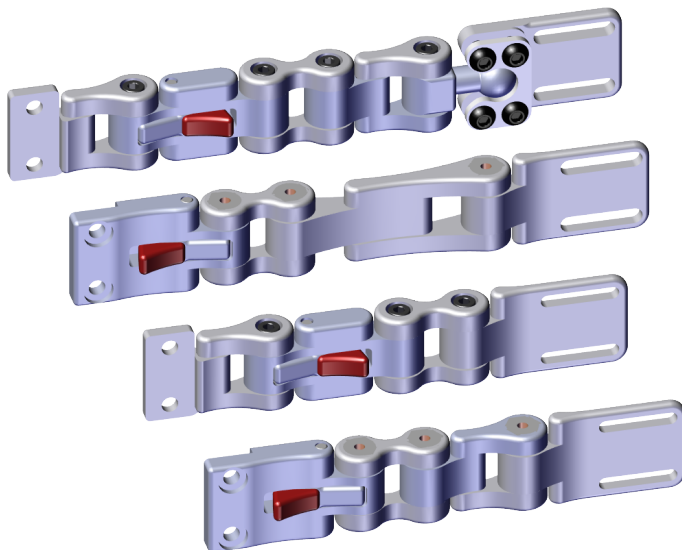
**Stealth Products**<sup>®</sup>

**Lateral and Trunk Support (EN)**  
**Seiten- und Torso-Unterstützung (DE)**  
**Support latéral et du tronc (FR)**  
**Supporto laterale e del tronco (IT)**  
**Soporte lateral y de tronco (ES)**

**UniLink Laterals**  
**UniLink Seitenteile**  
**Rampes latérales UniLink**  
**Laterali UniLink**  
**Laterales UniLink**

---

Owner's Manual - Maintenance Guide  
Benutzerhandbuch - Wartungsanleitung  
Manuel d'utilisation - Guide d'entretien  
Manuale d'uso - Guida alla manutenzione  
Manual de usuario - Guía de mantenimiento

**Customer Satisfaction .....i**  
    General..... i

**General Information .....ii**  
    Warranty..... ii  
    Supplier Reference Table..... ii

**Warning Labels.....iii**  
    Warning Labels..... iii  
    Limited Liability..... iii  
    Testing ..... iii

**Design & Function ..... 1-2**  
    Intended Use..... 1  
    Adjusting Adductors ..... 1  
    Operating Laterals ..... 2

**Installation ..... 2-5**  
    Standard Track Mounting ..... 2  
    Standard Cane Mounting..... 3  
    All-Season Mounting..... 3  
    135° Bracket Installation (BAXX Active)..... 4  
    Pad Mount Installation..... 4  
    Full-Surface Contact Mount Installation ..... 5  
    Full-Surface Contact Mount Adjustment..... 5

**First Time Use .....6**  
    Care & Cleaning ..... 6  
    Dealer Assistance ..... 6  
    User Testing ..... 6  
    Conditions of Use ..... 6  
    Disposal..... 6

**Stealth Products** is committed to 100% customer satisfaction. Your complete satisfaction is important to us. Please contact us with feedback or suggestions to help us improve the quality and usability of our products.

You may reach us at:

**Stealth Products, LLC**  
104 John Kelly Drive, Burnet, TX 78611

Phone: (512) 715-9995

Fax: (512) 715-9954

Toll Free: (800) 965-9229

Toll Free: (800) 806-1225

info@stealthproducts.com

www.stealthproducts.com

<p><b>EC REP</b></p> <p><b>MDSS GmbH</b> Schiffgraben 41 30175 Hannover Germany</p>	<p><b>CH REP</b></p> <p><b>SODIMED SA</b> Chemin Praz Devant 12 1032 Romanel sur-Lausanne Switzerland</p>	<p><b>UK Authorized Representative</b></p> <p><b>MDSS-UK RP</b> 6 Wilmslow Road, Rusholme Manchester M14 5TP United Kingdom</p>
---	---	---

## General

Read and understand all instructions prior to the use of the product. Failure to adhere to instructions and warnings in this document may result in property damage, injury, or death. Product misuse or failure to follow instructions will void the warranty.

Immediately discontinue use if any function is compromised, if parts are missing or loose, or if any component shows signs of excessive wear. Consult with your supplier for repair, adjustment, or replacement.

All persons responsible for fitting, adjustment, and daily use of the devices discussed in these instructions must be familiar with and understand all safety aspects of the devices mentioned. In order for our products to be used successfully, you must read and understand all instructions and warnings, and maintain our products according to our instructions on care and maintenance.

The installation instructions will guide you through this product's options and possibilities. Instructions are written with the expressed intent of use with standard configurations. They also contain important safety and maintenance information, as well as describe possible problems that can arise during use. For further assistance, or more advanced applications, please contact **Stealth Products** at (512) 715-9995 or toll free at (800) 965-9229 or your supplier.

Always keep the operating instructions in a safe place so they may be referenced as necessary. All information, pictures, illustrations, and specifications are based on the product information that was available at the time of printing. Pictures and illustrations shown in these instructions are representative examples and are not intended to be exact depictions of the various parts of the product.

 **CAUTION**

These products are designed to be fitted, applied, and installed exclusively by a healthcare professional trained for these purposes. The fitting, application, and installation by a non-qualified individual could result in serious injury.

## Warranty

Our products are designed, manufactured, and produced to the highest of standards. If any defect in material or workmanship is found, **Stealth Products** will repair or replace the product at our discretion. Any implied warranty, including the implied warranties of merchantability and fitness for a particular purpose, shall not extend beyond the duration of this warranty. **Stealth Products** does not warrant damage due to, but not limited to: Misuse, abuse, or misapplication of product, and/or modification of product without written approval from **Stealth Products, LLC**. Any alteration or lack of serial number, where applicable, will automatically void all warranty.

**Stealth Products, LLC** is liable for replacement parts only. **Stealth Products, LLC** is not liable for any incurred labor costs.

**Stealth Products** warrants against failure due to defective materials or workmanship:

**Covers: 2 years**  
**Hardware: 5 years**  
**Electronics: 3 years**

In the event of a product failure covered by our warranty, please follow the procedures outlined below:

Call **Stealth Products** at (512) 715-9995 or toll free at (800) 965-9229.

Request a Return Authorization (RA) form from the Returns Department and follow the documentation instructions.

You can download additional copies of this manual by accessing the Stealth website ([www.stlpro.site/stealth-docs](http://www.stlpro.site/stealth-docs)) and searching "UniLink Laterals" in the search bar at the top of the page.

## Supplier Reference

<b>Supplier:</b>	
<b>Telephone:</b>	
<b>Address:</b>	
<b>Purchase Date:</b>	
<b>Model:</b>	

## Warning Labels

Warnings are included for the safety of the user, client, operator, and property. Please read and understand what the signal words **DANGER**, **WARNING**, **CAUTION**, **NOTICE**, and **SAFETY** mean, and how they could affect the user, those around the user, and property.

### **DANGER**

Identifies an **imminent** situation which, if not avoided, may result in **severe injury, death, and property damage**.

### **WARNING**

Identifies a **potential** situation which, if not avoided, may result in **severe injury, death, and property damage**.

### **CAUTION**

Identifies a **potential** situation which, if not avoided, may result in **minor to moderate injury and property damage**.

### **NOTICE**

Identifies important information not related to injury, but possible **property damage**.

### **SAFETY**

Indicates steps or instructions for safe practices, reminders of safe procedures, or important safety equipment that may be necessary.

## Limited Liability

**Stealth Products, LLC** accepts no liability for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendations, warnings, and instructions in this manual.

**Stealth Products** does not hold responsibility for final integration of final assembly of product to end user. **Stealth Products** is not liable for user death or injury.

## Testing

Initial setup and driving should be done in an open area free of obstacles until the user is fully capable of driving safely.

## Intended Use

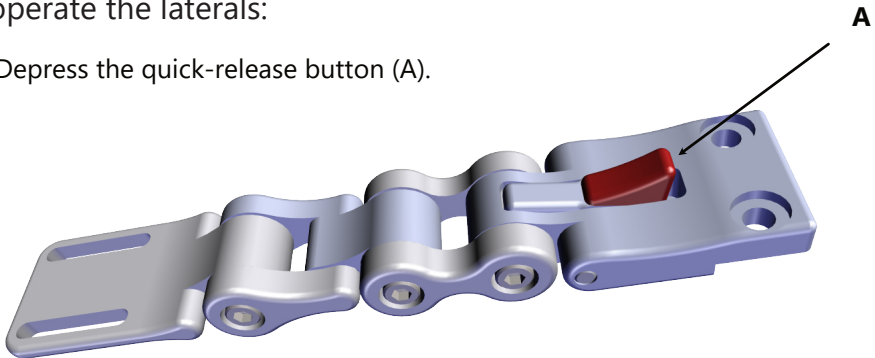
Trunk stability is essential to balancing the body in relation to gravity. The long-term goals of improved posture are to provide pressure relief, minimize orthopedic consequences, normalize muscle tone, reduce active muscle requirements, and increase the overall function of the user.

UniLink™ trunk support solutions are highly adjustable thoracic lateral supports that increase a user's stability and balance while in the chair, serve as tactile reminders of positioning, and correct lateral trunk flexion. They provide a precise, durable means of maintaining the user in a functional position, preventing further postural decay, and diminishing the need for repetitive positioning of the user throughout their day.


## Adjusting Adductors

To operate the laterals:

1. Depress the quick-release button (A).

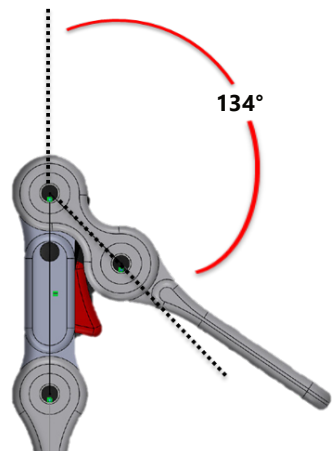2. The lateral will swing free through 134° on either side of the initial locked position.
3. The lateral will only lock in place when returned to its initial position.


## Operating Laterals

To adjust links (B) and (C):

1. Slightly loosen the M5 x .8 x 25mm Socket Head Screws at (B) and (C).
2. Set desired angles and re-tighten screws with a 4mm Hex Wrench.

To adjust pad position:

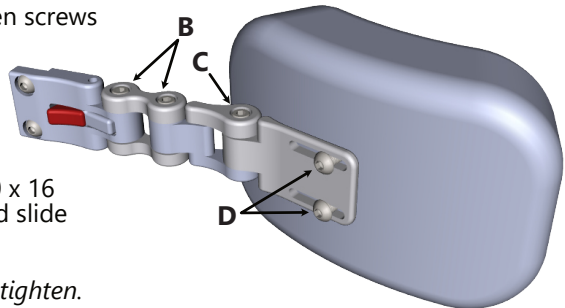
1. Slightly loosen the two M6 x 1.0 x 16 mm Button Head Screws (D) and slide the pad to the desired position.
2. Re-tighten screws. *Do not over-tighten.*

## Standard Track Mounting

To install and adjust standard track mounting:

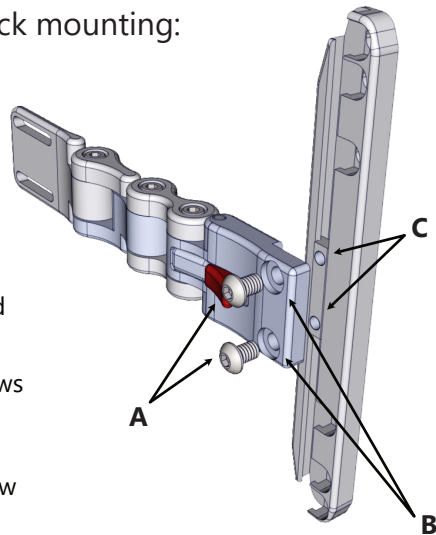1. Insert the M6 x 1.0 x12mm Button Head Screws (A) through the counter bored holes (B) and into the threaded holes of the Universal Track Nut (C).
2. Set desired position and tighten screws with a 4mm Hex Wrench.
3. To adjust the lateral on the track, loosen the screws, slide to the new position. Re-tighten the screws.


### CAUTION

The laterals should only be installed and/or adjusted by a qualified healthcare professional.

### Standard Cane Mounting

To install and adjust standard cane mounting:

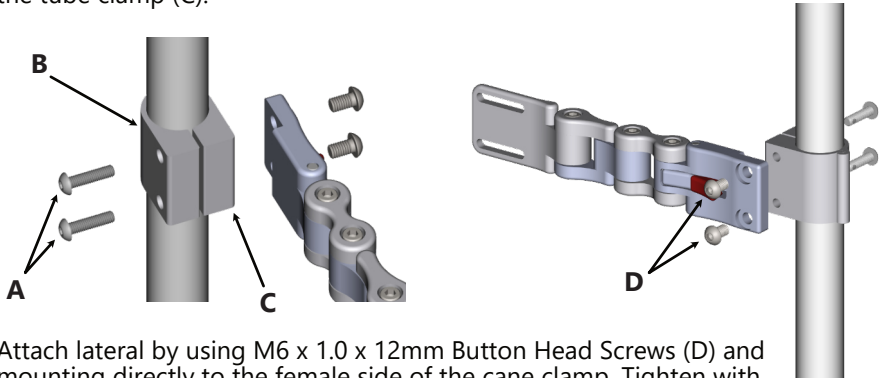1. Insert the M6 x 1.0 x 20mm Button Head Screws (A) into the male tube clamp (B) through the counter bored holes and into the female side of the tube clamp (C).


2. Attach lateral by using M6 x 1.0 x 12mm Button Head Screws (D) and mounting directly to the female side of the cane clamp. Tighten with an M4 Hex Wrench.
3. To adjust the lateral on the cane, loosen the M6 x 1.0 x 20mm Button Head Screws (A) and slide the clamp to desired position. Tighten the screws.

### All Season Mounting

To install and adjust the All Season mounts:

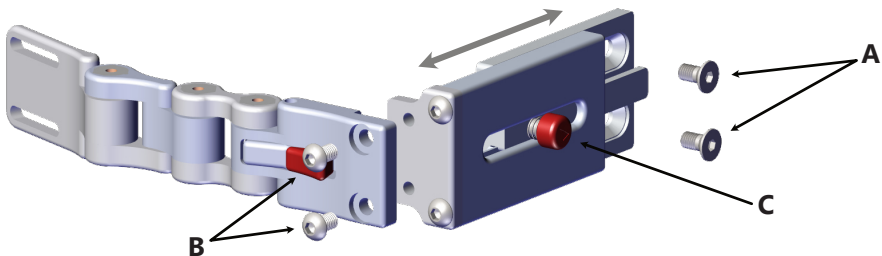
1. Attach the All Season bracket to the chair by using the M6 x 1.0 x 12mm Flat Head Screws (A). Tighten with a 4mm Hex Wrench.


2. Use the M6 x 1.0 x 12mm Button Head Screws (B) to directly mount the lateral to the all-season bracket. Tighten with a 4mm Hex Wrench.
3. Depress the red button (C) and slide the mount to make lateral adjustments in half-inch increments.

## 135° Bracket Installation (BAXX Active)

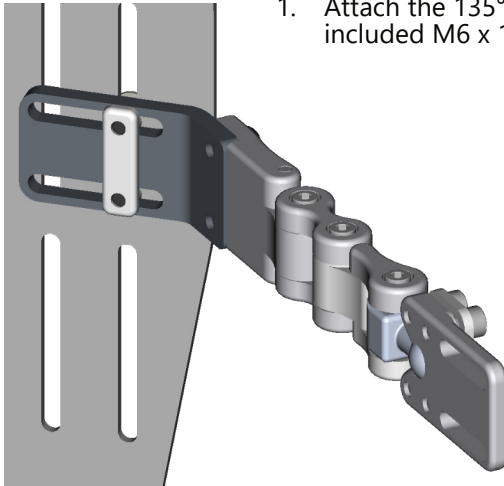
1. Attach the 135° Bracket to the lateral using the included M6 x 1.0 x 10mm button head screws (A).

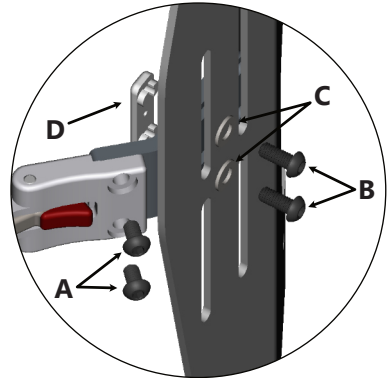2. Position the lateral on the inside of the back support and place the Lateral Track Nut (D) in the slots on the 135° Bracket.
3. Insert the M6 x 1.0 x 14mm Button Head Screws (B) through M6 Washers (C) and into the Lateral Track Nut (D).
4. Tighten the screws to secure the lateral in the desired position on the back support.


## Pad Mount Installation

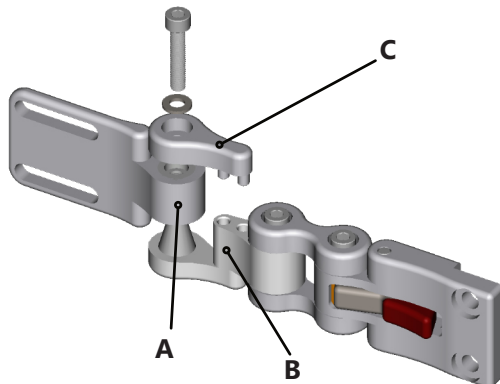1. Place the conical socket of the Pad Mount (A) onto the conical mount of the 1.5" Link (B).
2. Place the 1.5" Coupler (C) onto the 1.5" Link (B) as shown below and secure it with an M5 x 0.8 x 25mm Socket Head Screw and M5 Flat Washer.
3. Adjust the angle of the Pad Mount and tighten the M5 x 0.8 x 25mm Socket Head Screw.


### Full-Surface Contact Mount Installation

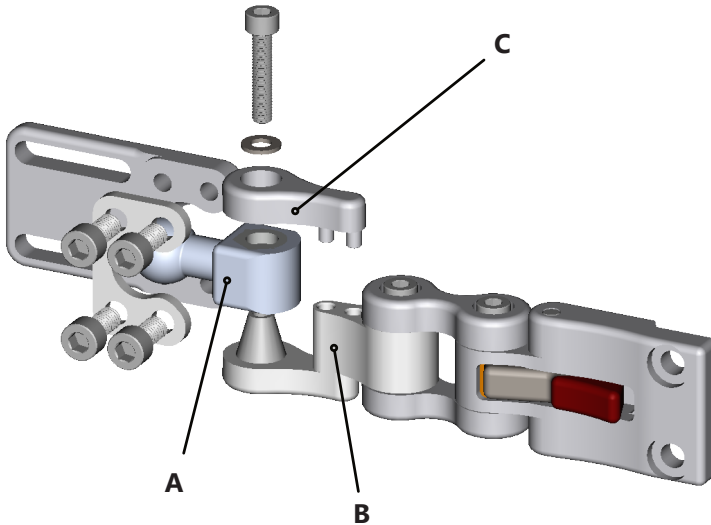1. Place the conical socket of the 17mm Ball Link (A) onto the conical mount of the 1.5" Link (B).
2. Place the 1.5" Coupler (C) onto the 1.5" Link (B) as shown below and secure it with an M5 Flat Washer and M5 x 0.8 x 25mm Socket Head Screw.

### Full-Surface Contact Mount Adjustment

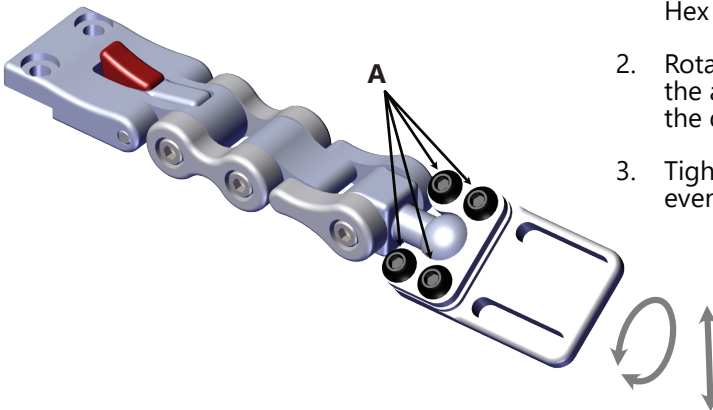1. Loosen the M6 x 1.0 x 12mm Button Head Screws (A) with a 4mm Hex Wrench.
2. Rotate and/or adjust the angle of the pad to the desired position.
3. Tighten the screws evenly.

## Care & Cleaning

Periodically check the hardware for loose screws or worn parts. Check the pad for any foam breakdown and/or metal coming through the foam, which could be potentially dangerous for the client. To wash the cover, remove it from the pad and lateral hardware. Machine wash in cold water, delicate cycle, and drip dry.

For minor stains, scrub aluminum with water and use a brush or sponge. For more serious stains, use an abrasive sponge with a gentle household cleaner or glass cleaner.

## Dealer Assistance

During first-time use by the user, it is advised that the dealer or service technician not only assembles the product, but also explains the configuration of user positioning to the user (i.e., the user and or the attendant). If needed, the dealer can make final adjustments.

## User Testing

It is important that the customer is fully aware of the installation of the **UniLink Lateral**, how to use them, and how they can be adjusted to fit the client comfortably. As a dealer, explain and show the customer how you have executed the installation and explain the function.

Have the user test the position of the **UniLink Lateral**. Is the hardware in the proper position for the client? Can all controls be operated safely and with minimal effort? If needed, make any adjustments to the positioning. Explain possible problems to the customer and how to address them.

## Conditions of Use

The **UniLink Lateral** is intended for use as installed by the dealer, in accordance with the installation instructions in this manual.

Ensure the foreseen conditions of use are communicated by the dealer or service technician to the user and/or attendant during the first-time use.

If usage conditions change significantly, please contact your dealer or a qualified service technician to avoid excessive wear and tear, or unintended damage.

## Disposal

For products that contain batteries or electronics, return the product to qualified personnel for disposal. All components of the product should be disposed of properly in accordance with respective local and national environmental regulations.

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**Stealth Products, LLC**  
104 John Kelly Drive, Burnet, TX 78611

Telefon: (512) 715-9995  
Fax: (512) 715-9954

Zollfrei: (800) 965-9229  
Zollfrei Fax: (800) 806-1225

info@stealthproducts.com  
www.stealthproducts.com

<table border="1" style="margin: auto; border-collapse: collapse;"> <tr> <td style="padding: 2px;">EC</td> <td style="padding: 2px;">REP</td> </tr> </table> <p><b>MDSS GmbH</b> Schiffgraben 41 30175 Hannover Germany</p>	EC	REP	<table border="1" style="margin: auto; border-collapse: collapse;"> <tr> <td style="padding: 2px;">CH</td> <td style="padding: 2px;">REP</td> </tr> </table> <p><b>SODIMED SA</b> Chemin Praz Devant 12 1032 Romanel sur-Lausanne Switzerland</p>	CH	REP	<p>Bevollmächtigter Vertreter in Großbritannien</p> <p><b>MDSS-UK RP</b> 6 Wilmslow Road, Rusholme Manchester M14 5TP United Kingdom</p>
EC	REP					
CH	REP					

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© 2022, Stealth Products, LLC
Stealth Products, LLC · info@stealthproducts.com · www.stealthproducts.com