


Epic Seating System Adapter Tube

Ki FOCUS (ECTUN-003)


This adapter tube kit is intended to securely fit the Epic Seating System to a Ki Focus wheelchair.

Part	Description	Qty
A	Adapter Tube	1
B	Tube Nut	2
C	M6 x 1 x 45mm	3
D	M6 Washer	6
E	M6 Nylon Lock Nut	3

Installation Instructions

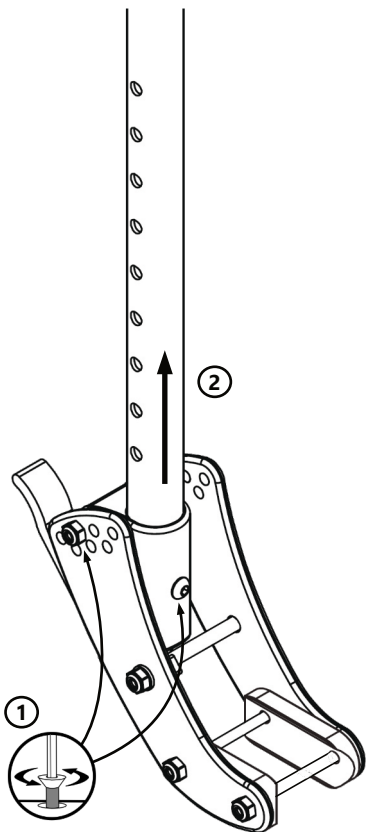

1. Loosen and remove the screw securing the cane to the pivot assembly. If the pivot assembly is set to a recline angle other than 0° , remove the screw and return the pivot assembly to the 0° position.* (First hole in the bottom row.)
2. Remove the cane from the pivot assembly.
3. Insert the ESCTUN-003 adapter tube into the pivot assembly. The heads of the screws installed in the adapter tube should face outwards from the chair, left on the left tube, right on the right.
4. Install the screw removed in Step 1 into the pivot assembly, through the adapter tube, and tighten it to secure the tube in place. Tighten the screw securing the recline angle.
5. Continue the installation as described in the installation section of the Epic Seating Owner's Manual.

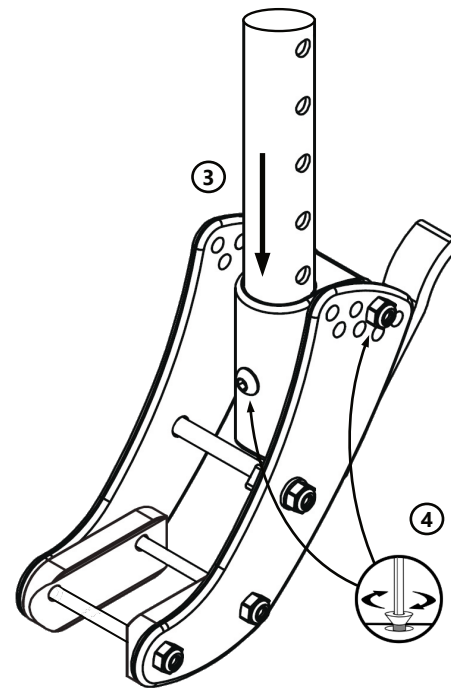

*For more information specific to the Ki Focus, please consult the manual that came with your wheelchair.