


Armrest Bolsters

Intended Use

The Armrest Bolster is a low-profile, covered foam bolster designed to fit over most 10- or 16-inch wheelchair armrests. It provides ample padding, improves posture support and protection from wheelchair components, and increases the overall comfort of the system. The armrest bolster helps to reduce the risk of shoulder subluxation, pain and contractures at the elbow, wrist, and hand. The additional padding helps to protect the upper extremity from injuries, such as skin tears, and helps reduce the risk of tissue breakdown over time. It helps to ensure optimal positioning of the upper extremities for power wheelchair user's access to joysticks or other control devices. The cover material protects against fluid absorption.

The armrest bolster is intended for users who require mild support to appropriately position and maintain their upper extremity on the armrest due to weakness, or for users who require protection due to uncontrolled arm movements. Armrest bolsters should not be used as the only postural intervention for users who exhibit increased tone and/or heavy leaning and should be paired with mounted trunk lateral supports.

Installation and Use

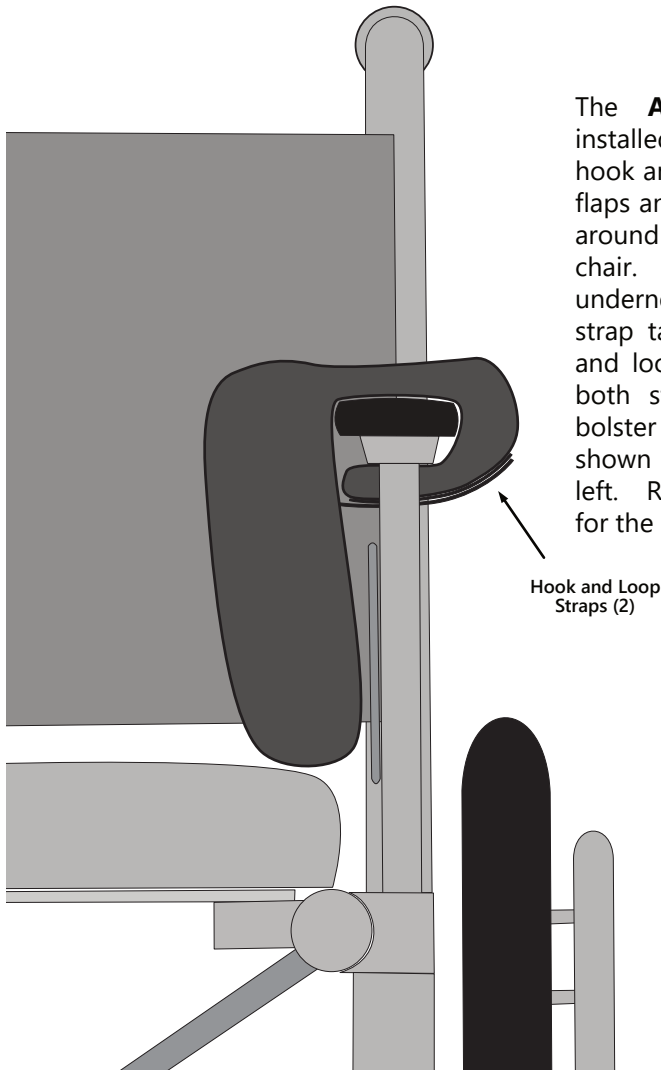

The **Armrest Bolster** is installed by separating the hook and loop straps on the flaps and placing the bolster around the armrests of the chair. Fold the bottom flap underneath and pull the loop strap taut. Press the hook and loop sides together on both straps to secure the bolster to the armrest, as shown in the image to the left. Repeat the procedure for the right armrest.

NOTICE: The armrest bolster should not be used as the only postural intervention for users who exhibit increased tone and/or heavy leaning and should be paired with mounted trunk lateral supports.

Care and Cleaning

We recommend that all products be examined on a regular basis for wear and function. Prior to use, check the foam for breakdown, the cover material and stitching on the webbing for stress or damage, and that hook-and-loop fasteners are securely fastened. If loose, frayed or broken cover material, webbing, or foam are discovered, discontinue use and contact your provider. Repairs should only be performed by a wheelchair service professional and with genuine Stealth Products parts, fasteners or items appropriate for the service required.

Surfaces may be cleaned of dust and dirt with a damp cloth. Ensure all cleaners are approved for finished steel, aluminum, plastic and upholstered surfaces. Cover may be removed for cleaning. Care should be taken not to damage foam during cover removal.

Important

Read and understand all instructions prior to the use of the product. Failure to adhere to instructions and warnings in this document may result in property damage, injury, or death. Product misuse or failure to follow instructions will void the warranty.

Immediately discontinue use if any function is compromised, if parts are missing or loose, or if any component shows signs of excessive wear. Consult with your supplier for repair, adjustment, or replacement.

All persons responsible for fitting, adjustment, and daily use of the devices discussed in these instructions must be familiar with and understand all safety aspects of the devices mentioned. In order for our products to be used successfully, you must read and understand all instructions and warnings, and maintain our products according to our instructions and warnings, and maintain our products according to our instructions on care and maintenance.

The installation instructions will guide you through this products options and possibilities.

Instructions are written with the expressed intent of use with standard configurations. They also contain important safety and maintenance information, as well as describe possible problems that can arise during use. For further assistance, or more advanced applications, please contact your supplier or Stealth Products at (512) 715-9995 or toll-free at (800) 965-9229.

Always keep the operating instructions in a safe place so they may be referenced as necessary.

All information, pictures, illustrations, and specifications are based on the product information that was available at the time of printing. Pictures and illustrations shown in these instructions are representative examples and are not intended to be exact depictions of the various parts of the product.

Warranty

Our products are designed, manufactured, and produced to the highest of standards. If any defect in material or workmanship is found, **Stealth Products** will repair or replace the product at our discretion. Any implied warranty, including the implied warranties of merchantability and fitness for a particular purpose, shall not extend beyond the duration of this warranty. **Stealth Products, LLC** does not warrant damage due to, but not limited to, misuse, abuse, or misapplication of products and modification of product without written approval from **Stealth Products, LLC**.

Any alteration or lack of serial number, where applicable, will automatically void this warranty. **Stealth Products, LLC** is liable for replacement parts only. **Stealth Products, LLC** is not liable for any incurred labor costs. No person is authorized to alter, extend, or waive the warranties of **Stealth Products, LLC**.

To learn more about our warranty and returns policy, please visit <https://stlpro.site/returns-n-warranty>.

About Us

Stealth Products is committed to 100% customer satisfaction. Your complete satisfaction is important to us. Please contact us with suggestions or feedback to help us improve the quality and usability of our products.

You may reach us at:

 **Stealth Products, LLC**
104 John Kelly Drive,
Burnet, TX 78611

Phone: (512) 715-9995 Toll Free: (800) 965-9229
Fax: (512) 715-9954 Toll Free: (800) 806-1225
info@stealthproducts.com - www.stealthproducts.com



EC REP MDSS GmbH Schiffgraben 41 30175 Hannover, Germany	CH REP SODIMED SA Chemin Praz Devant 12 1032 Romanel sur-Lausanne, Switzerland	UK Authorized Representative MDSS-UK RP 6 Wilmslow Road, Rusholme Manchester M14 5TP, United Kingdom
---	---	---

Warranty Information



Home Page



Online User Manual

