

Stealth Products[®]



**Lower
Extremities**



Calf Support Covers

Owner's Manual - Maintenance Guide

Calf Support Covers for Quantum

Table of Contents

| | |
|--|---------------|
| Customer Satisfaction | i |
| Important Information | ii-iii |
| Warranty | ii |
| Limited Liability..... | ii |
| Testing | ii |
| Warning Labels..... | iii |
| Intended Use..... | iii |
| Installation Instructions | 1 |
| Installation | 1 |
| First-Time Use | 2 |
| Dealer Assistance | 2 |
| User Testing..... | 2 |
| Conditions of Use | 2 |
| Cleaning..... | 2 |

CAUTION

These products are designed to be fitted, applied, and installed exclusively by a healthcare professional trained for these purposes. The fitting, application, and installation by a non-qualified individual could result in serious injury.

Supplier Reference

| | |
|-----------------------|--|
| Supplier: | |
| Telephone: | |
| Address: | |
| | |
| Purchase Date: | |
| Model: | |

Stealth Products is committed to 100% customer satisfaction. Your complete satisfaction is important to us. Please contact us with feedback or suggestions to help us improve the quality and usability of our products.

You may reach us at:



Stealth Products, LLC
104 John Kelly Drive, Burnet, TX 78611



Phone: (512) 715-9995
Fax: (512) 715-9954

Toll Free: (800) 965-9229
Toll Free: (800) 806-1225

info@stealthproducts.com
www.stealthproducts.com

| | | |
|---|---|---|
| <p>EC REP</p> <p>MDSS GmbH Schiffgraben 41 30175 Hannover Germany</p> | <p>CH REP</p> <p>SODIMED SA Chemin Praz Devant 12 1032 Romanel sur-Lausanne Switzerland</p> | <p>UK Authorized Representative</p> <p>MDSS-UK RP 6 Wilmslow Road, Rusholme Manchester M14 5TP United Kingdom</p> |
|---|---|---|

General

Read and understand all instructions prior to the use of the product. Failure to adhere to instructions and warnings in this document may result in property damage, injury, or death. Product misuse or failure to follow instructions will void the warranty.

Immediately discontinue use if any function is compromised, if parts are missing or loose, or if any component shows signs of excessive wear. Consult with your supplier for repair, adjustment, or replacement.

All persons responsible for fitting, adjustment, and daily use of the devices discussed in the these instructions must be familiar with and understand all safety aspects of the devices mentioned. In order for our products to be used successfully, you must read and understand all instructions and warnings, and maintain our products according to our instructions on care and maintenance.

The installation instructions will guide you through this product's options and possibilities.

Instructions are written with the expressed intent of use with standard configurations. They also contain important safety and maintenance information, as well as describe possible problems that can arise during use. For further assistance, or more advanced applications, please contact your supplier or **Stealth Products** at (512) 715-9995 or toll free at (800) 965-9229.

Always keep the operating instructions in a safe place so they may be referenced as necessary.

All information, pictures, illustrations, and specifications are based on the product information that was available at the time of printing. Pictures and illustrations shown in these instructions are representative examples and are not intended to be exact depictions of the various parts of the product.

Warranty

Our products are designed, manufactured, and produced to the highest of standards. If any defect in material or workmanship is found, **Stealth Products** will repair or replace the product at our discretion. Any implied warranty, including the implied warranties of merchantability and fitness for a particular purpose, shall not extend beyond the duration of this warranty. **Stealth Products** does not warrant damage due to, but not limited to: misuse, abuse, or misapplication of product, and/or modification of product without written approval from **Stealth Products, LLC**. Any alteration or lack of serial number, where applicable, will automatically void all warranty.

Stealth Products, LLC is liable for replacement parts only. **Stealth Products, LLC** is not liable for any incurred labor costs. **Stealth Products** warrants against failure due to defective materials or workmanship:

Covers: 2 years

Hardware: 5 years

Electronics: 3 years

In the event of a product failure covered by our warranty, please follow the procedures outlined below:

Call **Stealth Products** at (512) 715-9995 or toll free at (800) 965-9229.

Request a Return Authorization (RA) form from the Returns Department and follow the documentation instructions.

Additional warranty information may be found by visiting the website at <https://stlpro.site/returns-n-warranty>. Copies of this manual may be downloaded by accessing the Stealth website (<https://stlpro.site/stealth-docs>) and searching "Calf Support Covers Owners Manual" in the search bar at the top of the page.

Limited Liability

Stealth Products, LLC accepts no liability for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendations, warnings, and instructions in this manual.

Stealth Products does not hold responsibility for final integration of final assembly of product to end user. **Stealth Products** is not liable for user death or injury.

Testing

Initial setup and driving should be done in an open area free of obstacles until the user is fully capable of driving safely.

Warning Labels

Warnings are included for the safety of the user, client, operator, and property. Please read and understand what the signal words **DANGER**, **WARNING**, **CAUTION**, **NOTICE**, and **SAFETY** mean, and how they could affect the user, those around the user, and property.



DANGER

Identifies an **imminent** situation which, if not avoided, may result in **severe injury, death, and property damage**.



WARNING

Identifies a **potential** situation which, if not avoided, may result in **severe injury, death, and property damage**.



CAUTION

Identifies a **potential** situation which, if not avoided, may result in **minor to moderate injury and property damage**.

NOTICE

Identifies important information not related to injury, but possible **property damage**.

SAFETY

Indicates steps or instructions for safe practices, reminders of safe procedures, or important safety equipment that may be necessary.

Intended Use


The **Calf Support Covers** are intended to provide comfort and protection by adding a soft cover filled with open-cell foam over the standard molded calf supports. The pads have removable covers for easy cleaning.

Installation

1.



2.

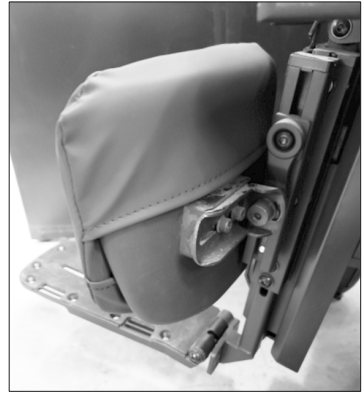


1. Align the shape of the calf support pad with the existing molded calf support. Place the top flap of the calf support cover onto the existing plastic calf support.
2. Slide the calf support cover down until the top flap is taut against the calf support.

3.



4.

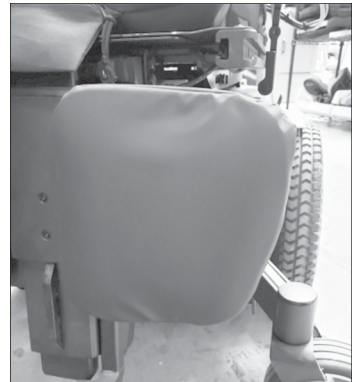

3. Pull the bottom flap down and under the bottom edge of the calf support.
4. Smooth out all wrinkles on the contact surface.

NOTICE

The Calf Support Covers are sized to fit the calf supports. Make sure to install them on the proper side and in the correct orientation.

Dealer Assistance

During first time use by the client, it is advised that the dealer or service technician assists and explains the configuration to the customer (the user and/or the attendant). If needed the dealer can make final adjustments.

User Testing

It is important that the customer is fully aware of the installation and what or how the product can be adjusted. If needed, adjust the **Calf Support Covers** so they function to the user's needs. Explain to the customer possible problems that may arise, and how to address them.

Conditions of Use

The **Calf Support Covers** are intended for use as installed by the dealer, in accordance with the installation instructions in this manual.

- The foreseen conditions of use are communicated by the dealer or service technician to the user and/or attendant during the first time use.
- If the usage conditions change significantly, please contact your dealer or service technician to avoid excessive wear and tear or unintended damage.

Cleaning

The **Calf Support Covers** may be wiped clean with a damp cloth during normal cleaning. Should they need to be washed, follow the guidelines below:

- Remove the Calf Support Covers from the calf supports.
- Remove the foam pad from the cover prior to washing. Do not get the foam pad wet.
- Machine wash the cover in cold water, delicate cycle with like colors. Do not use bleach or fabric softener. Do not dry clean the covers.
- Tumble dry on low heat. Remove promptly.
- Replace the foam pads inside the covers and install again as shown on the Installation page.



© 2023, Stealth Products, LLC

Warranty Information



Home Page



Online User Manual



Stealth Products, LLC · info@stealthproducts.com · www.stealthproducts.com
(800) 965-9229 | (512) 715-9995 | 104 John Kelly Drive, Burnet, TX 78611