

Stealth Products[®]



STEALTH POSITIONING



Cam Lock

Owner's Manual - Maintenance Guide

Stealth Positioning Cam Lock

Customer Satisfaction

Stealth Products is committed to 100% customer satisfaction. Your complete satisfaction is important to us. Please contact us with feedback or suggestions to help us improve the quality and usability of our products.

You may reach us at:

Stealth Products, LLC

104 John Kelly Drive, Burnet, TX 78611



Phone: (512) 715-9995

Fax: (512) 715-9954

Toll Free: (800) 965-9229

Toll Free: (800) 806-1225

info@stealthproducts.com

www.stealthproducts.com

<div><div>EU</div><div>REP</div></div> <div></div> <div>PRIDE MOBILITY PRODUCTS EUROPE BV De Zwaan 3 1601 MS Enkhuizen The Netherlands</div>	<div><div>CH</div><div>REP</div></div> <div>SODIMED SA Chemin Praz Devant 12 1032 Romanel sur-Lausanne Switzerland</div>	<div>UK Authorized Representative</div> <div>PRIDE MOBILITY PRODUCTS LTD. 32 Wedgwood Road, Bicester, Oxfordshire OX26 4UL United Kingdom</div> <div></div>
--	---	---

General

Read and understand all instructions prior to the use of the product. Failure to adhere to instructions and warnings in this document may result in property damage, injury, or death. Product misuse or failure to follow instructions will void the warranty.

Immediately discontinue use if any function is compromised, if parts are missing or loose, or if any component shows signs of excessive wear. Consult with your supplier for repair, adjustment, or replacement.

All persons responsible for fitting, adjustment, and daily use of the devices discussed in these instructions must be familiar with and understand all safety aspects of the devices mentioned. In order for our products to be used successfully, you must read and understand all instructions and warnings, and maintain our products according to our instructions on care and maintenance.

The installation instructions will guide you through this product's options and possibilities.

Instructions are written with the expressed intent of use with standard configurations. They also contain important safety and maintenance information, as well as describe possible problems that can arise during use. For further assistance, or more advanced applications, please contact your supplier or **Stealth Products** at (512) 715-9995 or toll free at (800) 965-9229.


Always keep the operating instructions in a safe place so they may be referenced as necessary.

All information, pictures, illustrations, and specifications are based on the product information that was available at the time of printing. Pictures and illustrations shown in these instructions are representative examples and are not intended to be exact depictions of the various parts of the product.

The Stealth Positioning Cam Lock is intended to provide a mechanically stable and secure connection point for support devices using 1", 1.5" or 2" webbing straps.

Installing the Strap

If the latch is closed, use a screwdriver or other similar tool to release it. With the latch open, slide the strap under the latch, around the serrated rod inside, and back under the latch. Properly adjust the tension of the strap and firmly press the latch to the closed position, where it will lock into place when fully closed. It will hold the strap securely in position and will require a screwdriver or other similar tool to open again.

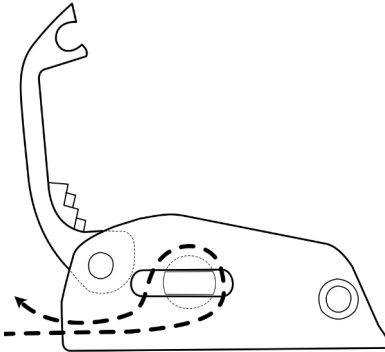

- 1. Lift latch.**
- 2. Thread strap.**
- 3. Close latch.**

Adjusting the Strap

Using a screwdriver, or other similar tool, pry the latch until it snaps open. The strap should loosen and can be adjusted. When satisfied with the adjustment, close the latch until it clicks, securing the strap in the new position.

Installing the Cam Lock

The Cam Lock is mounted directly to a chair rail or tube. Using a flat head screwdriver, or other suitable prying device, open the latch. Use a button head screw of sufficient size to fasten the Cam Lock to the mount point on the chair, ensuring that the latch is clear to close fully once installed.



© 2025, Stealth Products, LLC

Warranty Information



Home Page



Online User Manual



Stealth Products, LLC · info@stealthproducts.com · www.stealthproducts.com
(800) 965-9229 | (512) 715-9995 | 104 John Kelly Drive, Burnet, TX 78611