


Stealth Products
®



**Backs &
Support**

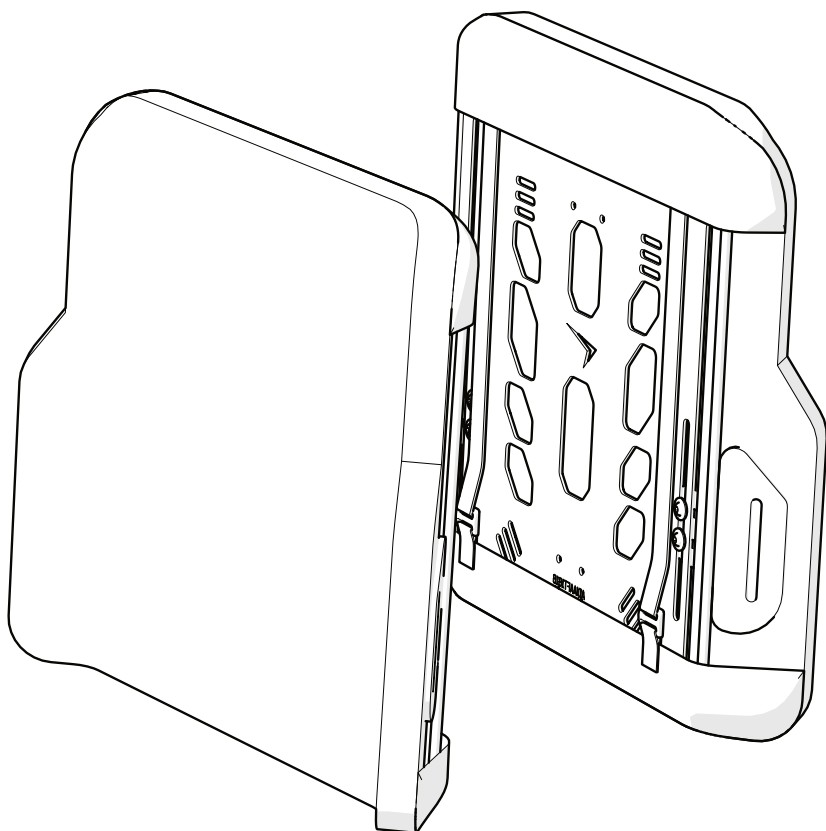

ADI Active Posture Back Support

Owner's Manual - Maintenance Guide
ADI Active Posture Back Support



Table of Contents

Customer Satisfaction i

Important Information ii-iii

 Warranty ii

 Supplier Reference..... ii

 Warning Labels..... iii

 Limited Liability..... iii

 Testing iii

Design and Function 1

 Tools Required for Installation and Adjustment 1

 Parts List..... 1

 Intended Use..... 1

Installation Instructions 2

 Installation 2

 Adjustment..... 2

 Testing 2

First-Time Use 3

 Dealer Assistance 3

 User Assistance 3

 Conditions of Use 3

Care and Maintenance 4

 Cleaning..... 4

 Disinfection..... 4

Notes 5-6

Stealth Products is committed to 100% customer satisfaction. Your complete satisfaction is important to us. Please contact us with feedback or suggestions to help us improve the quality and usability of our products.

You may reach us at:

Stealth Products, LLC
104 John Kelly Drive, Burnet, TX 78611



Phone: (512) 715-9995
Fax: (512) 715-9954

Toll Free: (800) 965-9229
Toll Free: (800) 806-1225

info@stealthproducts.com
www.stealthproducts.com

<div><div>EU</div><div>REP</div></div> <div></div> <div>PRIDE MOBILITY PRODUCTS EUROPE BV De Zwaan 3 1601 MS Enkhuizen The Netherlands</div>	<div><div>CH</div><div>REP</div></div> <div>SODIMED SA Chemin Praz Devant 12 1032 Romanel sur-Lausanne Switzerland</div>	<div>UK Authorized Representative</div> <div>PRIDE MOBILITY PRODUCTS LTD. 32 Wedgwood Road, Bicester, Oxfordshire OX26 4UL United Kingdom</div> <div></div>
--	---	---

General

Read and understand all instructions prior to the use of the product. Failure to adhere to instructions and warnings in this document may result in property damage, injury, or death. Product misuse or failure to follow instructions will void the warranty. Immediately discontinue use if any function is compromised, if parts are missing or loose, or if any component shows signs of excessive wear. Consult with your supplier for repair, adjustment, or replacement.

All persons responsible for fitting, adjustment, and daily use of the devices discussed in these instructions must be familiar with and understand all safety aspects of the devices mentioned. In order for our products to be used successfully, you must read and understand all instructions and warnings, and maintain our products according to our instructions on care and maintenance.

The installation instructions will guide you through this product’s options and possibilities.

Instructions are written with the expressed intent of use with standard configurations. They also contain important safety and maintenance information, as well as describe possible problems that can arise during use. For further assistance, or more advanced applications, please contact your supplier or **Stealth Products** at (512) 715-9995 or toll free at (800) 965-9229.

Always keep the operating instructions in a safe place so they may be referenced as necessary.

All information, pictures, illustrations, and specifications are based on the product information that was available at the time of printing. Pictures and illustrations shown in these instructions are representative examples and are not intended to be exact depictions of the various parts of the product.

Important Information



CAUTION

These products are designed to be fitted, applied, and installed exclusively by a healthcare professional trained for these purposes. The fitting, application, and installation by a non-qualified individual could result in serious injury.

Warranty

Our products are designed, manufactured, and produced to the highest of standards. If any defect in material or workmanship is found, Stealth Products will repair or replace the product at our discretion. Any implied warranty, including the implied warranties of merchantability and fitness for a particular purpose, shall not extend beyond the duration of this warranty. **Stealth Products** does not warrant damage due to, but not limited to: misuse, abuse, or misapplication of product, and/or modification of product without written approval from **Stealth Products, LLC**. Any alteration or lack of serial number, where applicable, will automatically void all warranty.

Stealth Products, LLC is liable for replacement parts only. **Stealth Products, LLC** is not liable for any incurred labor costs.

Stealth Products warrants against failure due to defective materials or workmanship:

Covers: 2 years
Hardware: 5 years
Electronics: 3 years

NOTICE

Any substitutions of supplied hardware void the warranty. Misuse of items and "acts of nature" (e.g., flood, tornado) are not covered by the warranty.

In the event of a product failure covered by our warranty, please follow the procedures outlined below:

Call **Stealth Products** at (512) 715-9995 or toll free at (800) 965-9229.

Request a Return Authorization (RA) form from the Returns Department and follow the documentation instructions.

Additional warranty information may be found by visiting the website at <https://stlpro.site/returns-n-warranty>. Copies of this manual may be downloaded by accessing the Stealth website (<https://stlpro.site/stealth-docs>) and searching "P116D838" in the search bar at the top of the page.

Supplier Reference

Supplier:	
Telephone:	
Address:	
Purchase Date:	
Model:	

Warning Labels

Warnings are included for the safety of the user, client, operator, and property. Please read and understand what the signal words **DANGER**, **WARNING**, **CAUTION**, **NOTICE**, and **SAFETY** mean, and how they could affect the user, those around the user, and property.



DANGER

Identifies an **imminent** situation which, if not avoided, may result in **severe injury, death, and property damage**.



WARNING

Identifies a **potential** situation which, if not avoided, may result in **severe injury, death, and property damage**.



CAUTION

Identifies a **potential** situation which, if not avoided, may result in **minor to moderate injury and property damage**.

NOTICE

Identifies important information not related to injury, but possible **property damage**.

SAFETY

Indicates steps or instructions for safe practices, reminders of safe procedures, or important safety equipment that may be necessary.

Limited Liability

Stealth Products, LLC accepts no liability for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendations, warnings, and instructions in this manual.

Stealth Products does not hold responsibility for final integration of final assembly of product to end user. **Stealth Products** is not liable for user death or injury.

Testing


Initial setup and driving should be done in an open area free of obstacles until the user is fully capable of driving safely.

Tools Required for Installation and Adjustment

4mm Hex Key



Parts List

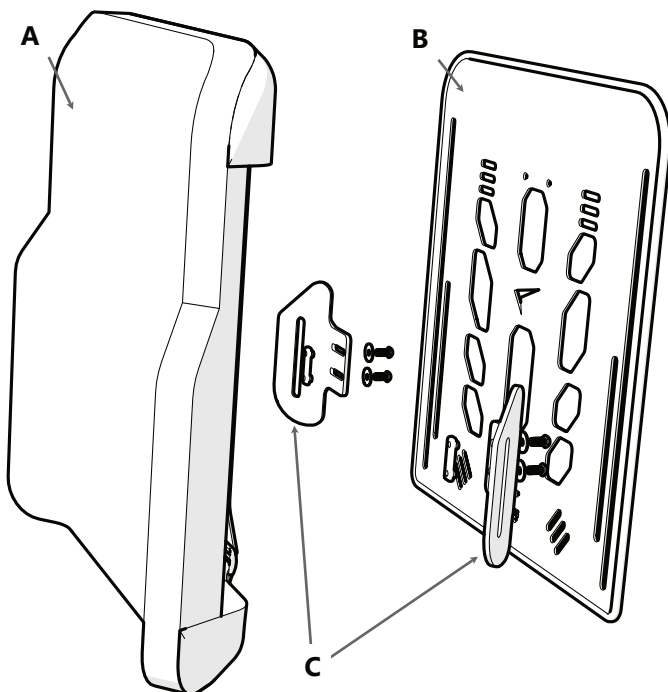

Fabric Cover & Foam Cushion (A)

Back Support Shell* (B)

Wing & Fasteners (C)

Intended Use

The **ADI Active Posture** back support is a lightweight replacement wheelchair back support intended to provide posterior trunk and pelvic support and positioning. It contains mild contouring to help facilitate upright sitting posture and act as a deterrent to progression of negative postural tendencies. An experienced seating and positioning clinician should identify appropriate solutions for the end user's seating and positioning needs within the wheelchair.

NOTICE

Use only the hardware supplied with the **ADI Active Posture** back support on the **ADI Active Posture** back support. Use of any other screws, brackets, clamps, alternative hardware or mounting brackets will void the warranty.

Installation

The installation instructions included with the mounting hardware will cover the installation of your **ADI Active Posture** back support on your wheelchair.

Adjustment

The support wings can be adjusted higher or lower to accommodate user needs by loosening the screws securing them to the back shell and sliding them into the new position. Tighten the screws to secure the wing in the new position.

The adjustment instructions included in your mounting hardware manual will detail the height, depth, and angle adjustments that can be made with your **ADI Active Posture** back support.

Testing

After installation of the **ADI Active Posture** back support, execute the following test(s) before the wheelchair is delivered or put into service.

- Test Drive

Test Drive

- Perform a test drive with the wheelchair
- Check that the **ADI Active Posture** back support is firmly attached to the chair and that no part of it is loose.
- Ensure the cushion is centered on the shell and that no metal surface can contact the user.



WARNING

Protect the back, and contour or lateral supports, if included, from blunt forces that may cause damage and/or breakage.



CAUTION

The back should be in the folded position, if possible, when transporting the wheelchair, especially before loading for air travel.

Dealer Assistance

During first-time use by the client, it is advised that the dealer or service technician assist and explain the setup to the client (i.e., the user and or the attendant). If needed, the dealer can make final adjustments.



CAUTION

Do not attach the **ADI Active Posture** back support with the user in the chair.

User Testing

It is important that the customer is fully aware of the installation, how to use it, and how it can be adjusted in order to gain as much mobility as possible. As a provider, complete the following steps and checks:

- Explain and show the customer how you have executed the installation, and explain the function of the hardware.
- If needed, adjust the hardware to the proper position.
- Explain possible problems to the customer and how to address them.



WARNING

Maximum Weight Capacity

300 lbs. (136.08 kg.)

Conditions of Use

The **ADI Active Posture** back support is intended for use as installed by the dealer, in accordance with the installation instructions listed in this manual.

The foreseen conditions of use are communicated by the dealer or service technician to the user/attendant during the first use.

If the usage conditions change significantly, please contact your dealer or a qualified service technician to avoid excessive wear and tear or unintended damage.

Cleaning

- Use a soft cloth to clean the back shell.
- Hand wash the cover with mild, soapy water or detergent and let dry.
- Foam pad should NOT get wet.

**WARNING**

Do not machine wash or dry.
Do not hand wash in water temperatures above 160°F (70°C)
Do not immerse the foam in water.

**CAUTION**

Institutions must clean and disinfect this product
between multiple patient uses.

Disinfection

- Wipe gently with a cloth dampened with a household disinfectant.

SAFETY

Always check all mounting hardware,
making sure each fastener is properly tightened,
prior to using the back support.

NOTICE

Improper installation and/or alteration of an ADI back
in any way will void the warranty.

Notes

[illegible]

[illegible]



© 2025, Stealth Products, LLC

Warranty Information



Home Page



Online User Manual

