

Stealth Products[®]



Accessories



ADI Push and Transfer Gloves

Owner's Manual - Maintenance Guide

ADI Push and Transfer Gloves

Customer Satisfaction

Stealth Products is committed to 100% customer satisfaction. Your complete satisfaction is important to us. Please contact us with feedback or suggestions to help us improve the quality and usability of our products.

You may reach us at:

Stealth Products, LLC

104 John Kelly Drive, Burnet, TX 78611



Phone: (512) 715-9995

Fax: (512) 715-9954

Toll Free: (800) 965-9229

Toll Free: (800) 806-1225

info@stealthproducts.com

www.stealthproducts.com

<div><div>EU</div><div>REP</div></div> <div></div> <div>PRIDE MOBILITY PRODUCTS EUROPE BV De Zwaan 3 1601 MS Enkhuizen The Netherlands</div>	<div><div>CH</div><div>REP</div></div> <div>SODIMED SA Chemin Praz Devant 12 1032 Romanel sur-Lausanne Switzerland</div>	<div>UK Authorized Representative</div> <div>PRIDE MOBILITY PRODUCTS LTD. 32 Wedgwood Road, Bicester, Oxfordshire OX26 4UL United Kingdom</div> <div></div>
--	---	---

General

Read and understand all instructions prior to the use of the product. Failure to adhere to instructions and warnings in this document may result in property damage, injury, or death. Product misuse or failure to follow instructions will void the warranty.

Immediately discontinue use if any function is compromised, if parts are missing or loose, or if any component shows signs of excessive wear. Consult with your supplier for repair, adjustment, or replacement.

All persons responsible for fitting, adjustment, and daily use of the devices discussed in these instructions must be familiar with and understand all safety aspects of the devices mentioned. In order for our products to be used successfully, you must read and understand all instructions and warnings, and maintain our products according to our instructions on care and maintenance.


The installation instructions will guide you through this product's options and possibilities. Instructions are written with the expressed intent of use with standard configurations. They also contain important safety and maintenance information, as well as describe possible problems that can arise during use. For further assistance, or more advanced applications, please contact your supplier or **Stealth Products** at (512) 715-9995 or toll free at (800) 965-9229.

Always keep the operating instructions in a safe place so they may be referenced as necessary.

All information, pictures, illustrations, and specifications are based on the product information that was available at the time of printing. Pictures and illustrations shown in these instructions are representative examples and are not intended to be exact depictions of the various parts of the product.

Design and Function

ADI Push and Transfer Gloves have an open-finger, half-thumb, wrap-around design that makes the gloves easy to put on and take off. Made of leather with padded palms and wrists, the gloves help prevent wear and tear on user's hands throughout a long day of active chair use. Three inches of hook-and-loop closure provide a highly adjustable and secure fit for most hand sizes.

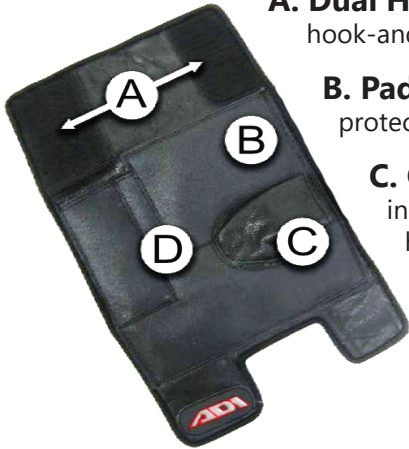

A. Dual Hook-and-Loop: 3 in. (7.62cm) of dual hook-and-loop closure for a secure fit.

B. Padded Palm: Reinforced padding offers extra protection against pressure and abrasion.

C. Open Finger: Half-thumb design enables independent utilization by persons with limited hand function.

D. Grip Options: Leather or rubber palm reinforcement to enhance grip for push, transfer, and braking.

Operation Instructions

To use the Push and Transfer Gloves:

1. Open the glove by separating the hook-and-loop closure to expose the interior of the glove. Insert your thumb into the half-thumb section of the glove.
2. Pull the "loop" flap of the glove around to the back of your hand. Bring the "hook" flap across your wrist and press to secure it. Adjust the fit for comfort.
3. Pull the other "hook" flap across the back of your hand and press it to the "loop" flap. Adjust the fit for comfort.
4. Reverse the procedure to remove the glove.

Notice: One size fits most. Before each use, it is important that you check the **Push and Transfer Gloves** for any damage or wear.

For product information visit: <https://stlpro.site/stealth-docs> and enter "Push and Transfer Gloves" in the search bar.



© 2025, Stealth Products, LLC

Warranty Information



Home Page



Online User Manual



Stealth Products, LLC · info@stealthproducts.com · www.stealthproducts.com
(800) 965-9229 | (512) 715-9995 | 104 John Kelly Drive, Burnet, TX 78611