


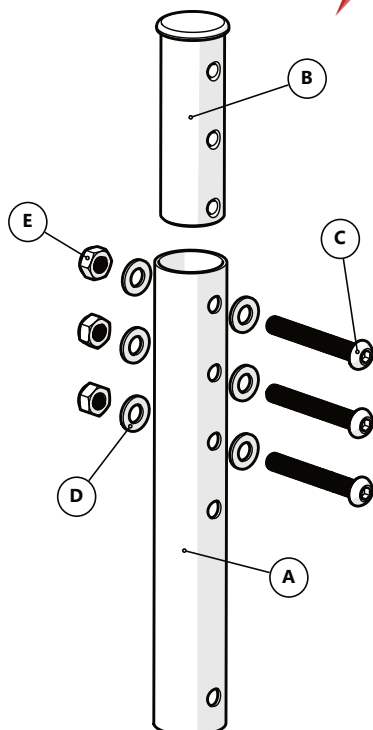

This adapter tube kit is intended to securely fit the Epic Seating System to an Invacare Solara 3G wheelchair.

Part	Description	Qty
A	Adapter Tube	1
B	Tube Nut	1
C	M6 x 1 x 45mm	3
D	M6 Washer	6
E	M6 Nylon Lock Nut	3

EPiC Seating System Adapter Set

Installation Guide

EPiC Seating System Adapter Tube Set Invacare Solara 3G

Customer Satisfaction

Stealth Products is committed to 100% customer satisfaction. Your complete satisfaction is important to us. Please contact us with feedback or suggestions to help us improve the quality and usability of our products.

You may reach us at:

Stealth Products, LLC

104 John Kelly Drive, Burnet, TX 78611



Phone: (512) 715-9995

Fax: (512) 715-9954

Toll Free: (800) 965-9229

Toll Free: (800) 806-1225

info@stealthproducts.com

www.stealthproducts.com

<table><tr><td>EU</td><td>REP</td></tr></table> EMERGO EUROPE BV Westervoortsedijk 60 6827 AT Arnhem The Netherlands	EU	REP	 GELBART AG Tribtschenstrasse 64 CH-6005 Luzern Switzerland	UK Authorized Representative EMERGO CONSULTING (UK) LTD. c/o Cr360 - UL International Compass House, Vision Park Histon Cambridge CB24 9BZ United Kingdom	
EU	REP				

General

Read and understand all instructions prior to the use of the product. Failure to adhere to instructions and warnings in this document may result in property damage, injury, or death. Product misuse or failure to follow instructions will void the warranty.

Immediately discontinue use if any function is compromised, if parts are missing or loose, or if any component shows signs of excessive wear. Consult with your supplier for repair, adjustment, or replacement.

All persons responsible for fitting, adjustment, and daily use of the devices discussed in these instructions must be familiar with and understand all safety aspects of the devices mentioned. In order for our products to be used successfully, you must read and understand all instructions and warnings, and maintain our products according to our instructions on care and maintenance.


The installation instructions will guide you through this product's options and possibilities.

Instructions are written with the expressed intent of use with standard configurations. They also contain important safety and maintenance information, as well as describe possible problems that can arise during use. For further assistance, or more advanced applications, please contact your supplier or **Stealth Products** at (512) 715-9995 or toll free at (800) 965-9229.

Always keep the operating instructions in a safe place so they may be referenced as necessary.

All information, pictures, illustrations, and specifications are based on the product information that was available at the time of printing. Pictures and illustrations shown in these instructions are representative examples and are not intended to be exact depictions of the various parts of the product.

Installation Instructions

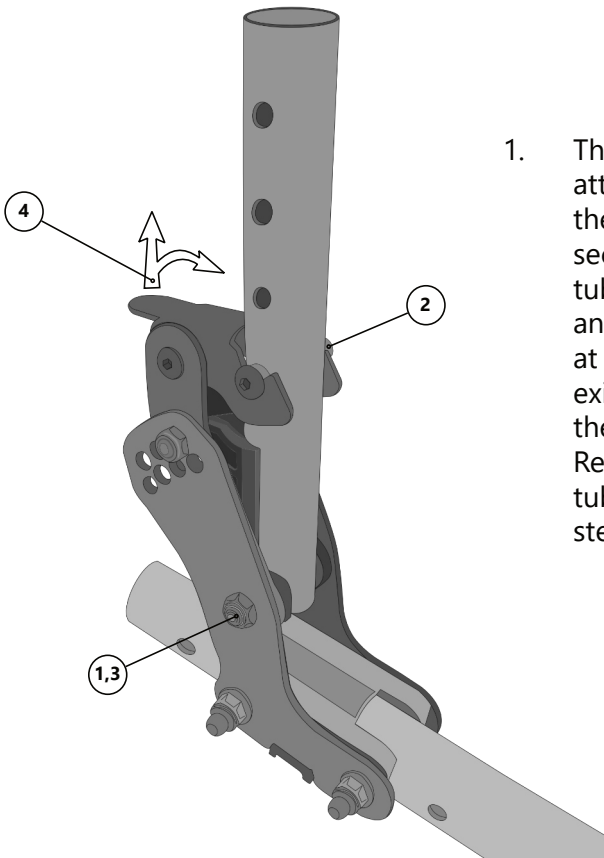

1. The Epic Seating attachment tubes utilize the existing hardware that secures the back support tubes to the chair. Locate and remove the fasteners at the pivot point of the existing tubes and remove them from the chair. Retain the fasteners and tube washers for the next step.

2. Locate and remove the latch bolts from the back support tubes. Reinstall these in the lowest of the four one-inch spaced holes in the adapter tubes.
3. Using the fasteners and tube washers removed in Step One, install the Epic Seating adapter tubes to the existing chair hardware.
4. The tube should tilt forward when the latch is released and should firmly engage with the latch when raised to the upright position.
5. Continue with the installation instructions in the Epic Seating Owner's Manual.



© 2025, Stealth Products, LLC

Warranty Information



Home Page



Online User Manual



Stealth Products, LLC · info@stealthproducts.com · www.stealthproducts.com
(800) 965-9229 | (512) 715-9995 | 104 John Kelly Drive, Burnet, TX 78611