


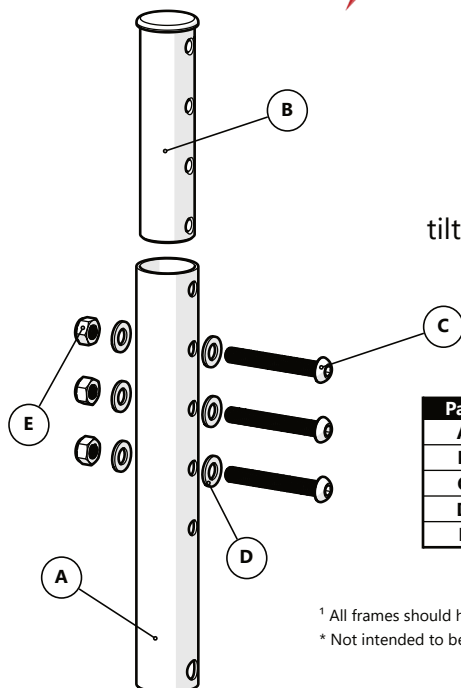

This attachment tube kit is intended to securely fit the Epic Seating System to a Ki tilt-in-space manual wheelchair¹.

Part	Description	Qty
A	Adapter Tube	1
B	Tube Nut	1
C	M6 x 1 x 45mm	3
D	M6 Washer	6
E	M6 Nylon Lock Nut	3

¹ All frames should have foot-operated tilt function and a non-recline back.

* Not intended to be used with the R82 x:panda configuration.

EPiC Seating System Adapter Set

Installation Guide

EPiC Seating System Adapter Tube Set
Ki Focus CR, Liberty, Little Wave “Flip”, Little Wave “Arc”

Customer Satisfaction

Stealth Products is committed to 100% customer satisfaction. Your complete satisfaction is important to us. Please contact us with feedback or suggestions to help us improve the quality and usability of our products.

You may reach us at:

Stealth Products, LLC

104 John Kelly Drive, Burnet, TX 78611



Phone: (512) 715-9995

Fax: (512) 715-9954

Toll Free: (800) 965-9229

Toll Free: (800) 806-1225

info@stealthproducts.com

www.stealthproducts.com

EU REP EMERGO EUROPE BV Westervoortsedijk 60 6827 AT Arnhem The Netherlands	CE GELBART AG Tribtschenstrasse 64 CH-6005 Luzern Switzerland	UK Authorized Representative EMERGO CONSULTING (UK) LTD. c/o Cr360 - UL International Compass House, Vision Park Histon Cambridge CB24 9BZ United Kingdom UK CA
--	--	--

General

Read and understand all instructions prior to the use of the product. Failure to adhere to instructions and warnings in this document may result in property damage, injury, or death. Product misuse or failure to follow instructions will void the warranty.

Immediately discontinue use if any function is compromised, if parts are missing or loose, or if any component shows signs of excessive wear. Consult with your supplier for repair, adjustment, or replacement.

All persons responsible for fitting, adjustment, and daily use of the devices discussed in these instructions must be familiar with and understand all safety aspects of the devices mentioned. In order for our products to be used successfully, you must read and understand all instructions and warnings, and maintain our products according to our instructions on care and maintenance.


The installation instructions will guide you through this product's options and possibilities.

Instructions are written with the expressed intent of use with standard configurations. They also contain important safety and maintenance information, as well as describe possible problems that can arise during use. For further assistance, or more advanced applications, please contact your supplier or **Stealth Products** at (512) 715-9995 or toll free at (800) 965-9229.

Always keep the operating instructions in a safe place so they may be referenced as necessary.


All information, pictures, illustrations, and specifications are based on the product information that was available at the time of printing. Pictures and illustrations shown in these instructions are representative examples and are not intended to be exact depictions of the various parts of the product.

Installation Instructions

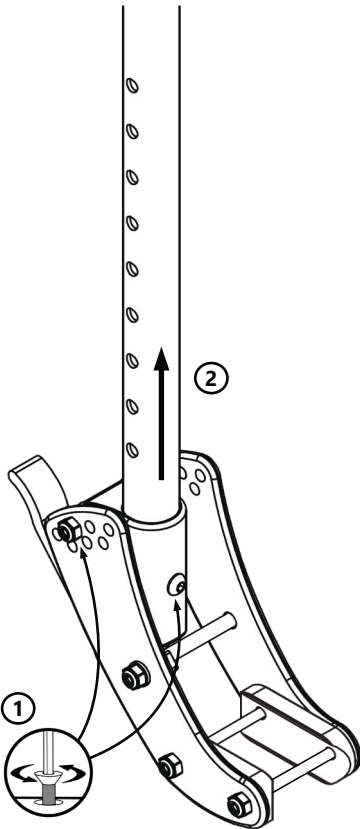

1. Loosen and remove the screw securing the cane to the receiver. If the receiver is set to a seat-to-back angle other than 0°, remove the screw and return the receiver to the 0° position.* (Front hole in the bottom row.)
2. Remove the cane from the receiver.
3. Insert the ESCTUN-003 adapter tube into the receiver. The heads of the screws installed in the adapter tube should face outwards from the chair, left on the left tube, right on the right.

4. Install the screw removed in Step 1 into the receiver, through the attachment tube, and tighten it to secure the tube in place. Tighten the screw securing the seat-to-back angle.
5. Continue with the installation of Epic Seating as described in the installation section of the Seating Owner's Manual.

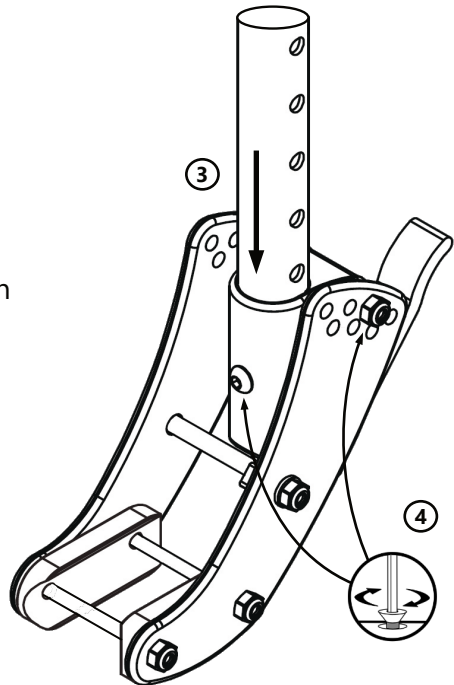



© 2025, Stealth Products, LLC

Warranty Information



Home Page



Online User Manual



Stealth Products, LLC · info@stealthproducts.com · www.stealthproducts.com
(800) 965-9229 | (512) 715-9995 | 104 John Kelly Drive, Burnet, TX 78611