


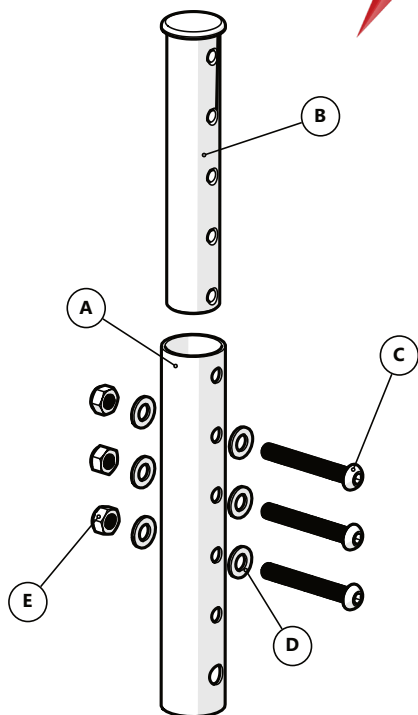

This adapter tube kit is intended to securely fit the Epic Seating System to a Leggero Enzo wheelchair.

Part	Description	Qty
A	Adapter Tube	1
B	Tube Nut	1
C	M6 x 1 x 45mm	3
D	M6 Washer	6
E	M6 Nylon Lock Nut	3

EPiC Seating System Adapter Set

Installation Guide

EPiC Seating System Adapter Tube Set Leggero Enzo

Customer Satisfaction

Stealth Products is committed to 100% customer satisfaction. Your complete satisfaction is important to us. Please contact us with feedback or suggestions to help us improve the quality and usability of our products.

You may reach us at:

Stealth Products, LLC

104 John Kelly Drive, Burnet, TX 78611



Phone: (512) 715-9995

Fax: (512) 715-9954

Toll Free: (800) 965-9229

Toll Free: (800) 806-1225

info@stealthproducts.com

www.stealthproducts.com

EU REP  PRIDE MOBILITY PRODUCTS EUROPE BV De Zwaan 3 1601 MS Enkhuizen The Netherlands	CH REP SODIMED SA Chemin Praz Devant 12 1032 Romanel sur-Lausanne Switzerland	UK Authorized Representative PRIDE MOBILITY PRODUCTS LTD. 32 Wedgwood Road, Bicester, Oxfordshire OX26 4UL United Kingdom 
--	---	---

General

Read and understand all instructions prior to the use of the product. Failure to adhere to instructions and warnings in this document may result in property damage, injury, or death. Product misuse or failure to follow instructions will void the warranty.

Immediately discontinue use if any function is compromised, if parts are missing or loose, or if any component shows signs of excessive wear. Consult with your supplier for repair, adjustment, or replacement.

All persons responsible for fitting, adjustment, and daily use of the devices discussed in these instructions must be familiar with and understand all safety aspects of the devices mentioned. In order for our products to be used successfully, you must read and understand all instructions and warnings, and maintain our products according to our instructions on care and maintenance.


The installation instructions will guide you through this product's options and possibilities.

Instructions are written with the expressed intent of use with standard configurations. They also contain important safety and maintenance information, as well as describe possible problems that can arise during use. For further assistance, or more advanced applications, please contact your supplier or **Stealth Products** at (512) 715-9995 or toll free at (800) 965-9229.

Always keep the operating instructions in a safe place so they may be referenced as necessary.

All information, pictures, illustrations, and specifications are based on the product information that was available at the time of printing. Pictures and illustrations shown in these instructions are representative examples and are not intended to be exact depictions of the various parts of the product.

Installation Instructions

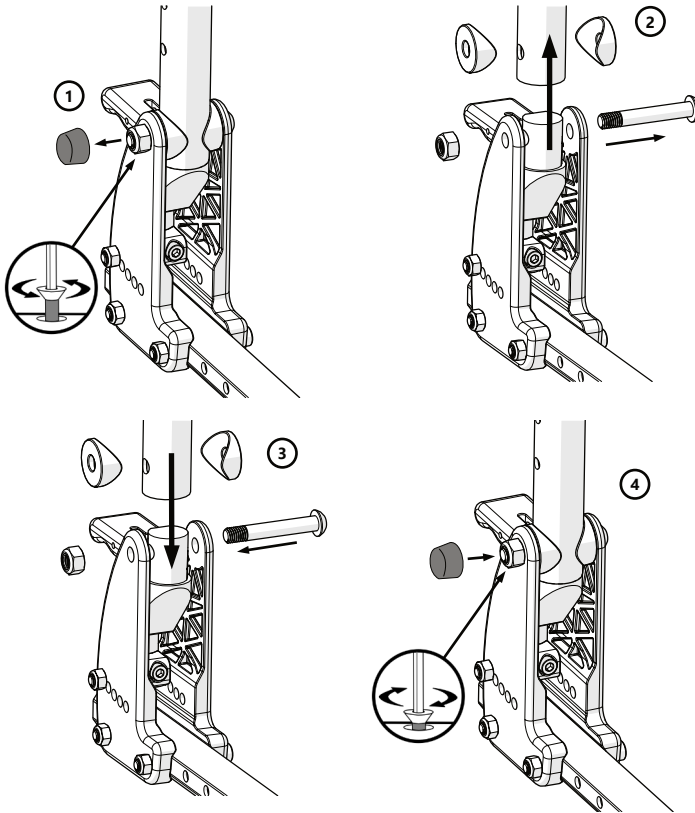

1. Remove the caps covering the nuts on the pivot bolts.
2. With a hex wrench and socket, remove the pivot bolts from the backrest tubes. Retain the tube washers, and set the backrest tubes aside.
3. Using the same tube washers and pivot bolts, install the attachment tubes on the pivot assemblies and bolt into place using the pivot bolts and nuts removed in steps 1 and 2.
4. Tighten the pivot bolts and nuts to secure the attachment tubes in place, but do not overtighten. Using the release lever, check the pivot assemblies for free movement. Adjust as necessary. Replace the caps.*
5. Continue the installation as described in the installation section of the Epic Seating Owner's Manual.

* For Leggero-specific information, please consult the manual that came with your Leggero chair.



© 2025, Stealth Products, LLC

Warranty Information



Home Page



Online User Manual



Stealth Products, LLC · info@stealthproducts.com · www.stealthproducts.com
(800) 965-9229 | (512) 715-9995 | 104 John Kelly Drive, Burnet, TX 78611