


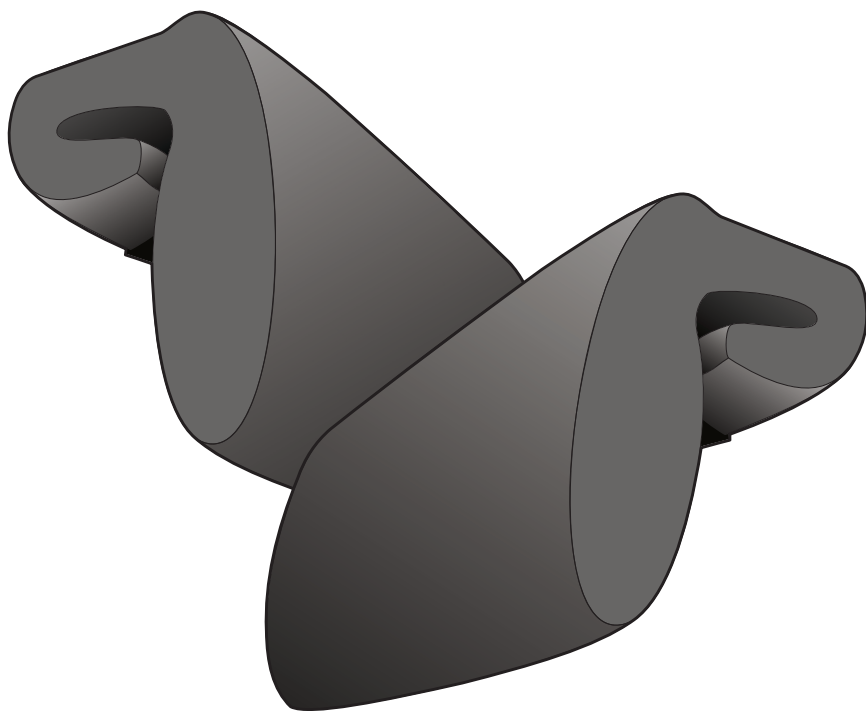


## Lateral Support Bolster

### Intended Use

The Lateral Support bolster is a low-profile, covered foam bolster designed to fit over most 10- or 16-inch wheelchair armrests. It may improve the user's overall comfort when seated in the wheelchair. The Lateral Support Bolster, when used in conjunction with an adjustable armrest, may help to reduce the risk of shoulder subluxation, pain and contractures at the elbow, wrist, and hand. The padding provides additional comfort at the elbow and wrist and may provide positioning support of the upper extremities. For power wheelchair users, this may also provide easier access to the joystick. The cover material protects the bolster foam from fluid absorption.

The lateral support bolster is intended for users who require mild support to appropriately position upper extremities, or for users who require protection due to uncontrolled arm movement. Lateral support bolsters should not be used as the only postural intervention for users who exhibit increased tone and/or heavy leaning and should be paired with mounted trunk lateral supports.

## Installation and Use

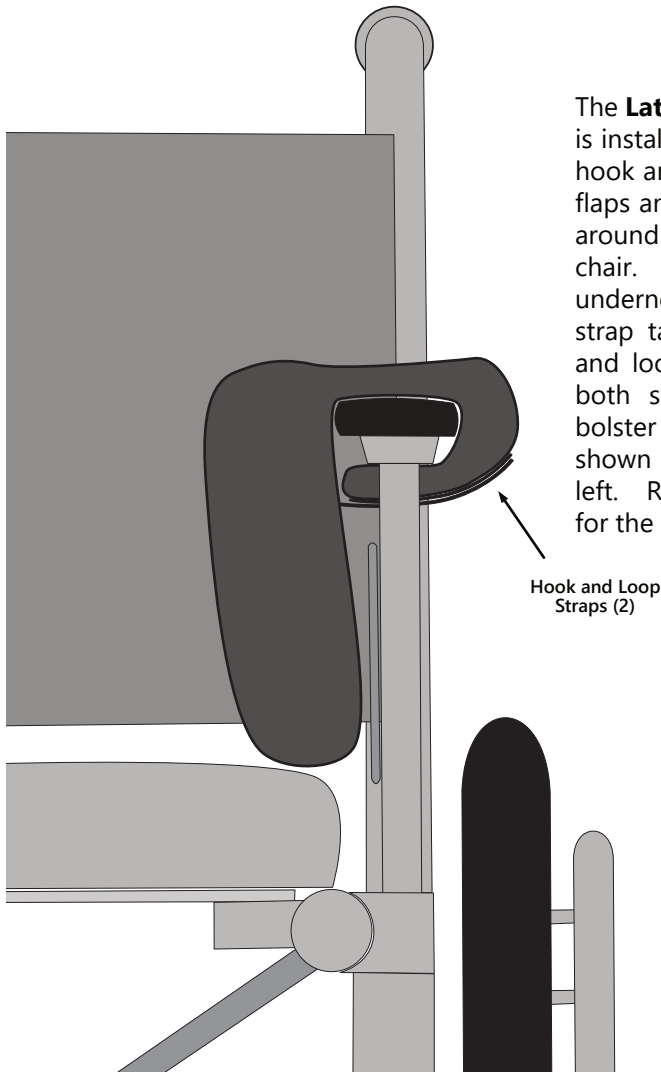

The **Lateral Support Bolster** is installed by separating the hook and loop straps on the flaps and placing the bolster around the armrests of the chair. Fold the bottom flap underneath and pull the loop strap taut. Press the hook and loop sides together on both straps to secure the bolster to the armrest, as shown in the image to the left. Repeat the procedure for the right bolster.

**NOTICE:** The lateral support bolster should not be used as the only postural intervention for users who exhibit increased tone and/or heavy leaning and should be paired with mounted trunk lateral supports.

## Care and Cleaning

We recommend that all products be examined on a regular basis for wear and function. Prior to use, check the foam for breakdown, the cover material and stitching on the webbing for stress or damage, and that hook-and-loop fasteners are securely fastened. If loose, frayed or broken cover material, webbing, or foam are discovered, discontinue use and contact your provider. Repairs should only be performed by a wheelchair service professional and with genuine Stealth Products parts, fasteners or items appropriate for the service required.

Surfaces may be cleaned of dust and dirt with a damp cloth. Ensure all cleaners are approved for finished steel, aluminum, plastic and upholstered surfaces. Cover may be removed for cleaning. Care should be taken not to damage foam during cover removal.

## Important

Read and understand all instructions prior to the use of the product. Failure to adhere to instructions and warnings in this document may result in property damage, injury, or death. Product misuse or failure to follow instructions will void the warranty.

Immediately discontinue use if any function is compromised, if parts are missing or loose, or if any component shows signs of excessive wear. Consult with your supplier for repair, adjustment, or replacement.

All persons responsible for fitting, adjustment, and daily use of the devices discussed in these instructions must be familiar with and understand all safety aspects of the devices mentioned. In order for our products to be used successfully, you must read and understand all instructions and warnings, and maintain our products according to our instructions and warnings, and maintain our products according to our instructions on care and maintenance.

The installation instructions will guide you through this product's options and possibilities.

Instructions are written with the expressed intent of use with standard configurations. They also contain important safety and maintenance information, as well as describe possible problems that can arise during use. For further assistance, or more advanced applications, please contact your supplier or Stealth Products at (512) 715-9995 or toll-free at (800) 965-9229.

Always keep the operating instructions in a safe place so they may be referenced as necessary.

All information, pictures, illustrations, and specifications are based on the product information that was available at the time of printing. Pictures and illustrations shown in these instructions are representative examples and are not intended to be exact depictions of the various parts of the product.

## About Us

**Stealth Products** is committed to 100% customer satisfaction. Your complete satisfaction is important to us. Please contact us with suggestions or feedback to help us improve the quality and usability of our products.

You may reach us at:



### Stealth Products, LLC

104 John Kelly Drive,  
Burnet, TX 78611

Phone: (512) 715-9995 Toll Free: (800) 965-9229

Fax: (512) 715-9954 Toll Free: (800) 806-1225

info@stealthproducts.com - www.stealthproducts.com



|                                                                                                        |                                                                                           |                                                                                                                                                                                           |
|--------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>EU REP</b><br><b>EMERGO EUROPE B.V.</b><br>Westervoortsedijk 60,<br>6827 AT Arnhem, The Netherlands | <b>CH REP</b><br><b>GELBART AG</b><br>Tribtschenstrasse 64<br>CH-6005 Luzern, Switzerland | <b>UK Authorized Representative</b><br><b>EMERGO CONSULTING (UK) LIMITED</b><br>C/o Cr360 - UL International,<br>Compass House, Vision Park Histon,<br>Cambridge CB24 9BZ, United Kingdom |
|--------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|

## Warranty

Our products are designed, manufactured, and produced to the highest of standards. If any defect in material or workmanship is found, **Stealth Products** will repair or replace the product at our discretion. Any implied warranty, including the implied warranties of merchantability and fitness for a particular purpose, shall not extend beyond the duration of this warranty. **Stealth Products, LLC** does not warrant damage due to, but not limited to, misuse, abuse, or misapplication of products and modification of product without written approval from **Stealth Products, LLC**.

Any alteration or lack of serial number, where applicable, will automatically void this warranty. **Stealth Products, LLC** is liable for replacement parts only. **Stealth Products, LLC** is not liable for any incurred labor costs. No person is authorized to alter, extend, or waive the warranties of **Stealth Products, LLC**.

To learn more about our warranty and returns policy, please visit <https://stlpro.site/returns-n-warranty>.

Warranty Information



Home Page



Online User Manual

