


Stealth Products
®



**Mounting
Hardware**

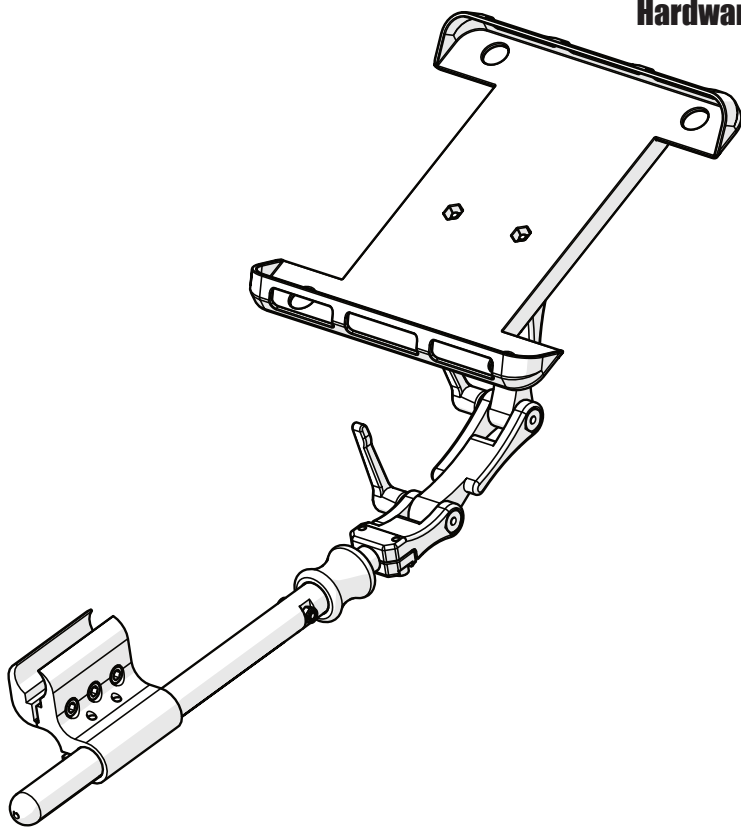

Swing-Away Link-Style Tablet Mount

Owner's Manual - Maintenance Guide

URPKTAB0000/URPKSMU0000



Table of Contents

Customer Satisfaction	i
General	i
Important Information	ii-iii
Warranty	ii
Supplier Reference.....	ii
Warning Labels.....	iii
Limited Liability	iii
Testing	iii
Design and Function	1
Intended Use.....	1
Tablet Mount Specifications.....	1
Parts and Accessories	2-3
URPKTAB000/URPKSMU0000 Packages.....	2
Replacement Parts	3
Tools	3
Torque Specifications.....	3
Installation Instructions	4-7
Replacing the FDM380.....	4
Link Replacement/Adjustment.....	5
Armrest Mounting	6
Depth Adjustment	6
Flip-Down Mount.....	7
First Time Use	8
Dealer Assistance	8
User Testing	8
Conditions of Use	8
Maintenance	9
Maintenance	9
Cleaning	9
Notes	10

Stealth Products is committed to 100% customer satisfaction. Your complete satisfaction is important to us. Please contact us with feedback or suggestions to help us improve the quality and usability of our products.

You may reach us at:

Stealth Products, LLC
104 John Kelly Drive, Burnet, TX 78611



Phone: (512) 715-9995
Fax: (512) 715-9954

Toll Free: (800) 965-9229
Toll Free: (800) 806-1225

info@stealthproducts.com
www.stealthproducts.com

EU REP EMERGO EUROPE BV Westervoortsedijk 60 6827 AT Arnhem The Netherlands	CH REP GELBART AG Tribtschenstrasse 64 CH-6005 Luzern Switzerland	UK Authorized Representative EMERGO CONSULTING (UK) LTD. c/o Cr360 - UL International Compass House, Vision Park Histon Cambridge CB24 9BZ United Kingdom
--	--	--

General

Read and understand all instructions prior to the use of the product. Failure to adhere to instructions and warnings in this document may result in property damage, injury, or death. Product misuse or failure to follow instructions will void the warranty.

Immediately discontinue use if any function is compromised, if parts are missing or loose, or if any component shows signs of excessive wear. Consult with your supplier for repair, adjustment, or replacement.

All persons responsible for fitting, adjustment, and daily use of the devices discussed in these instructions must be familiar with and understand all safety aspects of the devices mentioned. In order for our products to be used successfully, you must read and understand all instructions and warnings, and maintain our products according to our instructions on care and maintenance.

The installation instructions will guide you through this product's options and possibilities.

Instructions are written with the expressed intent of use with standard configurations. They also contain important safety and maintenance information, as well as describe possible problems that can arise during use. For further assistance, or more advanced applications, please contact your supplier or **Stealth Products** at (512) 715-9995 or toll free at (800) 965-9229.

Always keep the operating instructions in a safe place so they may be referenced as necessary.

All information, pictures, illustrations, and specifications are based on the product information that was available at the time of printing. Pictures and illustrations shown in these instructions are representative examples and are not intended to be exact depictions of the various parts of the product.

Important Information

CAUTION

These products are designed to be fitted, applied, and installed exclusively by a healthcare professional trained for these purposes. The fitting, application, and installation by a non-qualified individual could result in serious injury.

Warranty

Our products are designed, manufactured, and produced to the highest of standards. If any defect in material or workmanship is found, **Stealth Products** will repair or replace the product at our discretion. Any implied warranty, including the implied warranties of merchantability and fitness for a particular purpose, shall not extend beyond the duration of this warranty. **Stealth Products** does not warrant damage due to, but not limited to: Misuse, abuse, or misapplication of product, and/or modification of product without written approval from **Stealth Products, LLC**. Any alteration or lack of serial number, where applicable, will automatically void all warranty.

Stealth Products, LLC is liable for replacement parts only. **Stealth Products, LLC** is not liable for any incurred labor costs. **Stealth Products** warrants against failure due to defective materials or workmanship:

Covers, Headrest Pads, Cushions: 2 years

Hardware: 5 years

Electronics: 3 years

In the event of a product failure covered by our warranty, please follow the procedures outlined below:

- Call **Stealth Products** at (512) 715-9995 or toll free at (800) 965-9229.
- Request a Return Authorization (RA) form from the Returns Department and follow the documentation instructions.

Full warranty information may be found by visiting the website at <https://stlpro.site/returns-n-warranty>, or by scanning the Warranty Information QR code located on the back cover of this manual. You can download additional copies of this manual by accessing the Stealth website (www.stlpro.site/stealth-docs) and searching "P139D17" in the search bar at the top of the page.

Supplier Reference

Supplier:	
Telephone:	
Address:	
Purchase Date:	
Model:	

Warning Labels

Warnings are included for the safety of the user, client, operator, and property. Please read and understand what the signal words **DANGER**, **WARNING**, **CAUTION**, **NOTICE**, and **SAFETY** mean, and how they could affect the user, those around the user, and property.

DANGER

Identifies an **imminent** situation which, if not avoided, may result in **severe injury, death, and property damage**.

WARNING

Identifies a **potential** situation which, if not avoided, may result in **severe injury, death, and property damage**.

CAUTION

Identifies a **potential** situation which, if not avoided, may result in **minor to moderate injury and property damage**.

NOTICE

Identifies important information not related to injury, but possible **property damage**.

SAFETY

Indicates steps or instructions for safe practices, reminders of safe procedures, or important safety equipment that may be necessary.

Limited Liability

Stealth Products, LLC accepts no liability for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendations, warnings, and instructions in this manual.

Stealth Products does not hold responsibility for final integration of final assembly of product to end user. **Stealth Products** is not liable for user death or injury.

Testing

Initial setup and driving should be done in an open area free of obstacles until the user is fully capable of driving safely.

Intended Use


The **URPKTAB0000/URPKSMU0000** hardware is intended to provide a sleek, reliable, and easy-to-operate mounting solution for a digital tablet on a manual or power wheelchair.

Stealth's link-style mounting hardware provides nearly limitless adjustment capabilities to accommodate most positioning requirements. The flip-down mount allows the tablet to be rotated away without removing it from the chair, aiding transfers and assists, while mitigating the chance of loss due to misplacement of the device.


Tablet Mount Specifications

- Mounting dimensions for the URPKTAB0000 include:
 - *Height: 8.5 inches to 11.125 inches.*
 - *Width: 6.38 inches to 7.75 inches.*
 - *Depth: .13 inches to .56 inches.*
 - *Compatible devices: iPad 1 - 4, HP Touchpad, Asus EEE Pad Memo, Barnes & Noble NOOKColor, Blackberry Playbook, Dell Streak 7, HTC Flyer, Samsung Galaxy Tablets, MS Surface and more.*
- Mount dimensions for the URPKSMU0000 include:
 - - *Height: 7 inches to 10 inches.*
 - - *Width: 4 inches to 5.31 inches.*
 - - *Depth: .44 inches to 1.06 inches.*
 - - *Compatible with most 7" to 10" tablets*
 - *The TWBM480-U provides three sets of interchangeable cup ends.*

Both tablet mounts are designed with spring-loaded tension to provide secure support and quick release of the tablet.

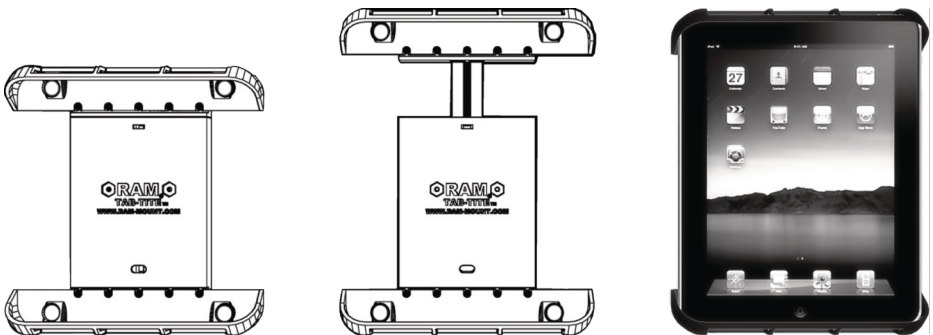


Parts

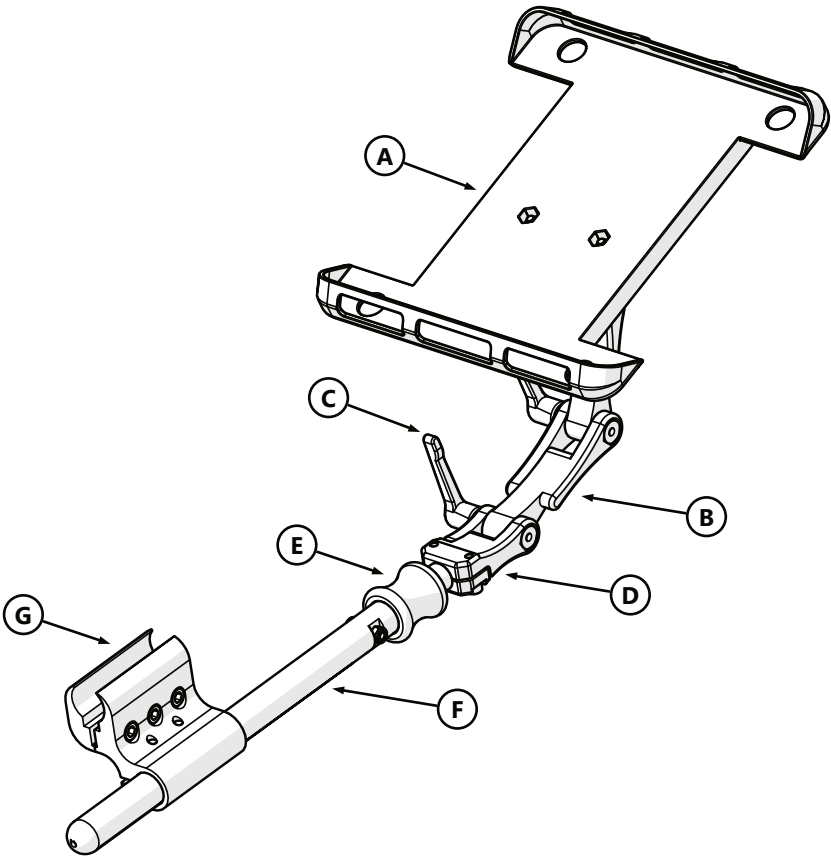

	Part Number	Description
A*	RAM Tab-Tite™	Tablet Mount for iPad 1-4
A*	RAM Tab-Tite™ Small	Universal Tablet Mounting Bracket
B	Inline Link	UniLink 3" Link
C	Adjustment Handle	Stud Adjustable Clamping Handle
D	17mm Headrest Mount	17mm Mount with Insert
E	TWB480-71	Flip Down Sleeve
F	TWB480-80	.75" FDM Rod, No Slot
G	ARM240 Mount	Mounting Bracket

* URPKTAB0000 mount shown above. URPKSMU0000 is the same except for Part A.

Tools

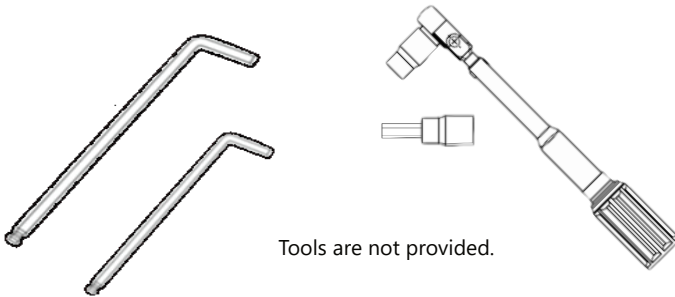

Tools are not provided.

Hardware
2.5 mm Hex Wrench
4mm Hex Wrench
5mm Hex Wrench
Torque Wrench
Hex Bits

CAUTION

Any connection must always be secured with all delivered screws.
Use only the screws provided in the package.

Torque Specifications


Screws	Torque Specifications
M5 x 0.8 x 14mm	6.8 N·m 5 ft-lb
M5 x 0.8 x 16mm	
M5 x 0.8 x 18mm	
M6 x 1.0 x 20mm	11 N·m 8.1 ft-lb
M6 x 1.0 x 25mm	

CAUTION

Ensure that all screws are tightened evenly on the hardware. Threads can strip out and cause damage to the hardware if not tightened properly.

Installation to the Armrest


- Step One:** Loosen the three M6 x 1.0 x 25mm Socket Head Screws until the gap between the claw and the jaw is wider than the tube onto which you are installing the clamp.
- Step Two:** Place the tube inside the claw of the clamp and squeeze the claw and jaw together to hold the tube inside the clamp.
- Step Three:** Tighten the M6 x 1.0 x 25mm Socket Head Screws until the clamp is secure on the tube.
- Step Four:** Verify the orientation of the tablet mount and torque the fasteners to the values shown on page 3.

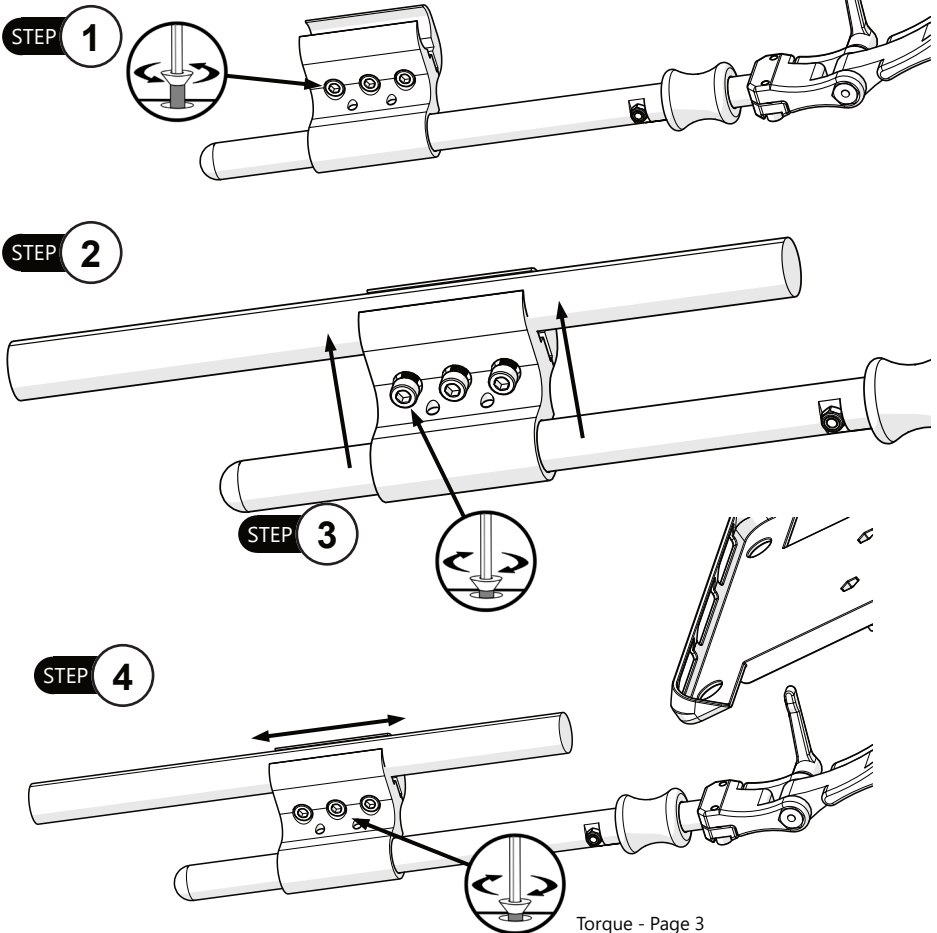

Torque - Page 3

Link Adjustment/Replacement

- Step One:** The fastener handle can be loosened or removed by turning it counterclockwise to loosen the link segment. When it is loose, the segment can be re-positioned or the fastener can be taken out to add in or remove a segment.
- Step Two:** If adjusting the the segment, rotate it to the new position and tighten the fastener handle to secure the segment.
- Step Three:** If removing or adding a link segment, remove the fastener handle and washer. Lift away the cap released by the fastener and gently twist to uninstall the link. To remove the link, or to replace it, remove the fastener at the other end of the link in the same fashion.
- Step Four:** After reassembly, be sure to tighten the fastener handles sufficiently to hold the links in the assigned positions.

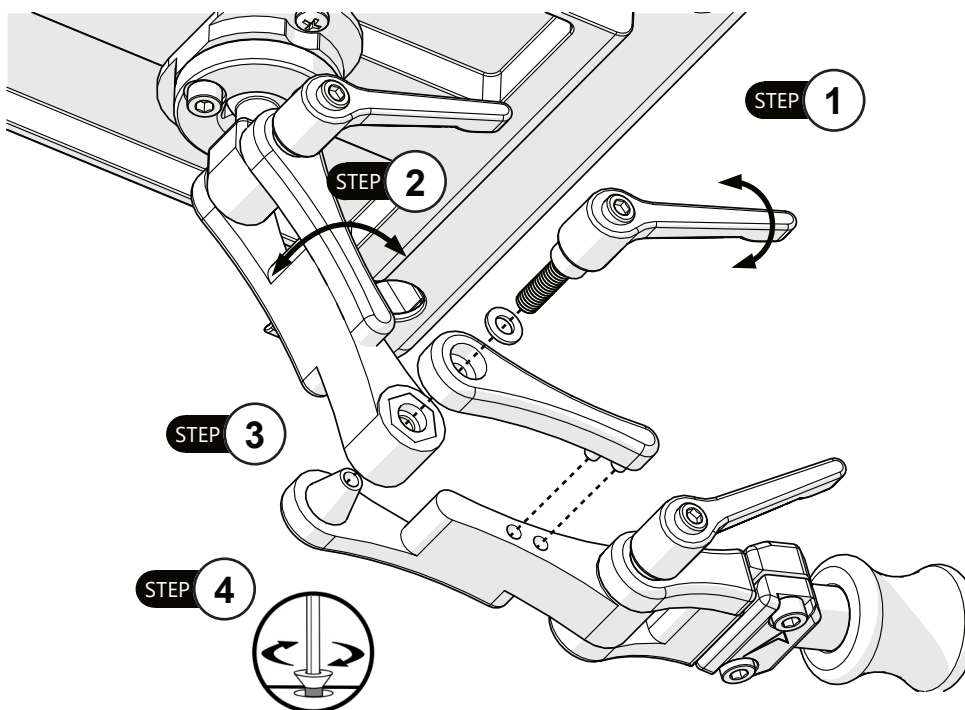


NOTICE

Both URPKTAB000 & URPKSMU000 follow the same installation and adjustment instructions as shown throughout the manual.


Depth Adjustment

- Step One:** Loosen the three bolts securing the clamp to the armrest tube, or the two bolts securing the clamp to the stem.
- Step Two:** Move the clamp or the stem forward or backward the required amount to achieve the depth desired.
- Step Three:** Tighten the bolts to the torque specification called out on Page 3.

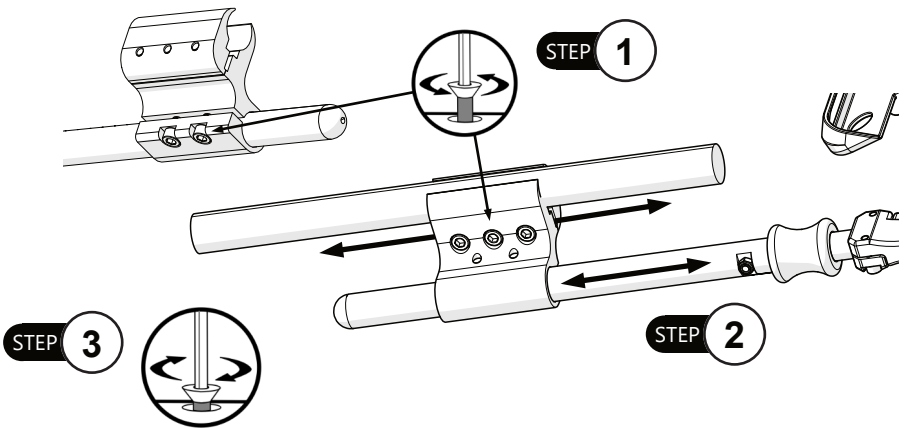


Lateral/Angle Adjustment

- Step One:** Locate the 17mm ball mount at the base of the tablet mount, or the clamp at the end of the stem.
- Step Two:** Loosen the three screws securing the tablet mount to the 17mm ball and adjust it to the desired angle, or loosen the two screws securing the link assembly to the end of the stem and rotate it left or right to the desired angle.
- Step Three:** Tighten the screws to secure the tablet mount at the desired angle.

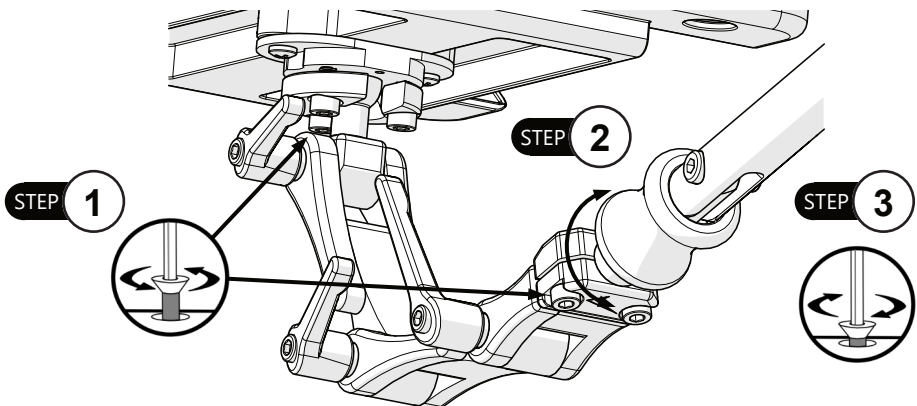

Flip-Down Operation

- Step One:** To flip down the tablet mount, locate the red flip-down sleeve where the stem meets the links. Press it forward and away from the stem.
- Step Two:** As the tablet and links flip down, remove your hand from the flip-down sleeve area, as the spring in the sleeve could cause it to pinch skin.
- Step Three:** To return the tablet to the upright position, rotate it up into position until the flip-down sleeve makes an audible click. Test it to make sure it is locked in the upright position prior to releasing the tablet.

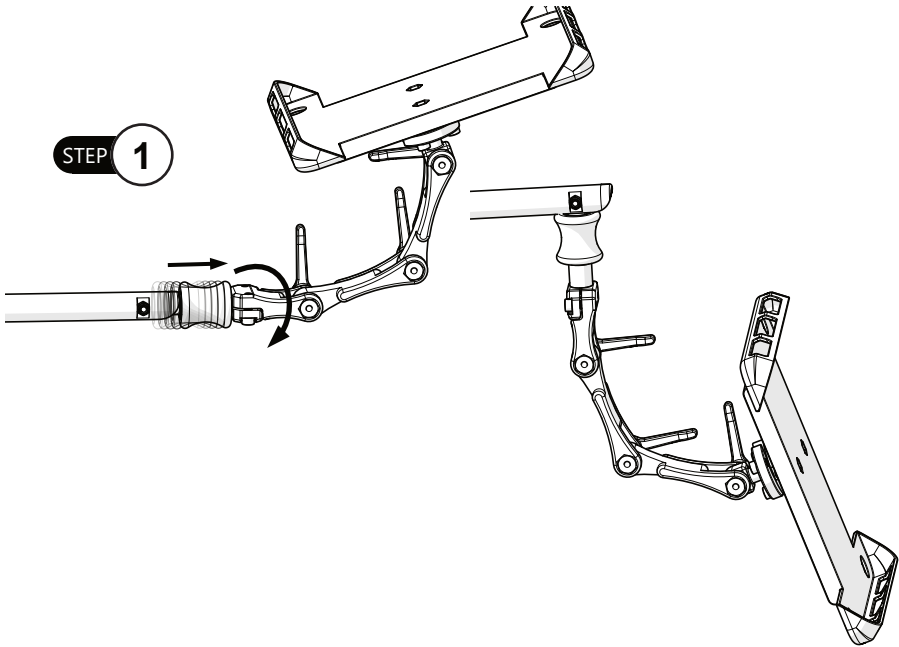

CAUTION

Ensure the red flip-down sleeve freely rotates 360°. If the flip-down sleeve does not rotate, the hardware will not work properly.

CAUTION

Ensure the flip down mount is fully engaged before use.

SAFETY

Move hands from the flip-down sleeve area when bringing it back to a vertical state. The sleeve could pinch when moving and could cause injury.

Dealer Assistance

During first time use by the client, it is advised that the dealer or service technician assists and explains the configuration to the customer (the user and/or the attendant). If needed, the dealer can make final adjustments.



WARNING

The **URPKTAB0000/URPKSMU0000** should only be prescribed and fitted by qualified health care professionals.

User Testing

It is important that the customer is fully aware of the installation of the **URPKTAB0000/URPKSMU0000**, how to use it, and how it can be adjusted to fit the client comfortably. As a dealer, proceed as follows:

- Explain and show the client how you have executed the installation, and explain the function.

Have the user test the position of the **URPKTAB0000/URPKSMU0000**:

- Is the hardware in the proper position for the client?
- Can the user safely operate the chair?

If needed, adjust the hardware to the proper position.

Explain to the customer any possible issues and how to address them.

Conditions of Use

The **URPKTAB0000/URPKSMU0000** is intended for use as installed by the dealer, in accordance to the installation instructions in this manual.

- The foreseen conditions of use are communicated by the dealer or service technician to the user and/or attendant during the first time use.
- If the conditions of use change significantly, please contact your dealer or a qualified service technician to avoid excessive wear and tear or unintended damage.

Maintenance

- All parts should be checked on a regular basis and repaired, if necessary, by a service technician.
- Discontinue use after identifying loose or broken components.
- Periodically check the flip-down sleeve. Test that it engages properly and makes an audible 'click' when moved back to the vertical position.
- Check that the tablet mount is secure.
- Check the M5 bolt on the vertical bar. Tighten if it has become loose.
- To prevent screws from seizing, reapply copper anti-seize as necessary.
- If you are experiencing excessive lateral movement with the hardware, refer to Step 5 on Page 5 for adjustment instructions.

Cleaning

- Wipe hardware clean with a damp cloth and mild, non-abrasive cleaner.
- Ensure any cleaners used to disinfect are approved for cleaning aluminum surfaces.



WARNING

Failure to comply with these warnings could result in serious bodily harm.

[illegible]



© 2023, Stealth Products, LLC

Warranty Information



Home Page



Online User Manual



Stealth Products, LLC · info@stealthproducts.com · www.stealthproducts.com
(800) 965-9229 | (512) 715-9995 | 104 John Kelly Drive, Burnet, TX 78611

P139D17R8

Aug 12, 2026