


Stealth Products[®]



**Alternative
Drive Controls**

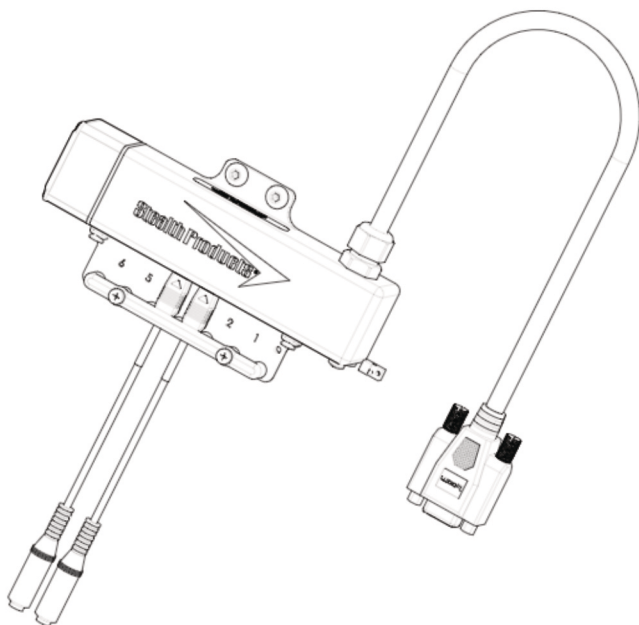

i-Drive 4.0 with Bluetooth[®]

Owner's Manual - Maintenance Guide

for the i-Drive 4.0 Interface
with *Bluetooth[®]* Wireless Technology



[®]



Table of Contents

Customer Satisfaction	i
General	i
Important Information	ii-iv
Warranty	ii
Supplier Reference	ii
Warning Labels	iii
Limited Liability	iii
Testing	iii
Federal Communications Commission (FCC) Statement (USA)	iv
Glossary of Terms	1
Design & Function	2-4
Intended Use	2
Included Parts IDHBT500-1	2
Mounting Hardware Tools	2
Port Assignment	3
i-Drive Bluetooth Minimum System Requirements	3
EMC Requirements	4
Control System Overview	4
Installation Instructions	5-12
Installing/Replacing IDHBT500 Interface	5
Installing/Replacing IDHBT500-1	6-9
Ring Mount Installation	10
Installing the IDHBT500 Interface	11
Uninstalling/Replacing the IDHBT500	12
Replacing/Installing Dongles	13
R-Net Omni Setup	14
R-Net Setup: Switched Output	15
Q-Logic Setup	16
Q-Logic 2 Setup: Proportional Output	17
Q-Logic 2 Setup: Switched Output	18
Q-Logic 2 Further Configuration	19
Q-Logic 3: Proportional Output	20
Q-Logic 3 Setup: Switched Output	21
Q-Logic 3: Further Configuration	21-22
MK6i Programmer Setup	23
MK6i Setup: Switched Output	24
Powered Seating Control	24
Troubleshooting	25-28
Software Configuration	27
Windows Configurator Software	28-29
Software Capabilities	30
Software Download	30-35
Diagnostics	36
Configuration	36
Channel Assignment	37
Chair Calibration	37
Output Mode	38
Factory Reset	38
Firmware Update	39
Testing	40
Testing	40
Operational Test	40
Test Drive	40
First Time Use	41
Dealer Assistance	41
User Testing	41
Conditions of Use	41
Cleaning and Maintenance	42
Care & Maintenance	42
Safety	42
Technical Data	42
Notes	43-45

Stealth Products is committed to 100% customer satisfaction. Your complete satisfaction is important to us. Please contact us with feedback or suggestions to help us improve the quality and usability of our products.

You may reach us at:

Stealth Products, LLC
104 John Kelly Drive, Burnet, TX 78611



Phone: (512) 715-9995
Fax: (512) 715-9954

Toll Free: (800) 965-9229
Toll Free: (800) 806-1225

info@stealthproducts.com
www.stealthproducts.com

EU REP EMERGO EUROPE BV Westervoortsedijk 60 6827 AT Arnhem The Netherlands	CE	CH REP GELBART AG Tribtschenstrasse 64 CH-6005 Luzern Switzerland	UK Authorized Representative EMERGO CONSULTING (UK) LTD. c/o Cr360 - UL International Compass House, Vision Park Histon Cambridge CB24 9BZ United Kingdom	UK CA
---	-----------	---	--	--------------

General

Read and understand all instructions prior to the use of the product. Failure to adhere to instructions and warnings in this document may result in property damage, injury, or death. Product misuse or failure to follow instructions will void the warranty.

Immediately discontinue use if any function is compromised, if parts are missing or loose, or if any component shows signs of excessive wear. Consult with your supplier for repair, adjustment, or replacement.

All persons responsible for fitting, adjustment, and daily use of the devices discussed in these instructions must be familiar with and understand all safety aspects of the devices mentioned. In order for our products to be used successfully, you must read and understand all instructions and warnings, and maintain our products according to our instructions on care and maintenance.

The installation instructions will guide you through this product's options and possibilities.

Instructions are written with the expressed intent of use with standard configurations. They also contain important safety and maintenance information, as well as describe possible problems that can arise during use. For further assistance, or more advanced applications, please contact your supplier or **Stealth Products** at (512) 715-9995 or toll free at (800) 965-9229.

Always keep the operating instructions in a safe place so they may be referenced as necessary.

All information, pictures, illustrations, and specifications are based on the product information that was available at the time of printing. Pictures and illustrations shown in these instructions are representative examples and are not intended to be exact depictions of the various parts of the product.

The Bluetooth® word mark and logo are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Stealth Products, LLC is under license. Other trademarks and trade names are those of their respective owners.

Important Information



CAUTION

These products are designed to be fitted, applied, and installed exclusively by a healthcare professional trained for these purposes. The fitting, application, and installation by a non-qualified individual could result in serious injury.

Warranty

Our products are designed, manufactured, and produced to the highest of standards. If any defect in material or workmanship is found, Stealth Products will repair or replace the product at our discretion. Any implied warranty, including the implied warranties of merchantability and fitness for a particular purpose, shall not extend beyond the duration of this warranty. **Stealth Products** does not warrant damage due to, but not limited to: misuse, abuse, or misapplication of product, and/or modification of product without written approval from **Stealth Products, LLC**. Any alteration or lack of serial number, where applicable, will automatically void all warranty.

Stealth Products, LLC is liable for replacement parts only. **Stealth Products, LLC** is not liable for any incurred labor costs.

Stealth Products warrants against failure due to defective materials or workmanship:

Covers: 2 years
Hardware: 5 years
Electronics: 3 years

In the event of a product failure covered by our warranty, please follow the procedures outlined below:

Call **Stealth Products** at (512) 715-9995 or toll free at (800) 965-9229.

Request a Return Authorization (RA) form from the Returns Department and follow the documentation instructions.

You can download additional copies of this manual by accessing the Stealth website (<https://stlpro.site/stealth-docs>) and searching "i-Drive 4.0" in the search bar at the top of the page.

Supplier Reference

Supplier:	
Telephone:	
Address:	
Purchase Date:	
Model:	

Warning Labels

Warnings are included for the safety of the user, client, operator, and property. Please read and understand what the signal words **DANGER**, **WARNING**, **CAUTION**, **NOTICE**, and **SAFETY** mean, and how they could affect the user, those around the user, and property.



DANGER

Identifies an **imminent** situation which, if not avoided, may result in **severe injury, death, and property damage**.



WARNING

Identifies a **potential** situation which, if not avoided, may result in **severe injury, death, and property damage**.



CAUTION

Identifies a **potential** situation which, if not avoided, may result in **minor to moderate injury and property damage**.

NOTICE

Identifies important information not related to injury, but possible **property damage**.

SAFETY

Indicates steps or instructions for safe practices, reminders of safe procedures, or important safety equipment that may be necessary.

Limited Liability

Stealth Products, LLC accepts no liability for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendations, warnings, and instructions in this manual.

Stealth Products does not hold responsibility for final integration of final assembly of product to end user. **Stealth Products** is not liable for user death or injury.

Testing

Initial setup and driving should be done in an open area free of obstacles until the user is fully capable of driving safely.

Federal Communications Commission (FCC) Statement (USA)

FCC ID: 2AJXVIDHBT500

FCC RF Exposure Statement

This device meets the FCC requirements for RF exposure in public or uncontrolled environments.

This device complies with FCC part 15 FCC Rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference
2. This device must accept any interference received, including interference that may cause undesired operation.

FCC Warning

Changes or modifications not approved by **Stealth Products, LLC** could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna;
- Increase the separation between the equipment and receiver;
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected; and/or
- Consult the dealer or an experienced radio or TV technician for help.

Glossary

Configurable Parameters (Switched): There are two additional configurable parameters in the i-Drive configuration application for switched inputs, sensor sensitivity and linked left/right.

Sensor Sensitivity: This parameter shifts the position of the switch no-change region. A higher value therefore means that a switch will always be interpreted as ON, whereas a lower value means that a switch will almost always be interpreted as OFF.

Linked Left/Right: The left and right inputs can be linked so the chair moves forward when both are engaged. This feature may be enabled under the sensor settings for Left or Right in the configurator apps, or by selecting *Linked Left* and *Linked Right* to the respective channels in the i-Drive Programmer Windows application.

Configurable Parameters (Proportional): Proportional inputs have the following additional configurable parameters. Note that it is best to handle these by first calibrating the input through *Joystick Calibration* in the i-Drive configuration applications.

Deadband Region: This may be used to define the deadband low and high relative to the proportional minimum and maximum. Calibration is required in order to calculate the center voltage.

Calibration: This feature is accessible directly through *Config Mode* rather than the *Channels* menu after a proportional function has been assigned. The calibration procedure calculates the center voltage as well as the recommended values for the proportional region and dead band region based on user input.

Configurable Parameters (Multi-Tap): The multi-tap feature is configurable under *Multi-Tap* in the i-Drive configuration applications. Two parameters are configurable, *Input Delay* and *Output Speed*.

Multi-Tap: The i-Drive system implements an extended double-tap timer to accommodate clients who need a longer double-tap input time. The double-tap extension time is shared with the triple-tap activation time in mouse mode.

Input Delay: This is the longest time that the i-Drive system will wait for additional taps after the first tap is detected. It may be set manually or calibrated using the Calibrate option in the i-Drive Configuration applications. No single- or double-tap input will be acknowledged until after time has expired; this delay should be set as short as possible based on how fast the user can reliably tap the input.

Output speed: This is the speed of the multi-tap pulse that is sent to the chair. The i-Drive system simulates a user inputting a multi-tap to the chair at a faster rate. This may be set manually in the i-Drive configuration application.



Intended Use

The i-Drive control system allows the use of alternative input devices to drive an electric wheelchair. Up to six input devices such as an egg switch, proximity pads, Sip & Puff, head array, and tray array are supported. The i-Drive system relies primarily on two types of input interpretations: switched and proportional. The interpretation used for each channel is dependent on the function assigned to it. The i-Drive system includes software that can display real-time diagnostics data, such as the desired direction of motion, to help setup and verify a configuration.

Included Parts

IDHBT500-1


	Description
A1	M5 x .8 x 6mm BHS* (2)
A2	Comfort Ring Mount
A3	#10-32 x 3/4" SHS (3)
A4	Power Cable
A5	Female Mono Dongles (2)
A6	#1 Phillips Machine Screw (2)

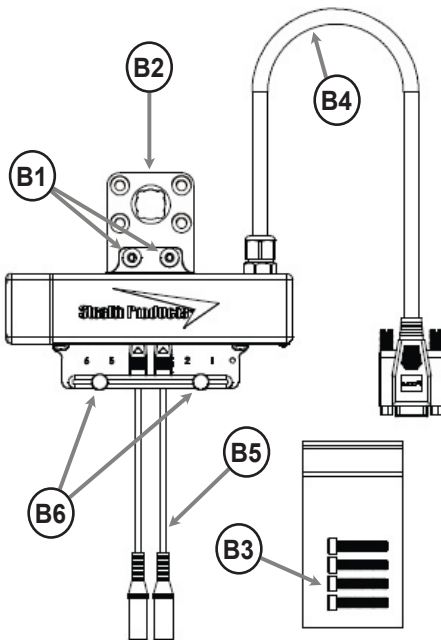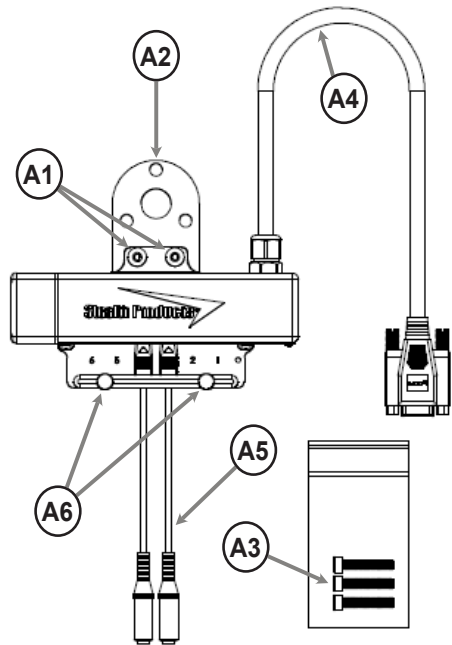

IDHBT500-17MM

	Description
B1	M5 x .8 x 6mm BHS* (2)
B2	Ring Mount
B3	M5 x 0.8 x 1" Socket Head Screw (4)
B4	Power Cable
B5	Female Mono Dongles (2)
B6	#1 Phillips Machine Screw (2)

Mounting Hardware Tools

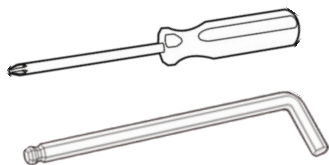

Required Tools
2.5mm Hex Wrench
4mm Hex Wrench
#1 Phillips Screwdriver




CAUTION

Use the proper tools to install or adjust the hardware. The use of improper tools may damage the hardware or parts and may lead to premature failure of the devices.

¹ Two Bluetooth dongles, one extension cable, and one USB plug are included with each interface package.

Port Assignment

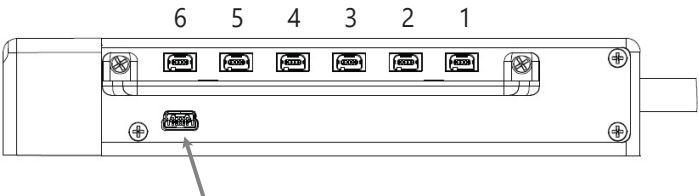

USB Port for configuration on PC

Port:	Direction Assignment
6	Left
5	Forward
4	Mode/Reset*
3	None*
2	Reverse
1	Right

* Ports 3 and 4 will always have dongles attached

The i-Drive 4.0 is capable of taking inputs from mechanical switches, proximity sensors, fiber optic sensors, joysticks, and Sip 'n' Puff modules. With the new i-Drive interface, the interface can be programmed with the associated Windows Configurator Software. Users now have the ability to download the i-Drive app and program the interface directly from a smartphone via *Bluetooth*[®] wireless technology.

i-Drive Bluetooth Minimum System Requirements

Processor	1 GHz
RAM	512 MB
Disk Space	7 MB
Operating System	Windows 8 or later (32-bit or 64-bit)
Screen Resolution	1024 x 768 minimum (1280x720 or larger recommended)
Additional Requirements	Internet access and a DAD account for configuration features, Bluetooth adapter, .NET framework 4.5.1

EMC Requirements

Electromagnetic compatibility (EMC) up to 20 V/m.



WARNING

Exposure to water can result in a short and/or intermittent failure of the device.

Control System Overview

The i-Drive system operates in different modes, including *RUN* and *CONFIG* modes. In *RUN* mode, changes to the settings *are not* allowed, and computed outputs *are* sent to the chair.

In *CONFIG* mode, settings changes *are* allowed, and computed outputs *are not* sent to the chair. To prevent unwanted input activations due to low voltage, the i-Drive system automatically switches to *CONFIG* mode if the system voltage falls below 10 volts.

Mode	Accepts Config Changes	Wheelchair Output
RUN	No	Yes
CONFIG	Yes	No
MOUSE	Yes	No (Sends to Bluetooth)

The USB cable allows an external device to communicate with i-Drive via the use of a virtual communications port. The Bluetooth interface allows an external device to communicate with the i-Drive via a secure wireless Bluetooth connection.



In either case, authentication is required for access.

Installing/Replacing the IDHBT500-1

Mounting IDHBT500 to the Comfort Ring Mount:

Step 1: Using a 2.5mm hex wrench, loosen the two (2) M5 x 0.8 x 6mm Button Head Screws (A1) from the IDHBT500.


Step 2: Place the tab on the interface into the recessed pocket opposite the mounting ring on the 1" mounting bracket. Ensure the mounting holes are aligned. Place the two (2) M5 x 0.8 x 6mm Button Head Screws (A1) into the mounting holes and tighten to secure.

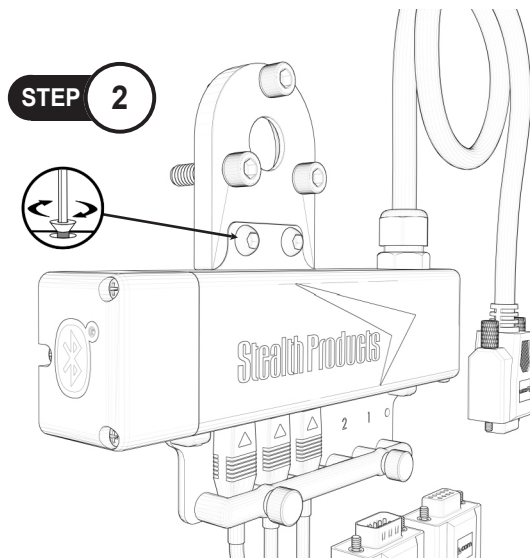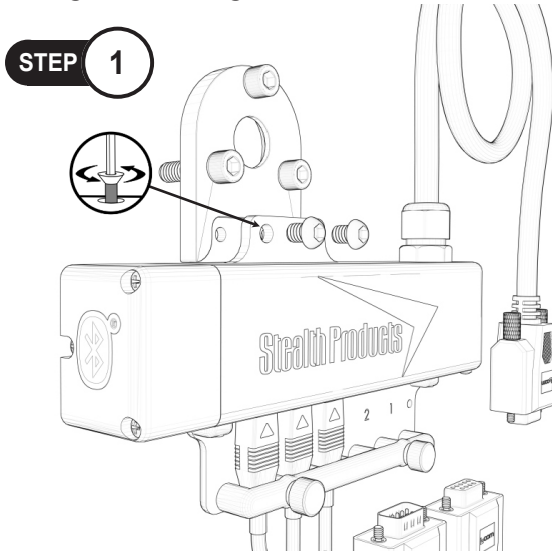

Uninstalling/Replacing IDHBT500-1

To uninstall the ball kit and interface:

Step 1: With a 3/16" hex key, completely loosen and remove the three (3) #10/32 x 3/4" SHS (A3) from the back of the headrest.

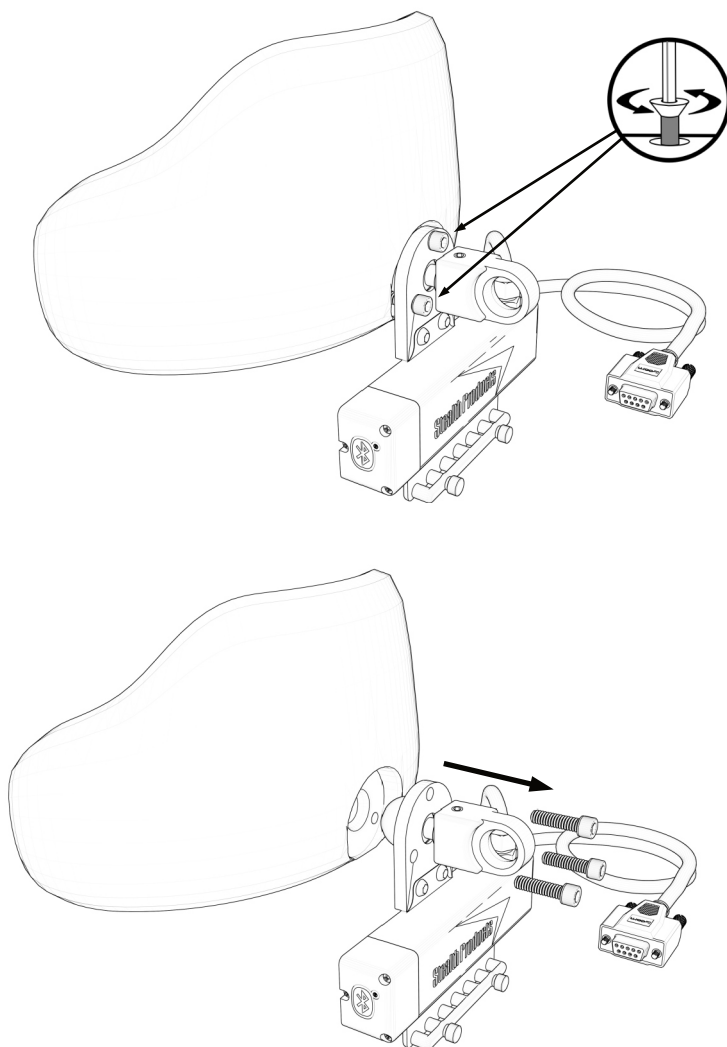

NOTICE



It may be easier to first remove the headrest from the client's chair before performing maintenance or replacing the interface.

(Note: 1" hardware is compatible with the Comfort Plus, Tri-Comfort Plus, Adjustable Comfort Plus, and Ottobock-style head supports.)

Step 2: With a 2.5mm hex wrench, remove the M5 set screw taper completely from the assembly.

Step 3: Loosen and remove the M5 set screw from the ball kit with a 2.5mm hex wrench.

Step 3: Once both set screws have been removed, the ball will freely move.

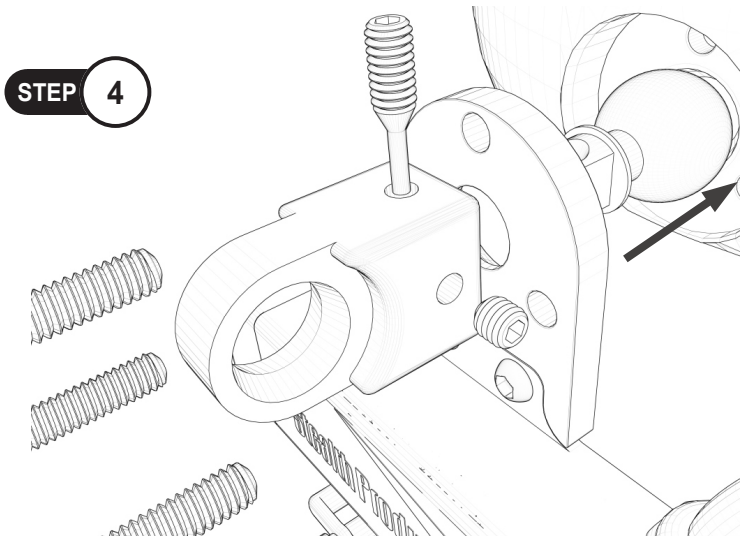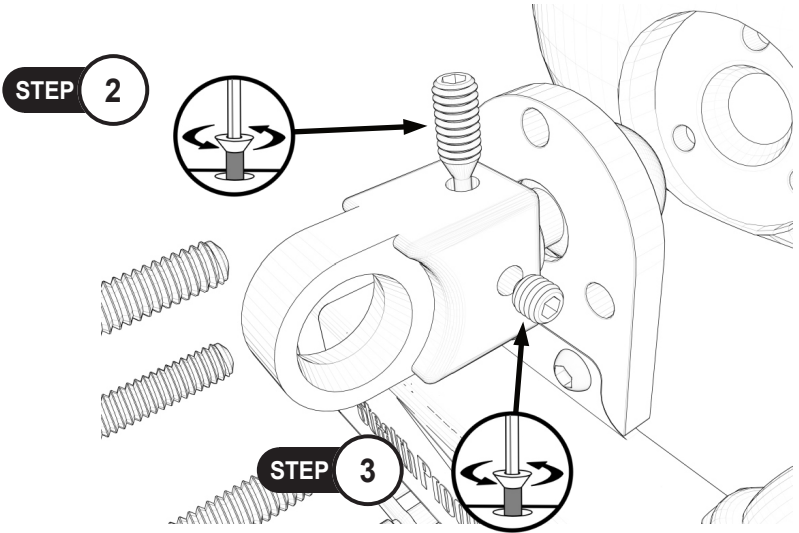


Installation Instructions

To reinstall the interface and ball kit:

Step 1: Put the 1" ball through the IDHBT500-1 interface mounting plate and into the hardware bracket.

Step 2: Ensure the M5 set screw taper is flush with the top of the bracket for proper installation.


Step 3: Insert M5 set screw and set accordingly.

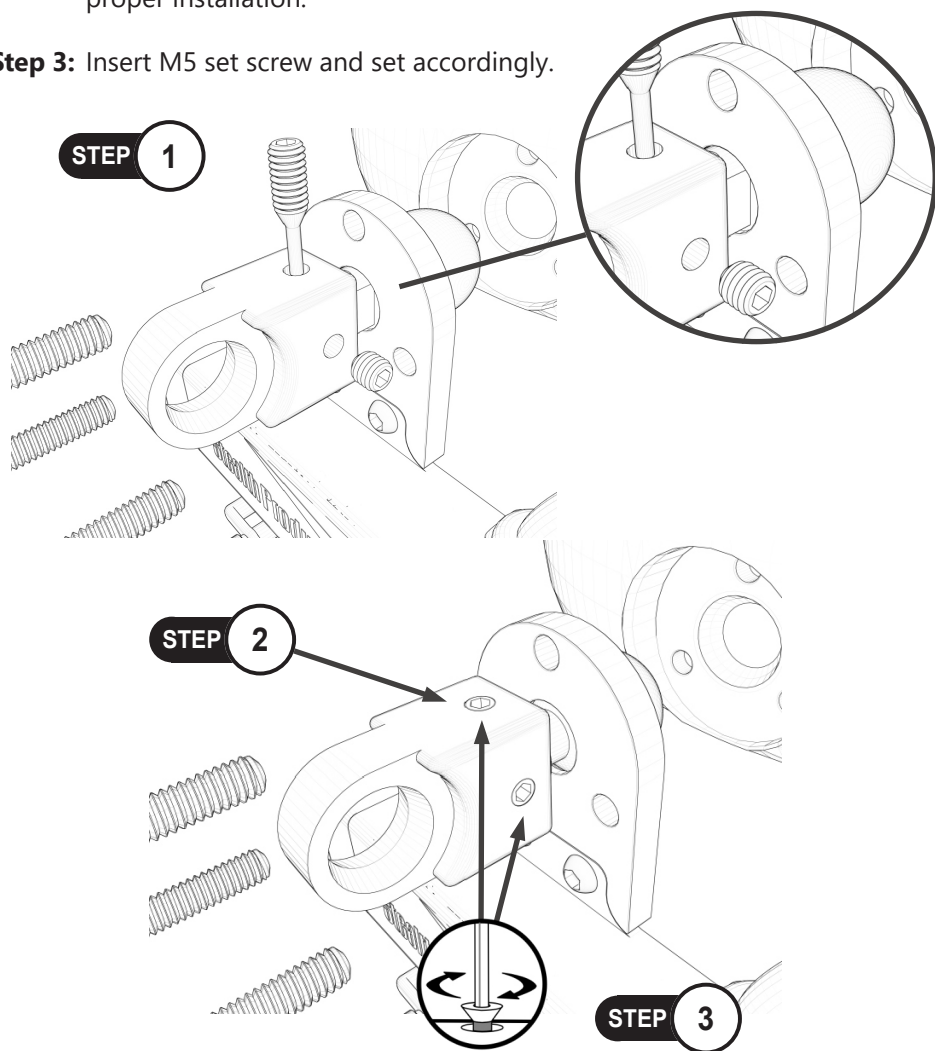

NOTICE

Ensure the logo on the interface is facing away from the headrest for proper installation as shown in the image above.

Step 4: Locate the three (3) #10-32 mounting screws (A3). Tighten screws evenly onto the headrest until snug and place the headrest back onto the client's chair.

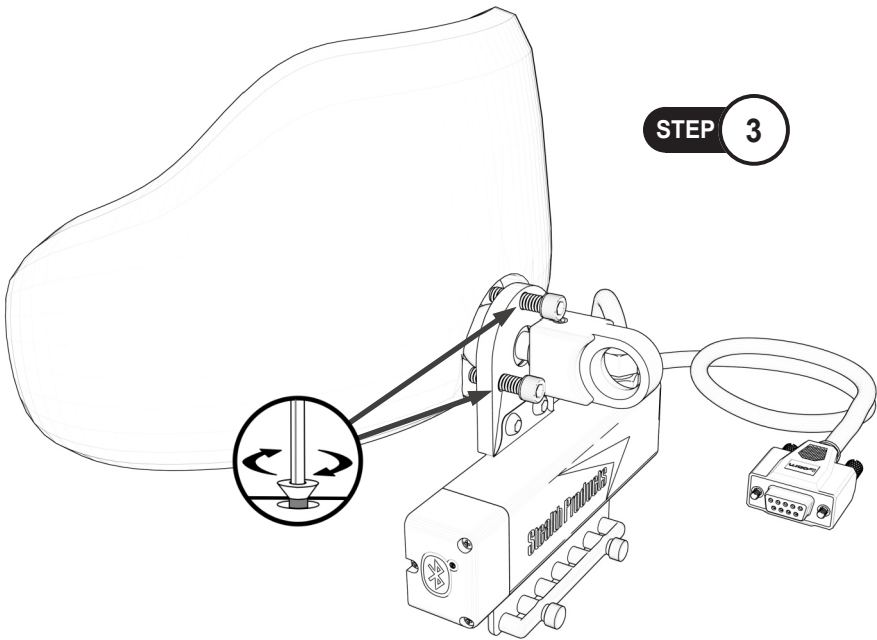

CAUTION

Tighten screws evenly on the hardware. Damage to the hardware could occur if the screws aren't secured evenly.



CAUTION

Removing the screws can cause the ring mount and interface to drop from the headrest. Exercise caution when removing screws.

NOTICE



Ensure all switches and dongles are disconnected from the interface to avoid damage to ports or connectors.

NOTICE

It is recommended to program the interface to the client's specifications before installing it on the chair.

Ring Mount Installation

Step 1: Align the mounting plate with the threaded holes in the back of the headrest. With the headrest and mounting plate held against the ball of the headrest hardware, slide the split ring mount behind the ball, aligned with the holes in the mounting plate.

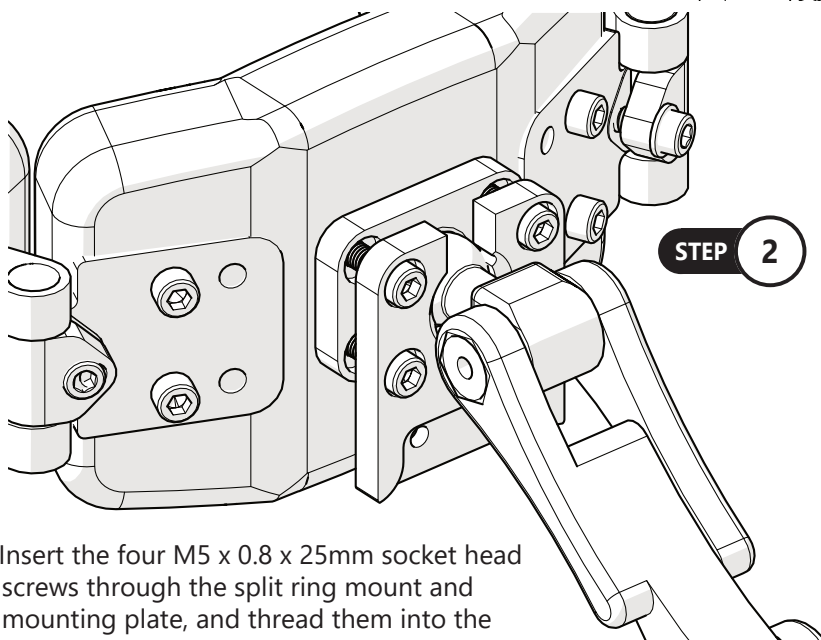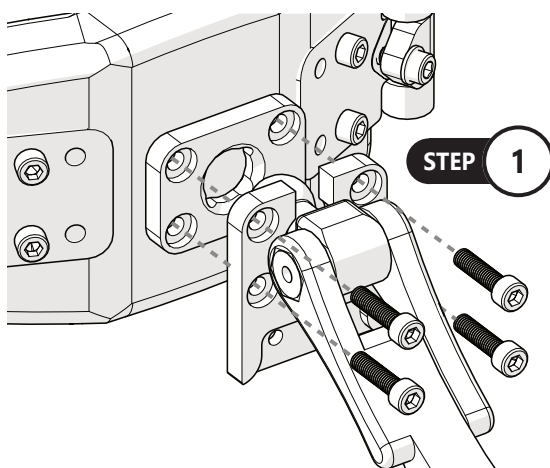



Step 2: Insert the four M5 x 0.8 x 25mm socket head screws through the split ring mount and mounting plate, and thread them into the threaded holes in the headrest pad. Tighten the screws evenly until the headrest pad is snug, but still able to be rotated or angled to allow access to mount the IDHBT interface.



i-Drive wires should be securely attached to the chair to avoid damage. Using cable ties to secure excess cords is recommended.


Installing the IDHBT500 Interface

Step 1: Align the mounting tab on the interface with the slot at the bottom of the split ring mount. Secure the inter-face to the ring mount with two M5 x 0.8 x 6mm button head screws. Optionally, rotate the headrest pad to allow easier access to install the interface.

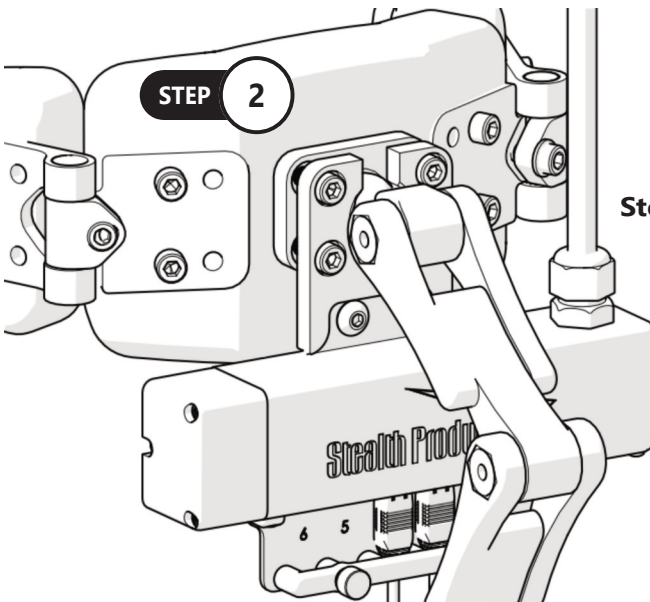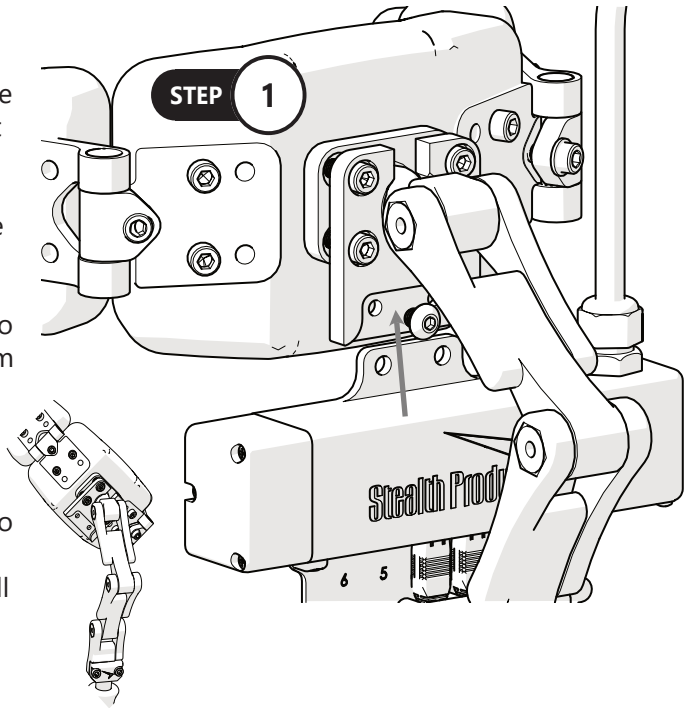


Step 2: Place the tab of the interface into the recessed pocket on the ring mount. Ensure the mounting holes are aligned, and secure the interface with the two (2) M5 x 0.8 x 6mm button head screws.

Uninstalling/Replacing the IDHBT500

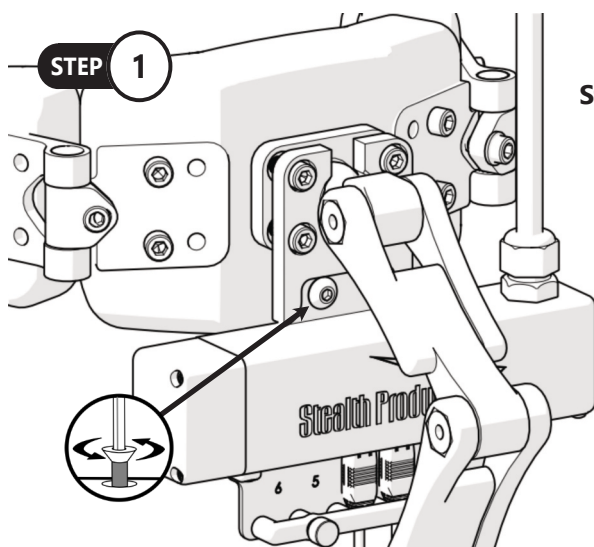

Step 1: Prior to removal, disconnect the IDHBT500 interface from the chair with the DB9 connector and disconnect any switches or proximity sensors. Be sure to note which ports the switches or sensors are connected to.

Step 2: Remove the M5 x 0.8 x 6mm screws to release the interface from the split ring mount. If replacing the interface, fasten the new one to the split ring mount using the M5 x 0.8 x 6mm screws. If removing the interface, replace the split ring mount with a regular split ring.

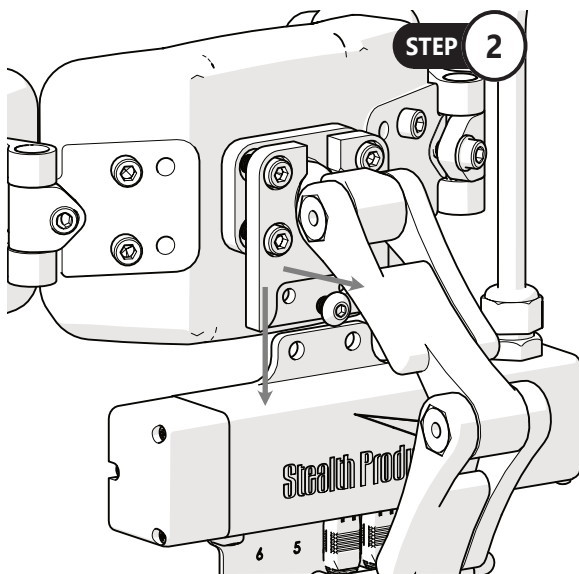

NOTICE

Ensure all switches and dongles are disconnected from the interface to avoid damage to ports or connectors.



CAUTION

Tighten screws evenly on the hardware. The hardware can sustain damage if its screws aren't secured evenly.

Replacing/Installing Dongles




Step 1: Loosen the #1 Phillips Machine Screws. Remove the retainer to expose the dongles plugged into the interface.

Step 2: To remove a dongle from the interface, grasp the micro-mini end of the dongle at the interface and pull firmly to remove.

To re-install a dongle, align the micro-mini plug with the arrow on the plug facing up, and gently push it into the correct port.

If a switch or sensor is malfunctioning, replace the dongle with a known good one. If the problem resolves, discard the defective dongle and replace with a new one.

Step 3: When changes are complete, replace and tighten the #1 Phillips Machine Screws. If the retainer is difficult to install or does not fit in place, check that all cables are plugged in properly, and try again.

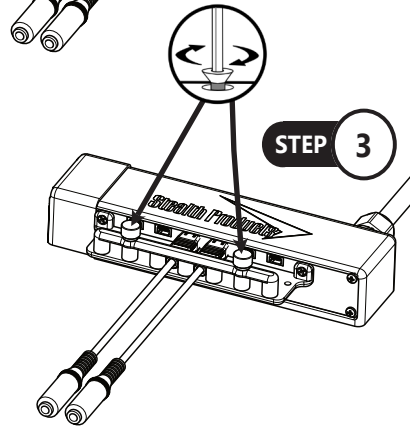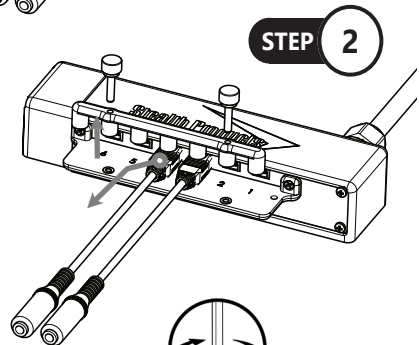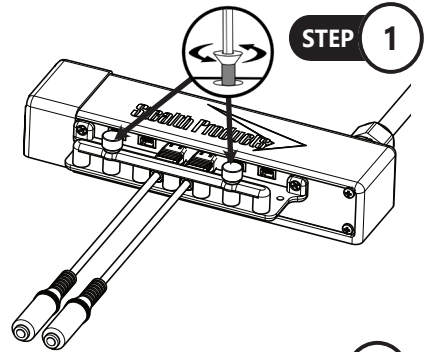

DANGER

Ensure the dongles and/or sensors are plugged into their proper ports. Attaching a dongle or sensor to an incorrect port could lead to unexpected movements or changes in chair direction.



WARNING

When reattaching the dongle, perform a diagnostic check to ensure the dongle is reading correctly.




WARNING

Use a #1 Phillips screwdriver for the machine screws to avoid stripping them.

R-net™ Omni Setup

NOTICE

An R-net™ programming dongle is required.

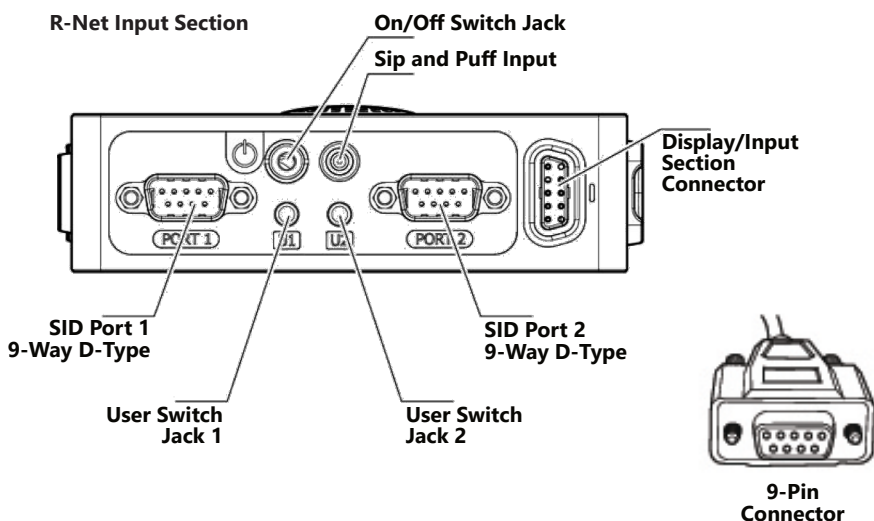

R-net Setup: Switched Output



WARNING

Manually disengage both drive motors before utilizing the Setup Wizard, Diagnostics, and/or Config. If the motors are not disengaged, the chair could move during setup.

1. Power off the chair using the power button on the joystick or display.
2. Connect the R-net programming dongle into the system.
3. Ensure all display cords are properly connected to the chair.
4. Connect i-Drive alternative control into Port 1 or Port 2 on the bottom of the display with a 9-pin connector.
5. With the chair powered off, open the app on a Bluetooth-enabled device and log in using the email and password provided to you. Power the chair on and connect to the application.
6. Press the *Mode* button until you enter the *On Board Programming (OBP)* menu.
7. In the *OBP* menu, use the *Up* and *Down* buttons to navigate to the *Omni* menu option. Use the *Right* button to select this option.
8. In the *Omni* menu, use the *Up* and *Down* buttons to navigate to the *Port 1* or *Port 2* option. Use the *Right* button to select this option. Ensure your chair is programmed for the port to which your i-Drive is connected.
9. In the *Omni* menu, navigate to the port to which you have the i-Drive connected (*Port 1* or *Port 2*).
10. In the *Port* menu, use the *Up* and *Down* buttons to navigate to the *SID* option, toggle to:
11. In the *Port* menu, use the *Up* and *Down* buttons to navigate to the *Switches* option. Use the *Right* button to select this option.
 - a) *3swi* for any configuration that does not have a reverse direction, such as a 3-switch head array or linked left/right driving.
 - b) *Swi* for any configuration that includes an assigned reverse direction, such as a 5-switch head array, a Sip 'n' Puff combo, or a PMPJ-1.
12. In the *Switches* menu, use the *Up* and *Down* button to navigate to the *Switch Detect* option. Use the *Up* and *Down* buttons to set this option to *Off*.
13. Press the *Left* button to return to the *Port* menu. Use the *Up* and *Down* button to navigate to the *Controls* option, then use the *Right* button to select it.
14. Scroll to the *Actuator Axis* option. Use the (+) and (-) buttons to set the appropriate selection:
 - a) *L/R* - a left *SID* command will select the available actuator axes and a right *SID* command will move the selected axis. The direction of movement is changed via a short operation by a right *SID* command.
 - b) *R/L* - a right *SID* command will select the available actuator axes and a left *SID* command will move the selected axis. The direction of movement is changed via a short operation by a left *SID* command.
 - c) *Swap* - forward and reverse *SID* commands will select the available actuator axes, and left and right commands will move the selected axis.
 - d) *Normal* - left and right *SID* commands will select the available actuator axes. Forward and reverse commands will move the selected axis.

Q-Logic™ Setup

Bookmark Buttons

Navigation


Arrows navigate the main menu. Up and Down navigate the menus, Right to open a menu item and Left to return to the previous menu.

Plus and Minus


Toggle settings or changes values of the highlighted parameter

Help Button

Displays information about options selected on the display.

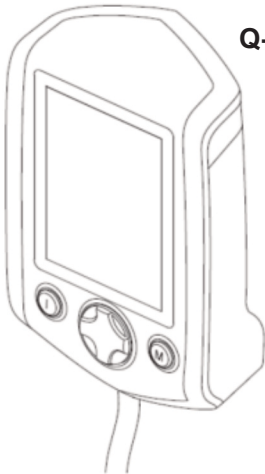

Q-Logic Enhanced Display

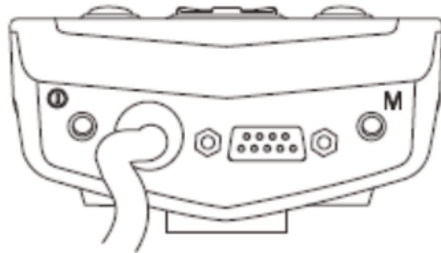

(**Note:** Q-Logic™ Enhanced Display and Q-Logic™ Handheld Programmer Required)

Q-Logic 2 Setup: Proportional Output



WARNING

Manually disengage both drive motors before utilizing the *Setup Wizard*, *Diagnostics*, and/or *Config*. If the motors are not disengaged, the chair could move during setup.

1. Power off the chair using the power button on the joystick or display.
2. Ensure all display cords are properly connected to the chair.
3. Connect the i-Drive alternative control into the port on the bottom of the display with the 9-pin connector.
4. With the chair powered off, open the app on a Bluetooth-enabled device and log in using the email and password provided to you. Power on the chair and connect to the application.
5. Plug in the Q-Logic Programmer (HHP) to the back of the Q-Logic display.
6. On the HHP, select the *Program Adjustments* option.
7. In the *Program Adjustment* menu, use the *Up* and *Down* buttons to navigate to the *Specialty Control* option. Use the *Right* button to select this option.
8. In the *Specialty Control* menu, use the *Up* and *Down* buttons to navigate to the *Active Device* option. Use the *Plus* and *Minus* buttons to toggle to:
 - a) *Proportional Head 3-Direction* for any configuration that does not have a reverse direction assigned, such as a 3-switch head array or linked left/right driving.
 - b) *Proportional* for any configuration that includes an assigned reverse direction, such as a 5-switch head array, a Sip'n'Puff combo, or a PMPJ-1.
9. Use the *Up* and *Down* buttons to scroll to the *Proportional Head-3 Direction* folder. Use the *Right* button to select and enter the folder. Scroll down to the *Joystick Setup* folder and use the *Right* button to select it.
10. In the *Joystick Setup* menu, use the *Up* and *Down* buttons to scroll down to the *Calibrate* option. Select this option and click *OK*.
11. The Q-Logic programmer will read, "Roll around joystick in two full circles."
12. On the phone, tablet or PC device, go to the i-Drive application. Select the *Config Mode* option. Select *Calibrate Chair* and click *OK*.
13. Wait until the calibration finalizes, then select *OK* with the Q-Logic programmer and verify the calibration was successful. (If calibration was not successful, power off the chair and repeat Steps 5-12.)
14. Return to the main menu and exit the Q-Logic programming.
15. Unplug the Q-Logic handheld programmer.
16. Power off the chair.
17. The chair is now programmed to recognize the i-Drive. Power the chair back on.
18. Check your configuration on the *Diagnostics* tab. If everything is working correctly, your i-Drive is ready for use. Re-engage your motors.



WARNING

The chair *must* be completely powered off before operation.

Q-Logic 2 Setup: Switched Output



WARNING

Manually disengage both drive motors before utilizing the *Setup Wizard*, *Diagnostics*, and/or *Config*. If the motors are not disengaged, the chair could move during setup.

1. Power off the chair with the power button on the joystick or display.
2. Ensure all display cords are properly connected to the chair.
3. Connect the i-Drive alternative control into the port on the bottom of the display with the 9-pin connector.
4. With the chair powered off, open the app on a Bluetooth-enabled device and log in using the email and password provided to you. Power on the chair and connect to the application.
5. Plug in the Q-Logic Programmer (HHP) to the back of the Q-Logic display.
6. On the HHP, select the *Program Adjustments* option.
7. Use the *Up* and *Down* buttons to navigate to the *Specialty Control* option, then select it with the *Right* button.
8. Use the navigation buttons to set this option to the setting that most closely matches the desired input configuration:
 - a) 2-Switch for any configuration utilizing *Linked L/R* mode.
 - b) 3-Switch for any tray array or IDPMPJ-1 combination with no assigned reverse, and for *Linked L/R* mode.
 - c) 4-Switch for any tray array combination with an assigned reverse.
 - d) 5-Switch for any tray array combination with an assigned reverse and mode change option.
 - e) 3-Switch Head for any 3-switch head array or a head array with Sip'n'Puff and no assigned reverse.
 - f) 4-Switch Head for any 4-switch head array, a head array with Sip'n'Puff and an assigned reverse, or any IDPMPJ-1/head array combination.
 - g) 5-Switch Head for any 4-switch head array, a head array with Sip'n'Puff and an assigned reverse, or any IDPMPJ-1/head array with a mode change option.



WARNING

The chair *must* be completely powered off before operation.

Q-Logic 2 Further Configuration

Back Pad Toggle

If you want to disable or enable the back pad's ability to toggle into forward or reverse:

- Reconnect the Q-Logic handheld programmer. Follow steps 1-7 from *Setup*.
- On the Q-Logic Programmer, select the *Program Adjustments* option.
- In the *Program Adjustments* folder, use the *Up* and *Down* buttons to navigate to the *Specialty Control* option, and use the *Right* button to select it.
- In the *Specialty Control* menu, use the *Up* and *Down* buttons to navigate to the *Proportional Head 3-Direction* option. Use the *Right* button to select it.
- In the *3-Switch Head* menu, use the *Up* and *Down* buttons to navigate to the *Back Switch Toggle* option. Use the *Right* button to select it.
- In the *Back Switch Toggle* menu, use the *Up* and *Down* buttons to navigate to the *m: Back Toggle* option, and use the *Plus* and *Minus* buttons to toggle to and select the *Enable* option. To disable, toggle to the *Disable* option and select it.
- After configuring the back pad, disconnect the Q-Logic programmer and restart the chair's electronics (follow steps 10-13), or continue to configure other operations.

Side Pad Toggle/Control

To disable or enable mode switching with the side pad.

- Reconnect the Q-Logic handheld programmer. (Follow steps 1-7 from *Setup*.)
- On the Q-Logic Programmer, select the *Program Adjustments* option.
- In the *Program Adjustments* menu, use the *Up* and *Down* buttons to navigate to the *Specialty Control* option, and use the *Right* button to select it.
- In the *Specialty Control* menu, use the *Up* and *Down* buttons to navigate to the *Proportional Head 3-Direction* option, and use the *Right* button to select it.
- In the *3-Switch Head* menu, use the *Up* and *Down* buttons to navigate to the *Device Options/ Timing* option, and use the *Right* button to select it.
- In the *Device Options/Timing* menu, use the *Up* and *Down* buttons to navigate to the *m:Device Double Command* option. Use the *Plus* and *Minus* buttons to toggle to the *Enable* option. To disable, toggle to the *Disable* option and select it.

Egg Switch Function

If you would like to change the function of the Egg Switch to Mode or Toggle Forward and Reverse:

- Reconnect the Q-Logic handheld programmer. (Follow steps 1-7 from *Setup*.)
- On the Q-Logic Programmer, select the *Program Adjustments* option.
- In the *Program Adjustments* menu, use the *Up* and *Down* buttons to navigate to the *Specialty Control* option, and use the *Right* button to select it.
- In the *Specialty Control* menu, use the *Up* and *Down* buttons to navigate to the *Proportional Head 3-Direction* option, and use the *Right* button to select it.
- In the *Proportional Head 3-Direction* menu, use the *Up* and *Down* buttons to navigate to the *Switch Options* option, and use the *Right* button to select it.
- In the *Switch Options/Timing* menu, use the *Up* and *Down* buttons to navigate to the *m:Mode Jack Switch Type* option. Use the *Plus* and *Minus* buttons to toggle to *Mode* to enable mode switch with the Egg Switch, or *Toggle* for forward/reverse toggle.
- After configuring the Egg Switch, disconnect the Q-Logic programmer and restart the chair's electronics (follow steps 10-13), or continue to configure other operations.

Q-Logic™ 3: Proportional Output



WARNING

Manually disengage both drive motors before utilizing the *Setup Wizard*, *Diagnostics*, and/or *Config*. If the motors are not disengaged, the chair could move during setup.

1. Power off the chair using the power button on the joystick or display.
2. Ensure all display cords are properly connected to the chair.
3. Connect the i-Drive 9-pin connection into the port on the bottom of the Enhanced Display.
4. With the chair powered off, open the i-Drive mobile app on a Bluetooth-enabled device and log in using the email and password provided to you. Power the chair on and connect to the application.
5. Plug in the Q-Logic Programmer (HHP) to the back of the Q-Logic display.
6. On the Programmer, navigate to *Program Adjustments* option and use the *Right* button to select it.
7. In the *Program Adjustment* menu, navigate to the *Enhanced Display* option and select it.
8. In the *Enhanced Display* folder, use the *Up* and *Down* buttons to navigate to the *Input Configuration* option. Set this to:
 - a) *3-Direction Proportional Head* for any configuration that does not have a reverse direction assigned, such as a 3-switch head array or linked left/right driving.
 - b) *Proportional* for any configuration that includes an assigned reverse direction, such as a 5-switch head array, a Sip 'n' Puff combo, or a PMPJ-1.
9. Use the *Left* navigation button to go back into the *Enhanced Display* folder.
10. Use the *Up* and *Down* buttons to scroll to the *Proportional Input* folder and use the *Right* button to select it.
11. In the *Proportional Input* folder, use the *Up* and *Down* buttons to navigate to and select the *Proportional Calibrations* option. Select *OK*. The programmer will read, "Roll around joystick in two full circles".
12. On your i-Drive mobile app, go to *Config Mode*, then *Calibrate Chair*.
13. Note the warning that appears and select *OK*. The app will now automatically begin calibration. Once calibration is complete, the app will automatically return to *Config Mode*.
14. Using the Q-Logic programmer, select *OK* and verify that calibration was successful.
15. Exit the programming. Unplug the programmer and cycle power.
16. Check your configuration on the app's *Diagnostics* tab. If everything is functioning correctly, your i-Drive is ready to use.



WARNING

The chair must be completely powered off before operation.

Q-Logic™ 3 Setup: Switched Output

1. Power off the chair using the power button on the joystick or display.
2. Ensure all display cords are properly connected to the chair.
3. Connect the i-Drive alternative control into the port on the bottom of the display with the 9-pin connector.
4. With the chair powered off, open the app on a Bluetooth-enabled device and log in using the email and password provided to you. Power on the chair and connect to the application.
5. Plug in the Q-Logic Programmer (HHP) to the back of the Q-Logic display.
6. On the HHP, select the *Program Adjustments* option.
7. Use the Up and Down buttons to navigate to the *Enhanced Display* (or SCIM) option, then select it with the *Right* button.
8. Scroll down to reach the *Input Configuration* option, then use the *Right* button to select it. Use the navigation buttons to set this option to the setting that most closely matches the desired input configuration:
 - a) *2-Switch* for any configuration utilizing *Linked L/R* mode.
 - b) *3-Switch* for any tray array or IDPMPJ-1 combination with no assigned reverse, and for *Linked L/R* mode.
 - c) *4-Switch* for any IDPMPJ-1 combination or a tray array with an assigned reverse.
 - d) *5-Switch* for any IDPMPJ-1 combination or a tray array with an assigned reverse.
 - e) *3-Switch Head* for any 3-switch head array or a head array with Sip 'n' Puff and no assigned reverse.
 - f) *4-Switch Head* for any 4-switch head array, a head array with Sip 'n' Puff and an assigned reverse, or any IDPMPJ-1/head array combination.
 - g) *5-Switch Head* for any 5-switch head array, a head array with Sip 'n' Puff and an assigned reverse, or any IDPMPJ-1/head array combination with a mode-change option.

Q-Logic 3 Further Configuration

Back Pad Toggle

If you want to disable or enable the back pad's ability to toggle into forward or reverse:

1. On the Q-Logic Programmer, select the *Program Adjustments* option.
2. In the *Program Adjustments* folder, use the *Up* and *Down* buttons to navigate to the *Enhanced Display* option, and use the *Right* button to select it.
3. In the *Enhanced Display* menu, use the *Up* and *Down* buttons to navigate to the *3-Direction Arrow Toggle Settings* option. Use the *Right* button to select it.
4. In the *3-Direction Arrow Toggle Settings* menu, use the *Up* and *Down* buttons to navigate to the *Toggle Command* option. Use the *Right* button to select it.
5. In the *Toggle Command* menu, use the *Plus* and *Minus* buttons to toggle to and select the *Enable* option. To disable, toggle to the *Disable* option and select it.
6. After configuring the back pad, disconnect the Q-Logic programmer and restart the chair's electronics (follow Steps 10-13), or continue to configure other operations.



WARNING

Manually disengage both drive motors before utilizing the Setup Wizard, Diagnostics, and/or Config. If the motors are not disengaged, the chair could move during setup.

Double Command

If you wish to enable a double activation to perform a Mode command:

1. On the Q-Logic Programmer, select the *Program Adjustments* option.
2. In the *Program Adjustments* folder, use the *Up* and *Down* buttons to navigate to the *Enhanced Display* option, and use the *Right* button to select it.
3. In the *Enhanced Display* menu, use the *Up* and *Down* buttons to navigate to the *Device Options/Timing* option. Use the *Right* button to select it.
4. In the *Device Options/Timing* menu, use the *Up* and *Down* buttons to navigate to the *Device Double Command* option. Use the *Right* button to select it.
5. In the *Device Double Command* menu, use the *Plus* and *Minus* buttons to toggle to and select either the *Enabled Right* or *Enabled Left* option.
6. After configuring the back pad, disconnect the Q-Logic programmer and restart the chair's electronics (follow Steps 10-13), or continue to configure other operations.

Tilt Threshold Head Control

To define the angle beyond which a head input device will be changed to two-way operation:

1. On the Q-Logic Programmer, select the *Program Adjustments* option.
2. In the *Program Adjustments* folder, use the *Up* and *Down* buttons to navigate to the *Seat* option, and use the *Right* button to select it.
3. In the *Seat* menu, use the *Up* and *Down* buttons to navigate to the *Tilt Threshold Head Control* option, and use the *Right* button to select it.
4. Use the *Plus* and *Minus* buttons to select the desired tilt angle.

Egg Switch Function

If you would like to change the function of the Egg Switch (plugged into the Enhanced Display):*

1. On the Q-Logic programmer, select the *Program Adjustments* option.
2. In the *Program Adjustments* menu, use the *Up* and *Down* buttons to navigate to the *Enhanced Display* (or SCIM) option, and use the *Right* button to select it.

The modes for switch commands can be adjusted for each jack input. The available options for switch commands are Mode, Mode Shortcut, Toggle, Sleep, Home, and Inactive.**

To adjust, for example, the switch command mode for Jack 1:

1. In the *Enhanced Display/SCIM* menu, use the *Up* and *Down* buttons to navigate to the *Jack Commands* option, and use the *Right* button to select it. Scroll to select Jack 1.
 - a) In the *Jack 1* menu, select *Mode Jack Short Command*. Use the *Plus* and *Minus* buttons to adjust this to the desired setting.
 - b) In the *Jack 1* menu, select *Mode Jack Long Command*. Use the *Plus* and *Minus* buttons to adjust this to the desired setting.
 - c) In the *Jack 1* menu, select *Mode Jack Double Command*. Use the *Plus* and *Minus* buttons to adjust this to the desired setting.

* When switching modes through a 9-pin plug or when the switch is plugged into the i-Drive, select D-Sub (9-pin) in the *Enhanced Display/SCIM* menu, then proceed as detailed above.

** If the option for a jack is set to *Inactive*, no switch command (short, long, or double) will be recognized for that parameter.

MK6i™ Programmer Setup

 **WARNING**

Completely disengage motors before programming any wheelchair. Failing to do so can cause serious injury and property damage.

PWR/INFO

Turn the programmer on and off, and display Help information while the programmer is on.

Navigation Keys


Up, Down, Left, and Right buttons scroll through menu structure and increase/decrease values.

SELECT

Display adjustable values or selection choices, choose new values, and/or begin memory card transfers.

SAVE

Press twice to save highlighted values.

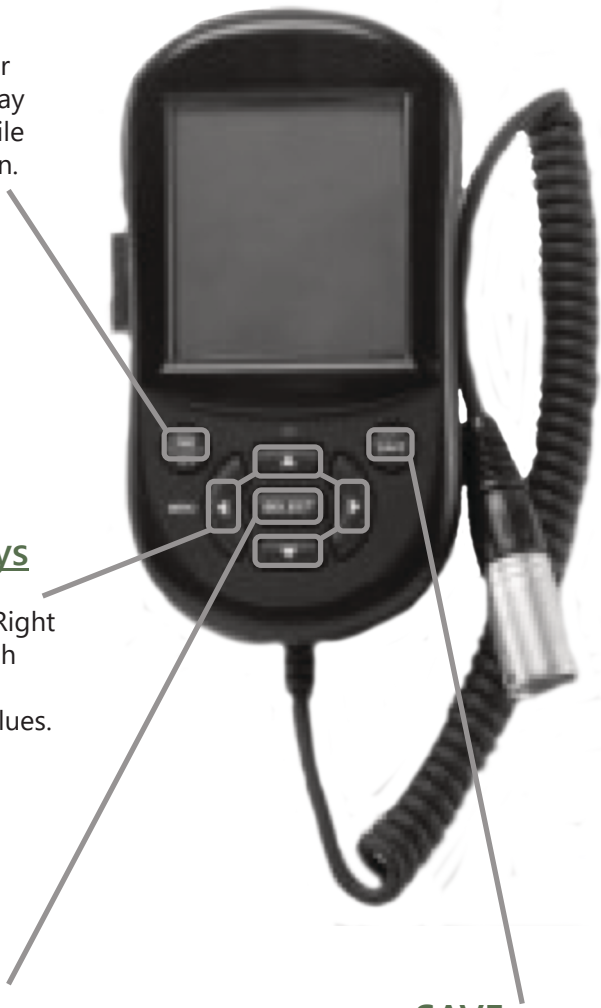

MK6i Setup: Switched Output



WARNING

Manually disengage both drive motors before utilizing the Setup Wizard, Diagnostics, and/or Config. If the motors are not disengaged, the chair could move during setup.

1. Power off the chair using the power button on the joystick or display. Disengage all motors.
2. Ensure all display cords are properly connected to the chair. Press the chair display power button to turn the chair on.
3. Plug the MK6i programmer into the back of the joystick. Turn the programmer on.
4. On the programmer, use the *Up* and *Down* buttons to scroll to the *Performance Adjust* option, then select it with the *Right* button.
5. In the *Performance Adjust* menu, use the *Up* and *Down* buttons to scroll to the *Input Type* option. If you wish to select all drive profiles (D1-D4), press the *Select* button to do so. If you wish to select a specific drive profile, use the *Left* and *Right* buttons to navigate to the desired profile. Highlight the profile and press the *Select* button.
6. After selecting the drive profile(s), use the *Up* and *Down* buttons to scroll to the *Digital* option from the i-Drive list, then press the *Select* button to select it.
7. If you would like to enable *Reverse Mode* with the reset switch, access the *Performance Adjust* menu and use the *Up* and *Down* buttons to scroll to the *RIM Control* option, then set the desired drive profile's *RIM Control* to *On* or *Off*.
8. Use the *Save* button to save your current settings. (A prompt should appear - press *Save* again to save, or to cancel, select *Cancel* with the *Select* button.)
9. After saving, the screen will display that the save was successful. Power off the programmer or continue to make further adjustments.
10. Unplug the programmer. Turn off the chair. (The chair is now programmed to recognize the i-Drive.)

Powered Seating Control

1. Reconnect the MK6i programmer.
2. On the programmer, use the *Up* and *Down* buttons to scroll to the *Powered Seating* option, then use the *Right* button to select it.
3. In the *Powered Seating* menu, use the *Up* and *Down* buttons to scroll to the *Actuator Control* option, then set your desired drive profile's *Actuator Controls* to an actuator control type.
4. Use the *Save* button to save your current settings. (A prompt should appear - press *Save* again to save, or to cancel, select *Cancel* with the *Select* button.)
5. After saving, the screen will display that the save was successful. Power off the programmer or continue to make further adjustments.
6. Unplug the programmer. Turn off the chair.



WARNING

The chair *must* be completely powered off before operation.

Troubleshooting

SYMPTOM	POSS. CAUSE	POSS. REMEDY	NOTES
No reaction from i-Drive	No power to i-Drive	Check Cables	- Find where power ends. - Ensure all connection cables are fully connected. - Check the cables for possible damage. - Try removing DB9 extension and plugging in without. - Using APS, plug in USB, go to diagnostics and see if i-Drive shows any voltage.
No reaction from i-Drive; can connect to i-Drive programming	i-Drive is in Config Mode	Put programming in another mode	i-Drive will not send a signal to the device in Config Mode. It must be in the Diagnostics view to send a signal.
	Inappropriate settings	Check Channel Assignments	Make sure that the channel is set for the appropriate assignment
		Check input settings on chair	- Check the setup guide - If the chair is set for a switched input device, the i-Drive may not function or may behave strangely.
No reaction from channel(s)	Inappropriate settings	Check channel assignments	Make sure that the channel is set for the appropriate assignment
	Inappropriate input	Check switches	Try an established, functioning switch in the place of the non-reactive) switch or sensor
		Check dongle	Try an established, functioning dongle in the non-reactive channel
		Check location	Ensure sensors & mechanicals are in the appropriate locations for old versions of i-Drive.
Difficult of inconsistent activations from individual channel.	User positioning issues	Ensure user is positioned correctly	Proper and consistent user position is key to consistent successful activations.
	Switch positioning issues	Ensure switch or sensor is in the proper location	Proper and consistent switch and sensor positioning is key to consistent successful activations
	Joystick calibration issues	Recalibrate joysticks	- Try recalibrating the PMPJ - Make sure all axes respond to deflections
	Sensitivity adjustment issues	Adjust sensitivity	Try slightly increasing the sensitivity on sensors, fiber optics, etc.
	Obstruction	Ensure the switch is making good contact	Is there anything the user is wearing obstructing contact?

Installation Instructions

SYMPTOM	POSSIBLE CAUSE	POSSIBLE REMEDY	NOTES
Channel(s) constantly on or activated	Cable Damage	Inspect cables and connections for damage	- Look for breaks, cuts, or tears in the cable, or if the cable is being crimped or restricted. - Try an established, functioning switch in the place of the non-reactive switch or sensor.
	Adjustment issues	Check the adjustment screw	i-Drive will not send a signal to the device in Config Mode. It must be in the Diagnostics view to send a signal.
	Issues with the cover	Check the cover for conductive materials	- Check for moisture on the cover. - Check for metallic objects or other conductive materials - Remove cover fully, then fully replace it.
Difficulty calibrating i-Drive to chair	Inappropriate sequence	recalibrate i-Drive to chair	- See the setup guide. - Ensure that the chair is placed into Calibration Mode, then i-Drive.
	Problem with DB-9 extension	Remove DB-9 and recalibrate i-Drive to chair	Occasionally a DB-9 extension will cause issues with calibration. Remove the DB-9 and plug the interface straight into the 9-pin port, no extension, and recalibrate.
Channel responds incorrectly	Inappropriate settings	Check channel assignments	Ensure the channel is set for the appropriate assignment
	Calibration issue	Recalibrate i-Drive to chair	Calibration might have been effective enough for the chair to function, but not precise enough
PMPJ-1 causes out of center error on chair	Calibration issue	Calibrate joystick	Calibrate joystick to i-Drive (Config Mode > Calibrate joystick)
Difficulty using proportional Sip 'n' Puff	Calibration issue	Calibrate Sip 'n' Puff	Please refer to the Sip 'n' Puff Owner's Manual
	Sip zones need adjustment	Adjust sip zones	Please refer to the Sip 'n' Puff Owner's Manual
	Hard detection time too fast	Adjust hard detection time	Please refer to the Sip 'n' Puff Owner's Manual
	Puff zones need adjustment	Adjust puff zones	Please refer to the Sip 'n' Puff Owner's Manual

SYMPTOM	POSSIBLE CAUSE	POSSIBLE REMEDY	NOTES
Difficulty toggling forward/reverse in Linked L/R Mode	Inappropriate settings	Check input settings on chair	- See setup guide - If the chair is set for a switched input device, the i-Drive may not function correctly
	Problem with R-net system	N/A	R-net requires an extremely quick and accurate click for reverse. It is highly difficult to consistently toggle forward/reverse
Invalid login credentials	Inaccurate entry	Ensure client has correct login information	Ensure the information provided and entered is accurate
	Issues with local app data	Manually remove login credentials for Windows APS	Delete any folders that begin with 'iDrivearray.exe'
APS/Windows app will not connect	Device not recognized	Update PC drivers	Download and install the USB drivers available from the download page
	APS is out of date	Download newest software	Older versions will not connect to 4.0 interfaces
	Communication error	Cycle power	'Wait 10 min' error displays if software does not receive an i-Drive response.
		Use a new USB cable	
Windows app/APS USB will not display a connectable device	Device not recognized	Establish a new COM port	In the device manager, uninstall the current i-Drive COM port.
		Update PC drivers	
APS Bluetooth will not display a connectable device	Device not connected	Pair device to PC Bluetooth	First connect with the PC Bluetooth, then connect in APS
	i-Drive is paired elsewhere	Disconnect and reconnect device	Disconnect from all mobile apps and PC's in range, then cycle the power and retry
PC Bluetooth will not pair with i-Drive	Incorrect entry of serial number	Enter serial number again	'Forget' the device, reconnect and try again
		Adjust hard detection time	Please refer to the Sip 'n' Puff Owner's Manual
Mobile app will not display a connectable device	i-Drive is paired elsewhere	Disconnect and reconnect device	Disconnect from all mobile apps and PC's in range, then cycle the power on the i-Drive and retry
Mobile app will not connect to system	Location services disabled	Enable location services	Go to your phone settings, then the app settings, then 'i-Drive Mobile', and enable location services.
	Incorrect entry of serial number	Enter serial number again	'Forget' the device, reconnect and try again
		Connect to the device via USB	Go to Diagnostics and ensure the serial number programmed matches the serial number sticker

Installation Instruction

SYMPTOM	POSSIBLE CAUSE	POSSIBLE REMEDY	NOTES
Mobile app will not allow for changes	Incorrect entry of serial number	Enter serial number again	• <i>'Forget' the device, reconnect, and try again</i>
		Connect to the device via USB	• Go to diagnostics and ensure that the serial number programmed matches the serial number sticker
Mobile app crashes when used	Incorrect entry of serial number	Enter serial number again	• <i>'Forget' the device, reconnect, and try again</i>
		Connect to the device via USB	• Go to diagnostics and ensure that the serial number programmed matches the serial number sticker
'Loonz' will not connect to i-Drive	Problem with firmware	Use latest version of firmware	• Update to Version 4.5 if necessary
	'Loonz' not connected	Pair device to PC Bluetooth	• <i>Connect with the PC Bluetooth first then connect with the APS</i>
	Mouse emulation connected	Disconnect mouse	• These two features cannot be paired simultaneously
'Loonz' has no reverse	Reverse has not been assigned	Double Tap	A double tap will toggle to reverse if no reverse is assigned in 'Loonz'
Mouse emulation will not pair with system	'Loonz' connected	Disconnect from Loonz'	• <i>These two features cannot be paired simultaneously</i>
Mouse emulation will not connect once paired	Multi Tap currently off	Enable Multi Tap	• <i>Multi tap enables the ability to put an i-Drive into</i>
	Triple click time too fast	Extend the Multi Tap time	• <i>It might be necessary to use the Calibrate Multi</i>
	issues with .net framework	Download the correct .net framework	• <i>Check that the framework level matches the requirements in the manual</i>
	Not reconnected after update	Reconnect to the Windows app	• <i>The app will store the settings and push them onto the next interface recognized with a serial number of 0</i>

Windows Configurator Software

i-Drive's Windows Configurator Software is not required to operate the drive control, but is available to ensure a customized fit and driving experience. Programming the i-Drive is simple: By connecting a computer, tablet, or smartphone, and using the Windows Configurator Software application, you can:

- control any of the outputs via any port;
- change the range of motion required for switch activation;
- provide an adjustable double-tap feature that extends chair double-tap limits; and/or
- use Bluetooth or any PC to configure and program.

Getting Started


The first step to begin use of the i-Drive 4.0 is downloading the software. The link to download the software is listed on page 33 (CHECK PAGE). If you experience any difficulties obtaining the software, contact Stealth Products, LLC. After the software has been installed onto your device, you can begin utilizing the Windows Configurator Software. Launch the application on your device, and with the provided cable, connect the i-Drive to your PC or tablet. Allow time for the i-Drive to connect fully to your computer.

Downloading the i-Drive Software

To program the i-Drive from your PC, the Windows Configurator Software will need to be downloaded from either the Microsoft Store, Apple's App Store, or Google Play. The USB interface enables an external device to communicate with the i-Drive via a virtual communications port. Authentication is required to gain access.

The Bluetooth interface enables an external device to communicate with the i-Drive via a secure Bluetooth connection.

The serial number from the interface must be entered in order to connect the i-Drive to the software. The figures below indicate a) the location of the serial number on the interface, and b) the location for the USB port.

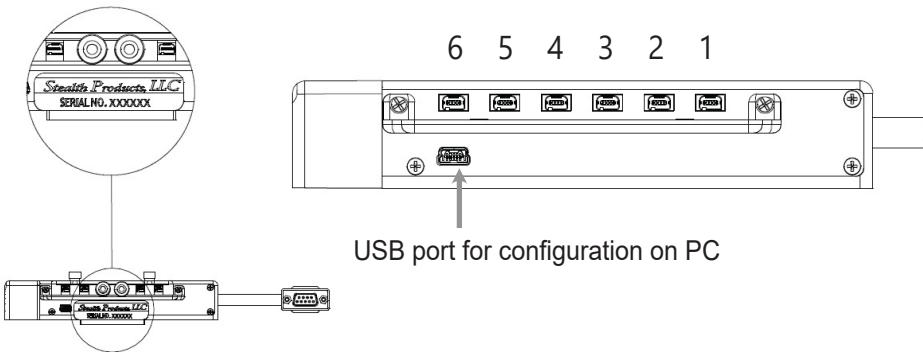

NOTICE

Contact Stealth Products and speak to a customer service representative to obtain your specific login information.


Software Capabilities

- The mo-Vis Configurator Software is compatible with Windows 7, Windows 8, and Windows 10 systems.
- The mo-Vis Configurator Software is NOT compatible with Android or iOS systems.
- The software is an optional feature that will allow minimal adjustments. All adjustments of the joystick including deadband, calibration, throw, etc. can be completed through the chair's display.


The All-Round Joystick positioning is adjustable with the mo-Vis Configurator Software.

Software Download


1. You can download the software on Stealth's website: www.stealthproducts.com.
2. Connect the mo-Vis Joystick to the PC using a mini USB cable.
3. Open the mo-Vis Configurator Software. Your computer will choose a COM port and recognize the product that is linked to the PC, but will still show the status "Disconnected".

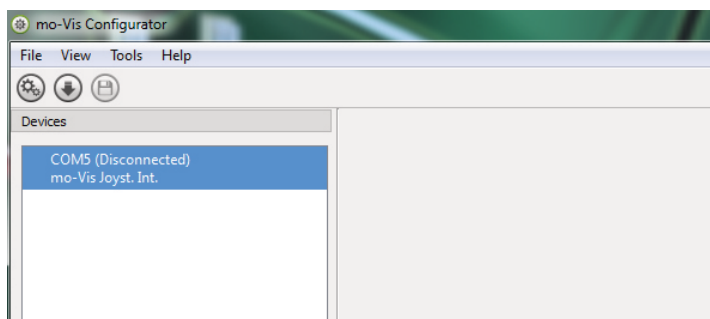


4. Click on the Connect icon  to make the connection between the Configurator Software and the All-Round Heavy Duty Joystick, or choose in the menu bar, File - Connect. To disconnect the device, click on the same icon to disconnect, or click on File and then click Disconnect.

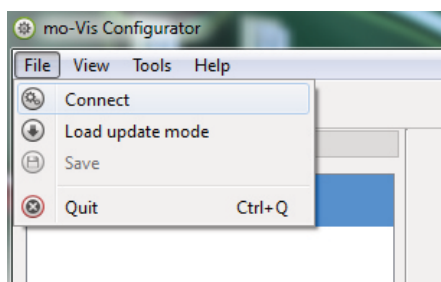


5. When the device is in sleep mode or not powered on, you will get this message:

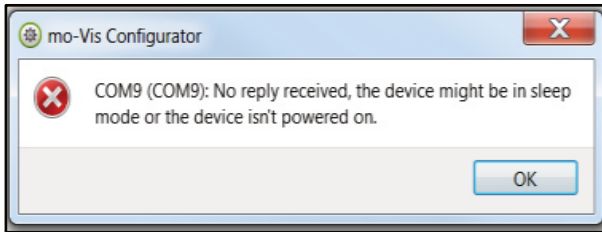

6. Once the connection is made, the software will receive all the details and information out of the mo-Vis device and will display them on the right hand side of the window.

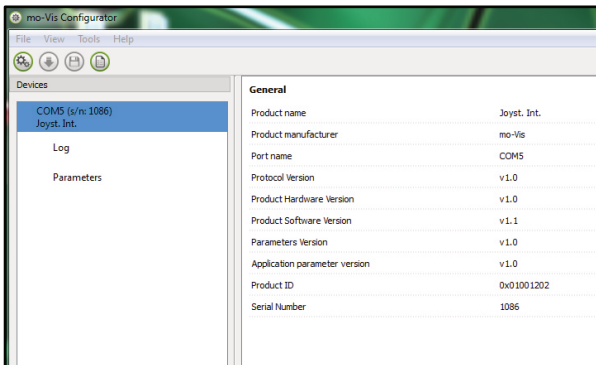


7. The software program starts at the lowest Access Level. In order to receive more information or see and change more parameters, you can upgrade your access level from User to Attendant or Dealer.

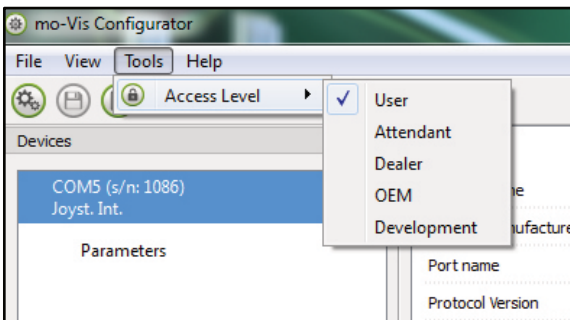


Software Configuration

- Click on the Tools tab in the menu bar and change the Access Level from User to Attendant or Dealer.
- You will be asked to enter your password. This password will need to be requested.

Attention: These Passwords are Case Sensitive!!

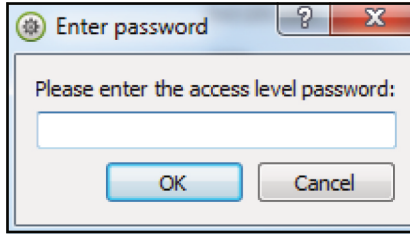


8. After the password has been correctly entered, you will be asked to restart the application. **Any unsaved changes will be lost. If you did make changes and want to save them, choose NO and first save your changes.** If it is okay to restart the application, click Yes. The device will now be disconnected.
 - Repeat Step 4 to connect the device back to the computer.
 - Depending on the Access Level, you will see *General Information*, *Parameters*, *Log*, and *Diagnostics*.
9. When *Parameters* is selected, a list of parameters will be shown, depending on the device that is connected to the PC.
 - By selecting the specific parameter you wish to adjust, the view on the right will display the stored *Default*, *Minimum*, and *Maximum* Values of the specific parameter.
 - This parameter can now be changed by entering a new value in the text box, by clicking the arrows next to the text box, or by selecting a new value from the drop down box.
 - Values that are being changed will be shown in green.

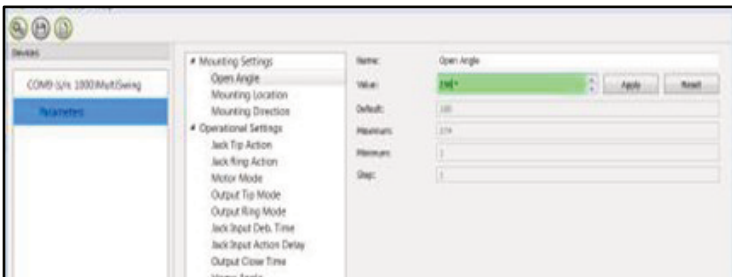


- Click *Apply* to write the temporary value in the device. You can immediately test it, but the new value is not stored yet.
- If you lose connection, any changes you made will be lost. If you would like to store the parameter values into the device, click on the *Save* icon or click in the menu bar, *File - Save*. The new parameter values will now be stored permanently in the device.


Attention: When a parameter is changed without clicking the *Apply* button, the new values are not stored. Once another parameter setting is selected, the previous parameter will revert to the original setting.

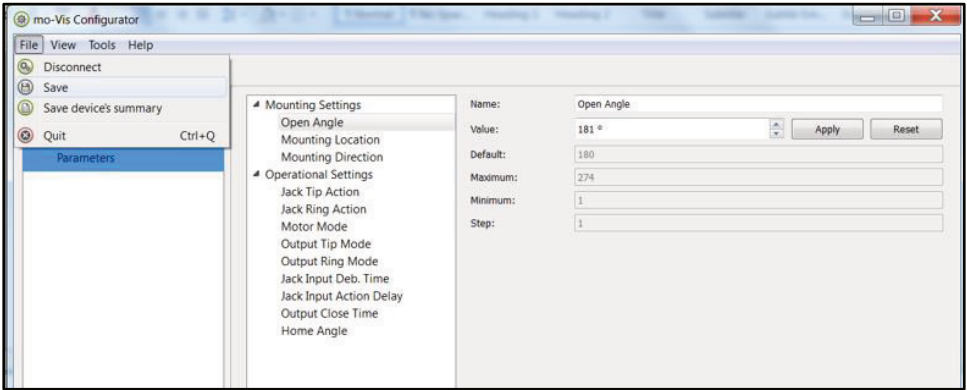


Clicking on the *Reset* button will save the parameter value back to the default state. Click *Apply* to store locally and *Save* to make the change to the device.

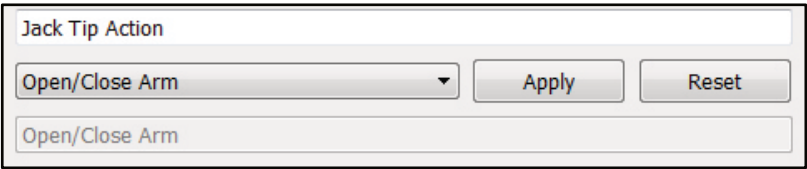


- When your Access Level is set to *Dealer Level* the categories *Log* and *Diagnostics* will be displayed as well
 - When *Log* is selected, and overview of all errors that have occurred and/or actions will be displayed with a number of times that they occurred, and a *Reset* button to set the counter back to zero, if desired.

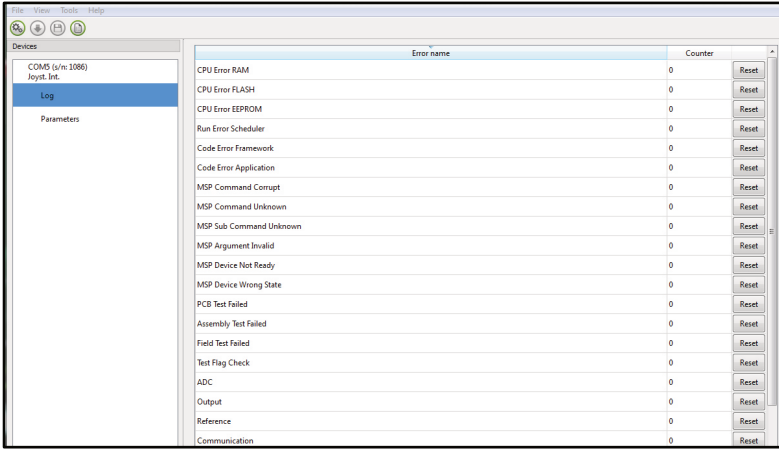

- When *Diagnostics* is selected, you will be able to run a *Field Diagnostic Test*. Click on *Field Diagnostic Test* and then choose *Run Test* to begin the Field Test.

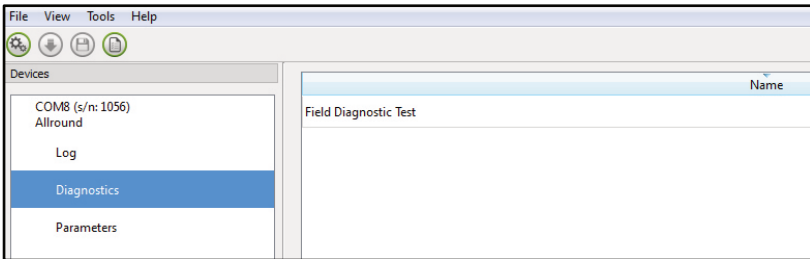


- To change the language of the Configurator Software, click File-View on the menu bar and select the language you prefer.

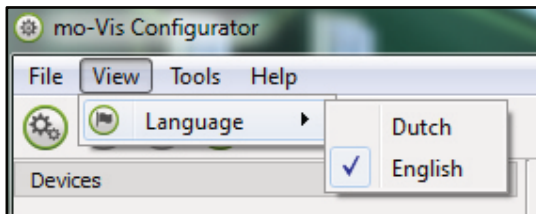

12. Save the device summary. You can save all general product information and values of the different parameters in any folder on your computer.
- Make sure the device is connected to the USB port of the PC.
 - If the device is recognized, it will appear in the *Device View*.
 - Select the correct device in *Device View*.
 - Click on the menu bar on *File - Save Device's Summary* or click directly on the *Save Device Summary icon* . Choose a location to save the file.
 - To close the Configurator Software, choose *File - Quit (Ctrl+Q)* in the menu bar, or click on the X in the upper right corner of the window.  .

Attention: You will lose all unsaved values if you quit the software without first clicking on *Apply and Save*. If you use the menu bar to quit, your values will be saved automatically once you click *Apply*.

If you use the X on the upper right corner of the window, the software will automatically ask if you would like to save your changes before losing the software.

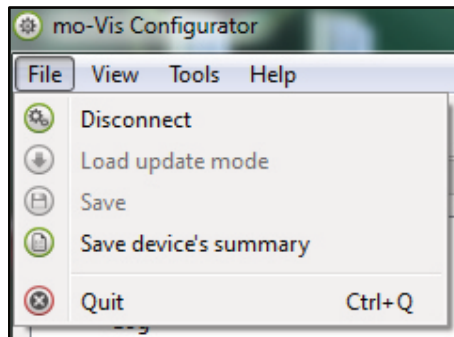

Diagnostics


The 'Diagnostics' feature shows the user the i-Drive's input activations, enabling real-time diagnoses of potential input issues. It also aids in finding and improving the location of switches to maximize the efficacy and consistency of switch activation.

The activation of an i-Drive's connected input through the 'Diagnostics' feature is shown below.



Configuration


The 'Configuration' page of the software provides users options for altering the settings of their channel assignments, calibrations, and various other items.

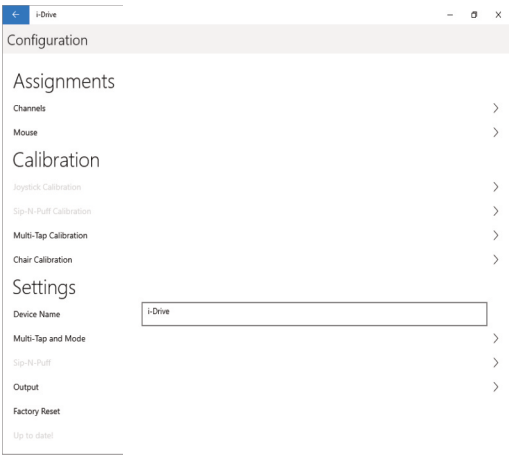

Assignments: Change the input channel assignments for any devices connected to the i-Drive.


Calibration: Calibrate the chair and the i-Drive to the same neutral state.

Channel Assignment

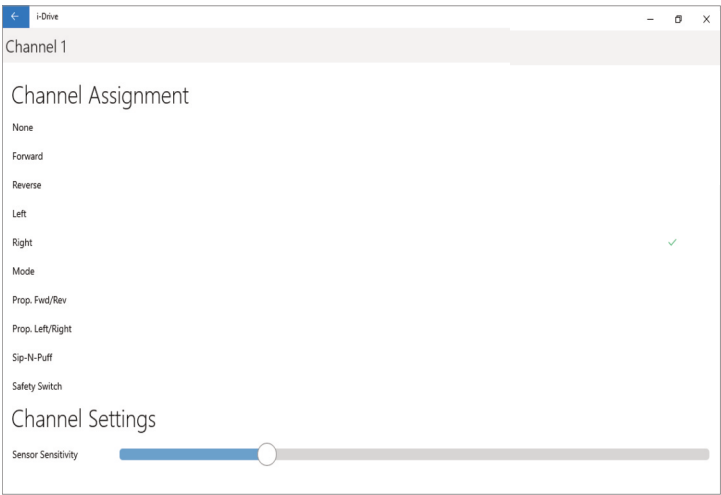

After having selected a particular channel, a screen will pop up listing the available selections for that channel, along with an adjustment bar for the input sensor sensitivity.


Chair Calibration

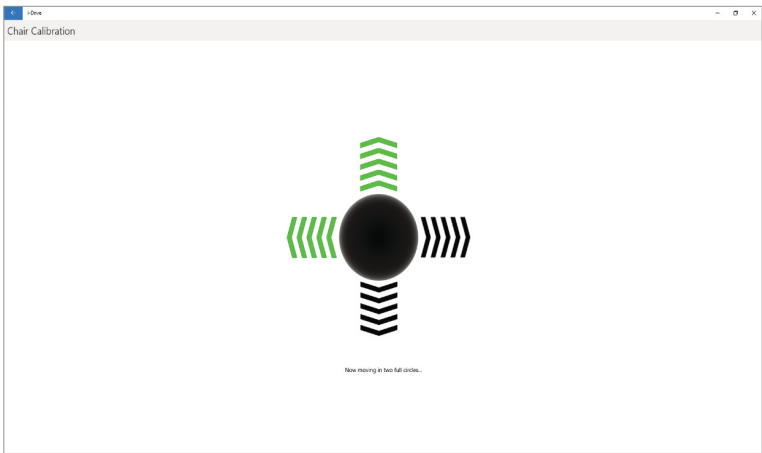

Selecting 'Chair Calibration' allows a user to calibrate the chair with the i-Drive, leveling both to the same neutral position. To get the best performance from the i-Drive interface, it is important to calibrate the interface to the wheelchair electronics. It is important to note that calibration is required, even if the interface is using switched inputs.

To properly calibrate the chair, follow the on-screen prompts after having selected 'Chair Calibration'. Note that only users with appropriate access rights may perform a chair calibration.

Output Mode

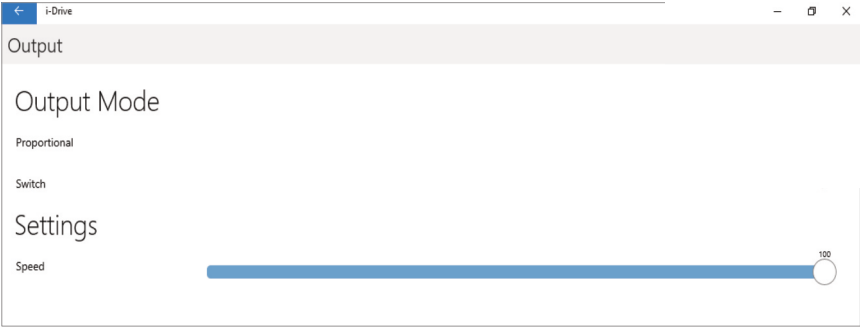

Selecting 'Output' allows a user to change the i-Drive's output mode from proportional to switched, or vice-versa. It also allows a user to adjust the output speed.

The latest version of the i-Drive firmware allows for either proportional or switched output, which improves functionality and compatibility with multiple wheelchair electronics systems.


The output mode capabilities for each electronics system are as follows:

The i-Drive plugs into the 9-pin port on the power chair's control interface. The location of the port depends on the manufacturer of the interface. Follow the instructions provided by your interface provider.

SYSTEM	PROPORTIONAL	SWITCHED
Q-Logic2	Yes*	Yes
Q-Logic 3	Yes*	Yes
R-Net	Yes	Yes*
MK6i	No	Yes
DX	No	Yes


* Optimal Output Mode for this system

Factory Reset

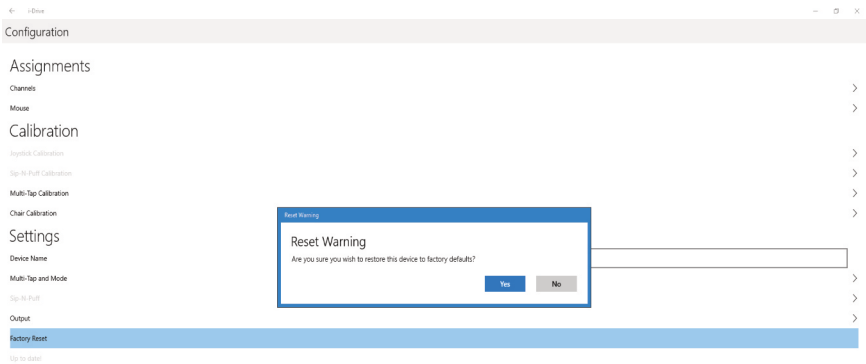

Selecting 'Factory Reset' allows a user to restore all i-Drive settings to their original out-of-the-box preset assignments.

Firmware Update

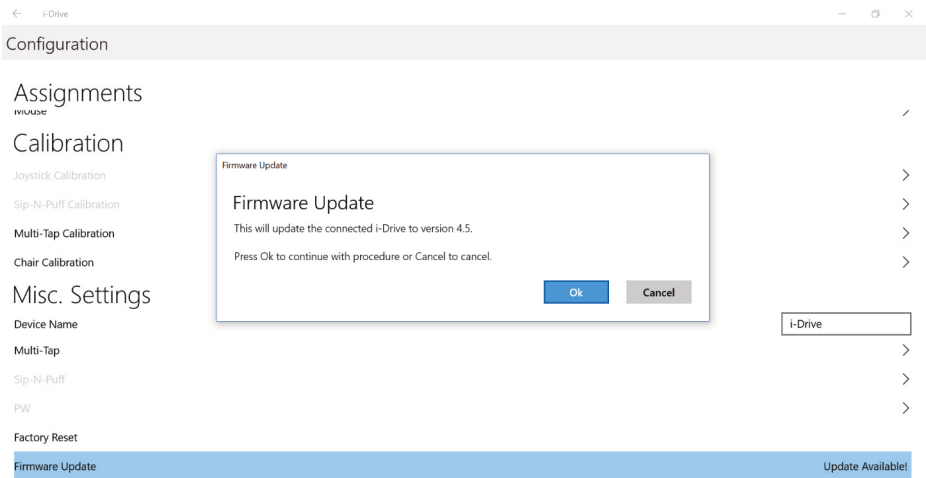

Updates for the i-Drive's firmware are sometimes made available. In order to update the firmware for your device, select 'Firmware Update' in the 'Configuration' menu (as pictured above) and follow the on-screen prompts.

Testing

After the installation of the i-Drive, execute the following tests before the wheelchair is delivered or put into service:

- Operational Test
- Test Drive
- Stop Test



WARNING

Execute this test only on a level surface,
in an open area free of obstacles.

Operational Test

1. Activate the wheelchair operating system.
2. Check for any error messages. Do not proceed until these errors are resolved.
3. Apply pressure to the head pad/sensor/joystick, etc. designated for moving the chair in a forward motion until you hear the parking brakes switch off. The wheelchair may start moving.
4. Immediately release the pressure from the head pad/sensor/joystick, etc. You should hear the parking brake react within a few seconds.
5. Repeat Steps 3 and 4 three times, while carefully applying pressure to the device.
6. Check whether the double-tap features function properly.



CAUTION

The wheelchair may start moving.

Test Drive

- Perform a test drive with the wheelchair.
- Check whether the wheelchair and all of its operations are fully functioning.
- Check that no cabling or parts may get damaged or hindered in any possible position of the wheelchair.

Dealer Assistance

During first time use by the client, it is advised that the dealer or service technician assists and explains the different drive configurations to the customer (the user and/or the attendant). If needed, the dealer can make final adjustments.

User Testing

It is important that the customer is fully aware of the installation of the i-Drive, how to use it, and what settings in the software or App can be adjusted in order to gain as much mobility as possible. As a dealer, proceed as follows:

- Explain and show the customer how you have executed the installation, and explain the functions that have been assigned to the device.

Have the user test all configurations of the device:

- Is the device being used by the client within easy reach and in an optimal position for the user?
- Can the user safely operate the wheelchair with minimal effort.

Conditions of Use

The i-Drive 4.0 with Bluetooth wireless technology is intended for use as installed by the dealer, in accordance to the installation instructions in this manual.

- The foreseen conditions of use are communicated by the dealer or service technician to the user and/or attendant during the first time use.
- If the conditions of use change significantly, please contact your dealer or a qualified service technician to avoid excessive wear and tear or unintended damage.

Care and Maintenance

Ensure the hardware you are using stays in working order by keeping it cared for a maintained.

- Keep electronics dry and out of the water.
- Periodically check the hardware for loose screws or worn parts. Replace or repair the parts as needed.
- To clean aluminum, use a mild, non-abrasive household cleaner.
- On the hardware, lightly tighten set screws until they are snug, then tighten them an extra quarter turn.
- Ensure strain relief is tight (lightly pull on power cord and ensure strain relief is tight.)



WARNING

Over-tightening set screws will prevent hardware from functioning properly and could cause irreversible damage to the hardware.

Safety



WARNING

Avoid getting electronics wet and working on electronics with wet or damp hands. This could cause minor to severe electrocution, resulting in personal injury and/or product damage.



CAUTION

Periodically check Diagnostics for real-time data information to verify that all sensors are functioning properly.

Technical Data

The i-Drive Bluetooth Interface was tested and found to be in compliance with the required criteria and met the standards of IEC60601-1-2:2014 (Edition 4.0); RESNA WC-2:2009.

[illegible]

Notes

[illegible]

[illegible]



© 2023, Stealth Products, LLC

Warranty Information



Home Page



Online User Manual

