


Stealth Products
®



**Alternative
Controls**

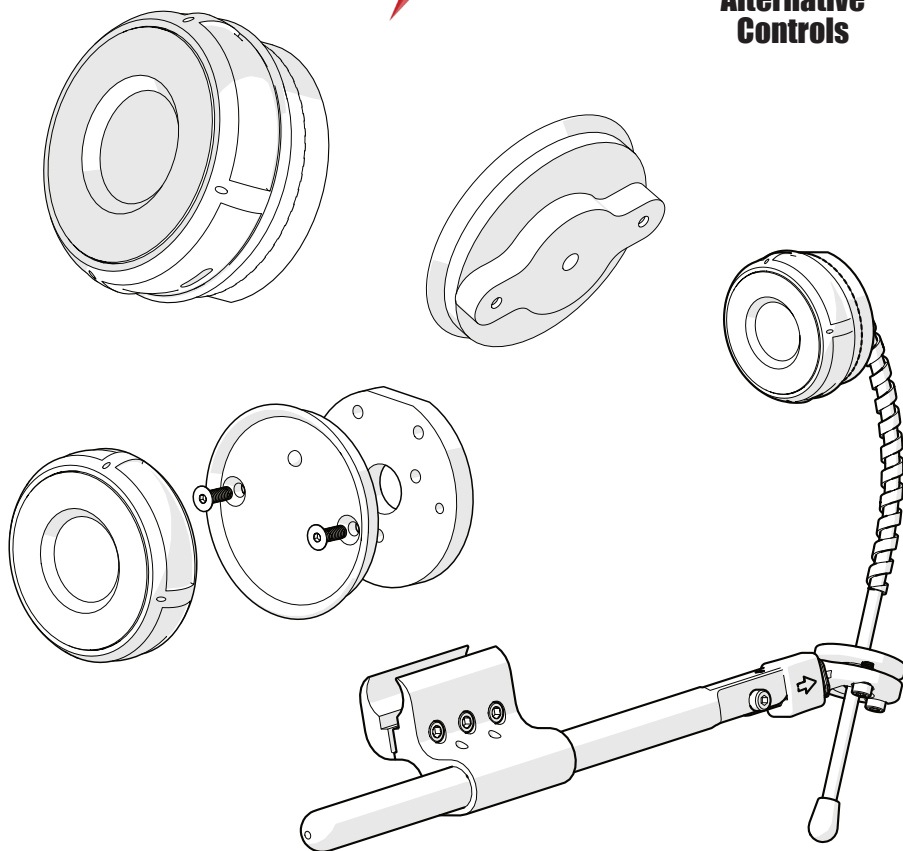

Linxura Interface Plates

Installation Guide

for

LNK-MTGPLT, LNK-ARTMNT,
LNK-ARMS260, LNKNC-ARMS260

Customer Satisfaction i

Important Information ii

Design and Function 1-2

 Parts Included LNX-MTGPLT 1

 Parts Included LNX-ARTPLT 1

 Parts Included LNX(NC)-ARMS260 2

 Torque Specification 2

Installation Instructions 3-4

 LNX-MTGPLT Mounting Plate Installation 3

 LNX-ARTPLT Articulating Arm Mount Installation 3

 LNX(NC)-ARMS260 Mount Installation 4

First Time Use 5

 Dealer Assistance 5

 User Testing 5

 Conditions of Use 5

Cleaning and Maintenance 6

 Cleaning 6

 Maintenance 6

 Notes 6

Stealth Products is committed to 100% customer satisfaction. Your complete satisfaction is important to us. Please contact us with feedback or suggestions to help us improve the quality and usability of our products.

You may reach us at:

Stealth Products, LLC

104 John Kelly Drive, Burnet, TX 78611



Phone: (512) 715-9995

Fax: (512) 715-9954

Toll Free: (800) 965-9229

Toll Free: (800) 806-1225

info@stealthproducts.com

www.stealthproducts.com

EU REP  EMERGO EUROPE BV Westervoortsedijk 60 6827 AT Arnhem The Netherlands	CH REP GELBART AG Tribtschenstrasse 64 CH-6005 Luzern Switzerland	UK Authorized Representative  EMERGO CONSULTING (UK) LTD. c/o Cr360 - UL International Compass House, Vision Park Histon Cambridge CB24 9BZ United Kingdom
--	---	---

General

Read and understand all instructions prior to the use of the product. Failure to adhere to instructions and warnings in this document may result in property damage, injury, or death. Product misuse or failure to follow instructions will void the warranty.

Immediately discontinue use if any function is compromised, if parts are missing or loose, or if any component shows signs of excessive wear. Consult with your supplier for repair, adjustment, or replacement.

All persons responsible for fitting, adjustment, and daily use of the devices discussed in these instructions must be familiar with and understand all safety aspects of the devices mentioned. In order for our products to be used successfully, you must read and understand all instructions and warnings, and maintain our products according to our instructions on care and maintenance.

The installation instructions will guide you through this product's options and possibilities.

Instructions are written with the expressed intent of use with standard configurations. They also contain important safety and maintenance information, as well as describe possible problems that can arise during use. For further assistance, or more advanced applications, please contact your supplier or **Stealth Products** at (512) 715-9995 or toll free at (800) 965-9229.

Always keep the operating instructions in a safe place so they may be referenced as necessary.

All information, pictures, illustrations, and specifications are based on the product information that was available at the time of printing. Pictures and illustrations shown in these instructions are representative examples and are not intended to be exact depictions of the various parts of the product.

Important Information



CAUTION

These products are designed to be fitted, applied, and installed exclusively by a healthcare professional trained for these purposes. The fitting, application, and installation by a non-qualified individual could result in serious injury.

Warranty

Our products are designed, manufactured, and produced to the highest of standards. If any defect in material or workmanship is found, **Stealth Products** will repair or replace the product at our discretion. Any implied warranty, including the implied warranties of merchantability and fitness for a particular purpose, shall not extend beyond the duration of this warranty. **Stealth Products** does not warrant damage due to, but not limited to: misuse, abuse, or misapplication of product, and/or modification of product without written approval from **Stealth Products, LLC**. Any alteration or lack of serial number, where applicable, will automatically void all warranty.

Stealth Products, LLC is liable for replacement parts only. **Stealth Products, LLC** is not liable for any incurred labor costs. **Stealth Products** warrants against failure due to defective materials or workmanship:

Covers, Headrest Pads, Cushions: 2 years

Hardware: 5 years

Electronics: 3 years

In the event of a product failure covered by our warranty, please follow the procedures outlined below:

- Call **Stealth Products** at (512) 715-9995 or toll free at (800) 965-9229.
- Request a Return Authorization (RA) form from the Returns Department and follow the documentation instructions.

Full warranty information may be found by visiting the website at <https://stlpro.site/returns-n-warranty>, or by scanning the Warranty Information QR code located on the back cover of this manual. Copies of this manual may be downloaded by accessing the Stealth website (<https://stlpro.site/stealth-docs>) and searching "Linxura Controller" in the search bar at the top of the page.

Supplier Reference

Supplier:	
Telephone:	
Address:	
Purchase Date:	
Model:	

Warning Labels

Warnings are included for the safety of the user, client, operator, and property. Please read and understand what the signal words **DANGER**, **WARNING**, **CAUTION**, **NOTICE**, and **SAFETY** mean, and how they could affect the user, those around the user, and property.



DANGER

Identifies an **imminent** situation which, if not avoided, may result in **severe injury, death, and property damage**.



WARNING

Identifies a **potential** situation which, if not avoided, may result in **severe injury, death, and property damage**.



CAUTION

Identifies a **potential** situation which, if not avoided, may result in **minor to moderate injury and property damage**.

NOTICE

Identifies important information not related to injury, but possible **property damage**.

SAFETY

Indicates steps or instructions for safe practices, reminders of safe procedures, or important safety equipment that may be necessary.

Limited Liability

Stealth Products, LLC accepts no liability for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendations, warnings, and instructions in this manual.


Stealth Products does not hold responsibility for final integration of final assembly of product to end user. **Stealth Products** is not liable for user death or injury.

Testing

Initial setup and driving should be done in an open area free of obstacles until the user is fully capable of driving safely.


Parts Included

LNx-MTGPLT

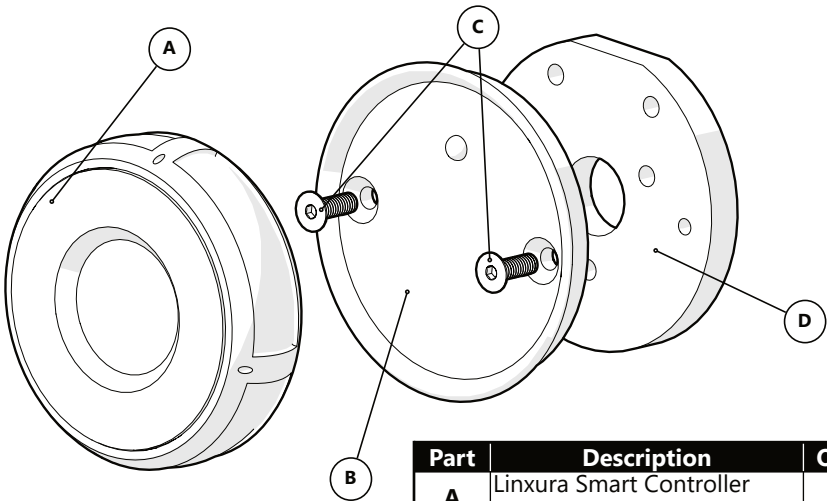


Part	Description	Qty
A	Linxura Smart Controller (INCLUDED WITH LINXURA BUNDLE)	1
B	Linxura Wall Mount (INCLUDED WITH LINXURA BUNDLE)	1
C	M4 x 0.7 x 12MM FHSN	2
D	Interface Plate	1

LNx-ARTPLT

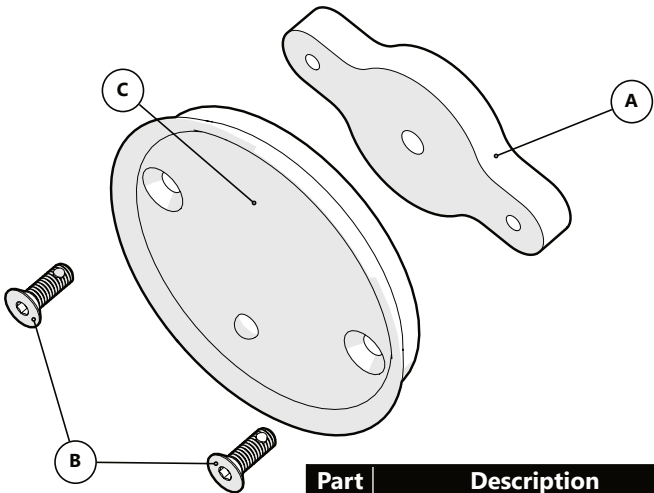

Part	Description	Qty
A	1/4" x 20 Articulating Arm Mount	1
B	M4 x 0.7 x 12MM FHSN	2
C	Linxura Wall Mount (INCLUDED WITH LINXURA BUNDLE)	1

**LNX-ARMS260,
LNXNC-ARMS260**

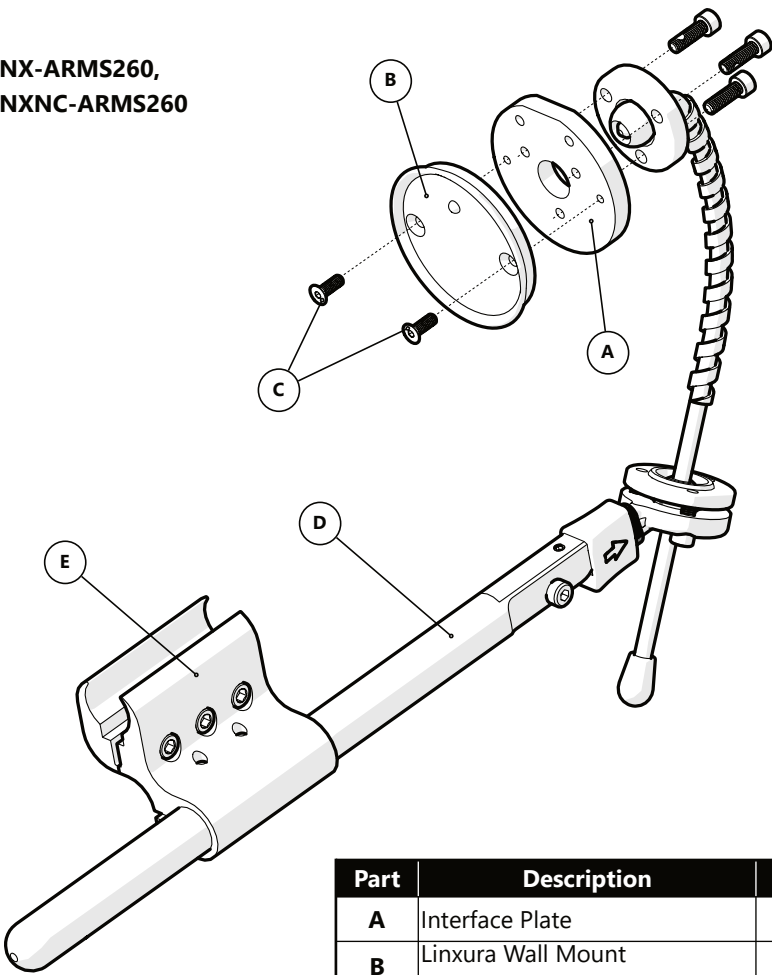

Part	Description	Qty
A	Interface Plate	1
B	Linxura Wall Mount (INCLUDED WITH LINXURA BUNDLE)	1
C	M4 x 0.7 x 12MM FHSN	2
D	ARMS260 Swing Away Mount	1
E	Cane Mount Clamp*	


* Clamp not included with LNXNC-ARMS260

Torque Specifications	
Fastener	Value (N·m)
M4 x 0.7 x 12MM FHSN	3.2

LNx-MTGPLT Mounting Plate Installation

1. Align the Interface Plate (2) with the holes in the Linxura Wall Plate (1) as shown in the image to the right.


2. Use the two included M4 x 0.7 x 12MM FHSN (C) to fasten the Wall Plate to the Interface Plate.

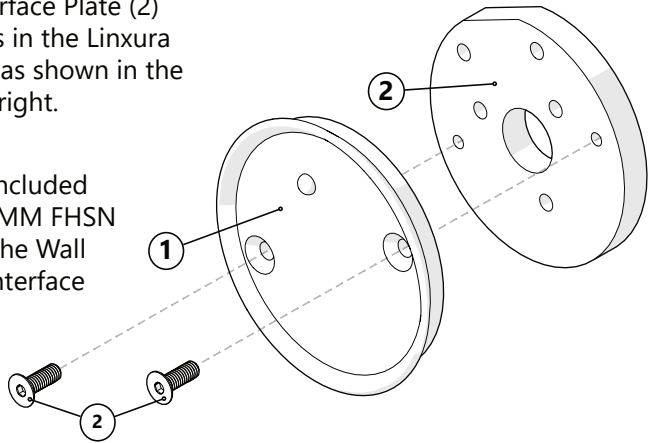

3. Use the threaded holes on the mounting plate to mount the device on the hardware of your choice.

LNx-ARTPLT Articulating Arm Mount Installation

1. Align the 1/4"-20 threaded mounting plate (1) with the holes in the Linxura Wall Plate (2) as shown in the image below.

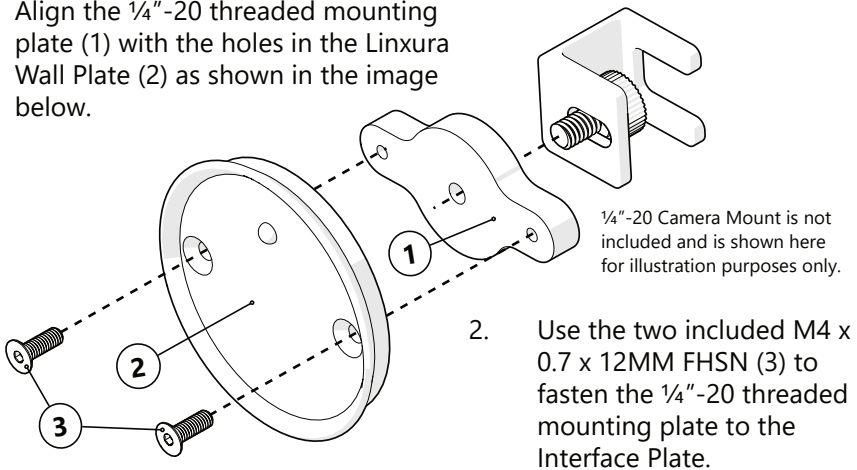


2. Use the two included M4 x 0.7 x 12MM FHSN (3) to fasten the 1/4"-20 threaded mounting plate to the Interface Plate.
3. Use the 1/4"-20 threaded hole on the mounting plate to mount the device to any articulating arm with a 1/4"-20 thumbscrew.

NOTICE

The mounting plate will not accommodate 3/8"-16 camera mounts. A readily-available adapter will be required to convert the 3/8"-16 thread to 1/4"-20.

LNx(NC)-ARMS260 Mount Installation

1. Mount the ARMS-260 Swing Away Arm (4) to the chair using the Cane Clamp (5), or other mounting method.

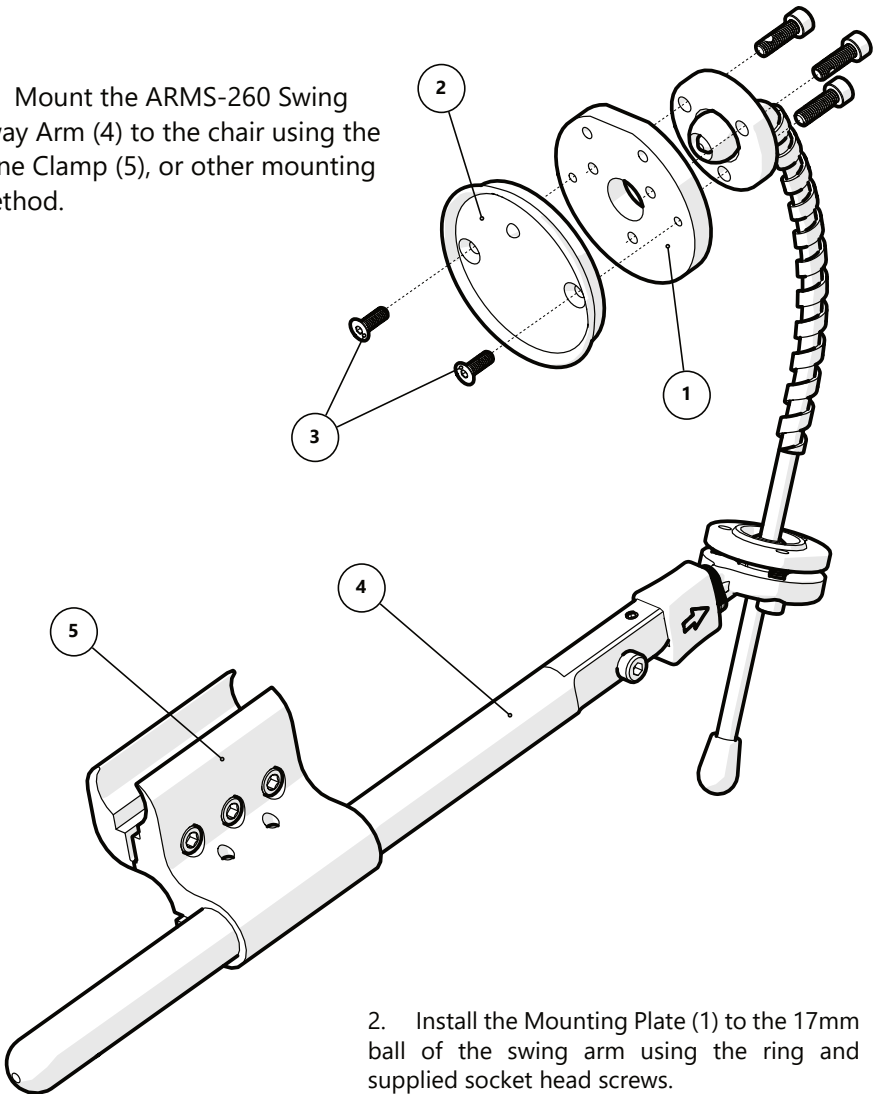

2. Install the Mounting Plate (1) to the 17mm ball of the swing arm using the ring and supplied socket head screws.

3. Install the Linxura Wall Plate (2) to the Mounting Plate (1) using the M4 x 0.7 x 12MM Flat Head Screws (3).

4. Adjust the ARMS-260 Swing Away Arm to the desired position as instructed in the ARMS-260 owner's manual.

Dealer Assistance

During first time use by the client, it is advised that the dealer or service technician assists and explains the different configurations to the customer (the user and/or the attendant). If needed the dealer can make final adjustments.

User Testing

It is important that the customer is fully aware of the installation, how to use it, and what can be adjusted in order to gain as much mobility as possible. As a dealer, proceed as follows:

- Explain and show the customer how you have executed the installation, and explain the function of every new item.

Have the user test all positions and functions of the Linxura Controller:

- Is the Linxura Controller well positioned in accordance with the client's needs?
- Does the Linxura Controller have proper clearance for and accessibility to the user?
- Can the user safely operate the Linxura Controller with the least effort?

If needed, adjust the mounting hardware and retest until it is in the optimal position and functions properly.

Explain to the customer any possible problems and how to address them.

Conditions of Use

The Linxura Controller and associated mounting hardware is intended for use as installed by the dealer, in accordance with instructions in this manual and the manual included with the Linxura Controller.

- The foreseen conditions of use are communicated by the dealer or service technician to the user and/or attendant during the first time use.
- If the conditions of use change significantly, please contact your dealer or qualified service technician to avoid excessive wear and tear or unintended damage.

Cleaning

Clean all parts of the Linxura Controller and associated mounting hardware on a regular basis.

- Gently remove dust and dirt with a dry cloth.
- If more thorough cleaning becomes necessary, use only a non-aggressive disinfectant cleaning agent.

Maintenance

Periodically check the fasteners on clamps and other hardware to ensure that they have not become loose. Loose parts can lead to user discomfort or injury, and wear or premature part failure.

Notes



© 2025, Stealth Products, LLC

Warranty Information



Home Page



Online User Manual



Stealth Products, LLC · info@stealthproducts.com · www.stealthproducts.com
(800) 965-9229 | (512) 715-9995 | 104 John Kelly Drive, Burnet, TX 78611