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Stealth Products

2-Point PRO Montagezubehör

Für

Aluminium- und Carbon-Rückenserie

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Installation Manual

2 Point PRO Mounting Hardware

For Aluminum & Carbon Series Back Supports

Stealth Products strives for 100% customer satisfaction. Your complete satisfaction is important. Please contact us with feedback or suggested changes that will help improve the quality and usability of these products. You may reach us at:

Stealth Products, LLC

104 John Kelly Drive, Burnet, TX 78611

Phone: (512) 715-9995

Fax: (512) 715-9954

Toll Free: (800) 965-9229

Toll Free: (800) 806-1225

info@stealthproducts.com

www.stealthproducts.com

EU REP CE PRIDE MOBILITY PRODUCTS EUROPE BV De Zwaan 3 1601 MS Enkhuizen The Netherlands	CH REP SODIMED SA Chemin Praz Devant 12 1032 Romanel sur-Lausanne Switzerland	UK Authorized Representative UK CA PRIDE MOBILITY PRODUCTS LTD. 32 Wedgwood Road, Bicester, Oxfordshire OX26 4UL United Kingdom
---	---	---

General

Read and understand all instructions prior to the use of the product. Failure to adhere to instructions and warnings in this document may result in property damage, injury, or death. Product misuse due to failure of the following instructions will void the warranty.

Immediately discontinue use if any function is compromised, parts are missing, loose, or shows signs of excessive wear. Consult with you supplier for repair, adjustment, or replacement.

If this document contains information you do not understand, or there are concerns about safety or operation, contact your supplier.

Important Information!

All persons responsible for fitting, adjustment, and daily use of the devices discussed in these instructions must be familiar with and understand all safety aspects of the devices mentioned. In order for our products to be used successfully, you must:

- Read and understand all instructions and warnings.
- Maintain our products according to our instructions on care and maintenance.
- Devices should be installed and adjusted by a trained technician.

Ordering Documentation

You can download additional copies of this manual on the Stealth website:
www.stealthproducts.com/gui/docs/?type=usr&

Installation Date: _____

Dealer: _____

Dealer Stamp:

Serial Number:

No person is authorized to alter, extend, or waive the warranties of Stealth Products, LLC.

Stealth Products warrants against failure due to defective materials or workmanship:

Covers: 180 days

Hardware: 5 years

Electronics: 3 years

Limitations

4.1

Stealth Products, LLC does not warrant damage due to, but not limited to:

- Misuse, abuse, or misapplication of products.
- Modification of product without written approval from Stealth Products, LLC.
- Any alteration or lack of serial number, where applicable, will automatically void this warranty.
- Stealth Products, LLC is liable for replacement parts only.
- Stealth Products, LLC is not liable for any incurred labor costs.

NOTICE

Any substitutions of supplied hardware void the warranty. Misuse of items and "acts of nature" (ex. Flood, tornado, etc.) are not covered by the warranty.

1.0 Customer Satisfaction	i
2.0 Important Information	ii
3.0 Introduction.....	iii
4.0 Warranty	iv
4.1 Limitations.....	iv
5.0 Table of Contents	v
6.0 Warning Labels	vi
6.1 Warning Labels.....	vi
6.2 Limited Liability	vi
6.3 Testing	vi
7.0 Parts and Accessories	vii
7.1 Tools Required for Installation & Adjustment.....	vii
7.2 Parts Required	vii
7.3 2-Point PRO Mounting Hardware Assembly	vii
7.4 2-Point PRO Torque Specification	viii
8.0 Installation Instructions	1
8.1 Hardware Installation	1
8.2 Angle Adjustment	3
8.3 Depth Adjustment.....	3
8.4 Width Adjustment.....	3
9.0 First Time Use	4
9.1 Dealer Assistance	4
9.2 User Testing.....	4
9.3 Conditions of Use	4
10.0 Maintenance	5
10.1 Cleaning.....	5
10.2 Disinfection	5

Warning Labels

6.1

Warnings are included for the safety of the user, client, operator and property. Please read and understand what the signal words **SAFETY**, **NOTICE**, **CAUTION**, **WARNING** and **DANGER** mean, how they could affect the user, those around the user, and property.

 DANGER	Identifies an imminent situation which (if not avoided) will result in severe injury, death, and property damage .
 WARNING	Identifies a potential situation which (if not avoided) will result in severe injury, death, and property damage .
 CAUTION	Identifies a potential situation which (if not avoided) will result in minor to moderate injury, and property damage .
NOTICE	Identifies important information not related to injury, but possible property damage .
SAFETY	Indicates steps or instructions for safe practices, reminders of safe procedures, or important safety equipment that may be necessary.

Limited Liability

6.2

Stealth Products, LLC accepts no liability for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendations, warnings, and instructions in this manual.

Testing

6.3

Initial setup and driving should be done in an open area free of obstacles until the user is fully capable of driving safely.

The product should always be tested without any person sitting in the wheelchair until every alteration of the physical installation or adjustment is complete.

Tools Required for Installation & Adjustment


7.1

- 4mm Allen Wrench
- 10mm wrench

Parts Required


7.2

- Back Support Shell, 1 each (A)
- U-Block Assembly, 2 each (BR1104 attached to back) (B)
- Left Side 2 Point PRO Clamp Assembly, 1 each (BR1103L) (C)
- Right Side 2 Point PRO Clamp Assembly, 1 each (BR1103R) (D)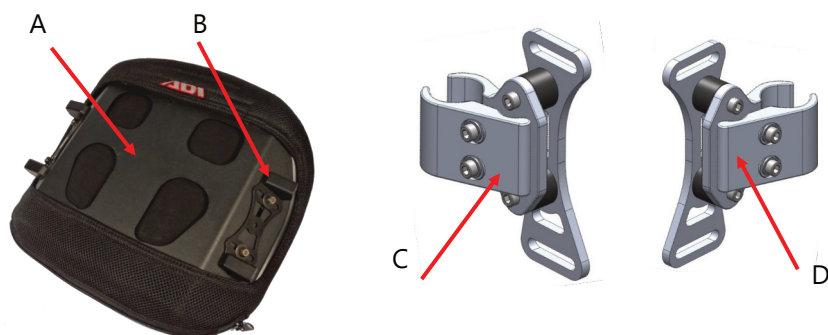

2 Point PRO Mounting Hardware Assembly

7.3

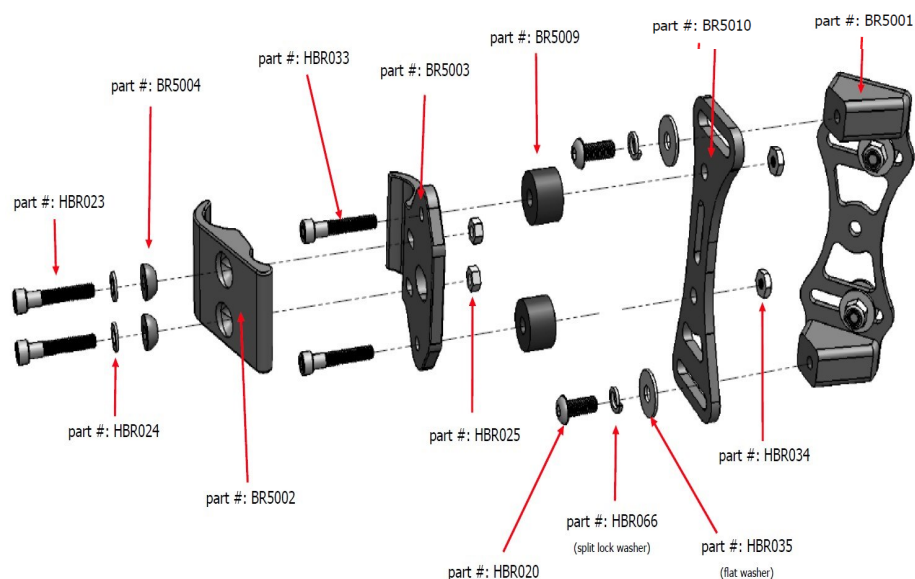2 Point PRO Torque Specification

7.4

Part Number: HBR020

M6 x 20mm Button Head SS

Use: 4mm Drive Allen Wrench

Torque Spec: 77.0 in-lbs/ 8.7N-m

Part Number: HBR033

M6 x 30mm Low Socket Cap Screw
18-8S/S

Use: 4mm Drive Allen Wrench

Torque Spec: 31.7 in-lbs/3.6N-m

Part Number: HBR022 (for 3/4" or 7/8" tubing)

M5 -.8 x 22mm, Socket C/S

Use: 4mm Drive Allen Wrench

Torque Specs: 45.1 in-lbs/5.1 N-m

AL & CF Series Backs

Part Number: BR5001

Stud Plate

Use: 10mm hex socket or wrench

Torque Spec: 117 in-lbs/ 13.2 N-m

Part Number: HBR023 (for 7/8" & 1" tubing)

M5 -.8 x 25mm, Socket C/S

Use: 4mm Drive Allen Wrench

Torque Specs: 45.1 in-lbs/5.1N-m

NOTICE

Only use the hardware supplied with the ADI back on the ADI back. Do not substitute any other screws, brackets, or clamps. Use of alternative hardware or mounting brackets void all warranties.

Hardware Installation

8.1

1. Remove the existing wheelchair back according to the original manufacturer's instructions.
2. Visually determine the desired height of the back support.
3. Mark on the wheelchair back posts the desired location of mounting hardware.
4. Attach mounting hardware to the left and right side of the wheelchair back post tubing.

Step 2

Step 3

Step 4

5. Attach the 2 Point PRO mounting hardware to the wheelchair back posts. Mounting hardware should be equal heights on the left and right sides, and parallel to the seat frame.
- Loosen the clamp screws with a 4mm hex wrench.
 - Attach and align mounting hardware to the wheelchair back post.
 - Tighten clamp screws, but allow for movement so you can adjust when attaching to U-Blocks.

NOTICE


Clamp assembly should be rotated out slightly. Do not tighten completely at this stage.

6. Attach each side mounting hardware to left and right side U-Block with the 4 m6 screws with flat washers and split lock washer as shown.

NOTICE

It is essential that the included split lock washers (HBR066/4 each) and flat washers (HBR035/4 each) be used at all four attachment points to maintain desired angle and depth adjustment of back.

7. Adjust angle and depth to desired user preferences.
8. After final adjustment is complete, tighten all screws to the proper torque specification. This will ensure secure placement and safe operation.

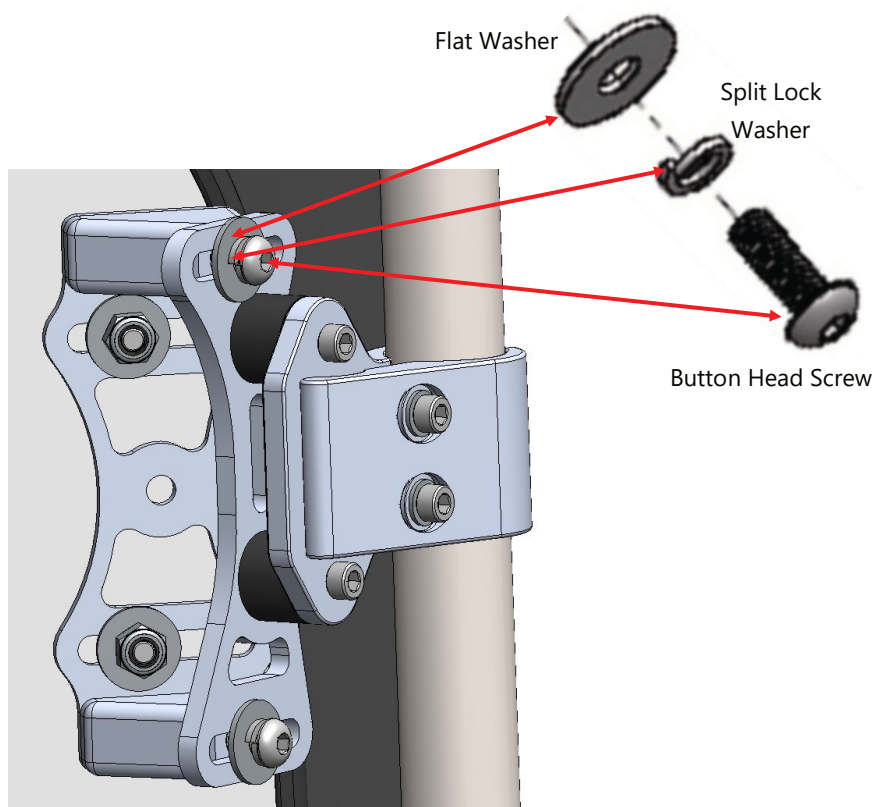**⚠ CAUTION**

If you are able to fold the back canes, the back should be in the folded position when transporting the wheelchair, especially before loading for air travel.

Angle Adjustment

8.2

1. On each side of the hardware, loosen the top and bottom screws with a 4mm Allen Wrench.
2. Adjust back support to achieve desired user angle.
3. After final adjustments are complete, tighten all screws to the proper torque specification.

WARNING

Protect all backs from blunt forces that may cause damage and/or breakage.

Depth Adjustment

8.3

1. On each side of the hardware, loosen the top and bottom screws with a 4mm Allen Wrench.
2. Slide back support forward or aft to achieve desired users depth.
3. After final adjustments are complete, tighten all screws to the proper torque specification.


Width Adjustment

8.4

1. Remove fabric cover from back support.
2. On each side of the hardware, loosen the top and bottom screws using a 4mm Allen Wrench and a 10mm wrench.
3. Adjust hardware inward or outward to achieve required mounting hardware width.
4. After final adjustments are complete, tighten all screws to the proper torque specification.

CAUTION

When positioning an ADI back, the interior portion of the back support should be positioned to make a contact at the PSIS to provide optimal pelvic support and trunk control.

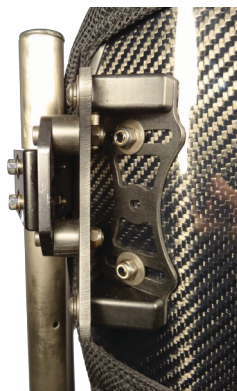Dealer Assistance

9.1

During first time use by the client, it is advised that the dealer or service technician assists and explains the set-up to the customer (the user and/or the attendant). If needed, the dealer can make final adjustments.

CAUTION

ADI backs may not fit every chair. Please consult your clinician or supplier providing you equipment.

CAUTION

Do not attach the ADI backs with the user in the chair.

User Testing

9.2

It is important that the customer is fully aware of the installation, how to use it, and what can be adjusted in order to gain as much mobility as possible. As a dealer, proceed as follows:

- Explain and show the customer how you have executed the installation, and explain the function of the hardware.
- If needed, adjust the hardware to the proper position.
- Explain to the customer possible problems and how to address them.

CAUTION

Do not use ADI backs during vehicle transportation.

Conditions of Use

9.3

The ADI hardware is intended for use as installed by the dealer, in accordance to the installation instructions in this manual.

- The foreseen conditions of use are communicated by the dealer or service technician to the user and/or attendant during the first time use.
- If the usage conditions change significantly, please contact your dealer or a qualified service technician to avoid excessive wear and tear or unintended damage.

Cleaning

10.1

- Use a soft cloth to clean the back shell.
- Hand wash the foam pad cover with mild, soapy water or detergent and let dry.
- Foam pad should NOT get wet.

**WARNING**

Do NOT hand wash in water temperatures about 160°F (70°C).

**CAUTION**

Institution **MUST** clean and disinfect product between multiple patient uses.

**WARNING**

Do NOT wash or machine dry.

Disinfection

10.2

- Wipe gently with a cloth dampened with a household disinfectant.
- Allow foam to air dry thoroughly.

**WARNING**

Do NOT immerse the foam in water.

SAFETY

Always check all mounting hardware, making sure each fastener is properly tightened before using the back support.

**WARNING**

250lbs. (113.40 kg) user weight limit.

NOTICE

Altering an ADI back in any way and/or improper installation will void the warranty.

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Stealth Products, LLC. • info@stealthproducts.com • www.stealthproducts.com