


Stealth Products[®]



STEALTH POSITIONING

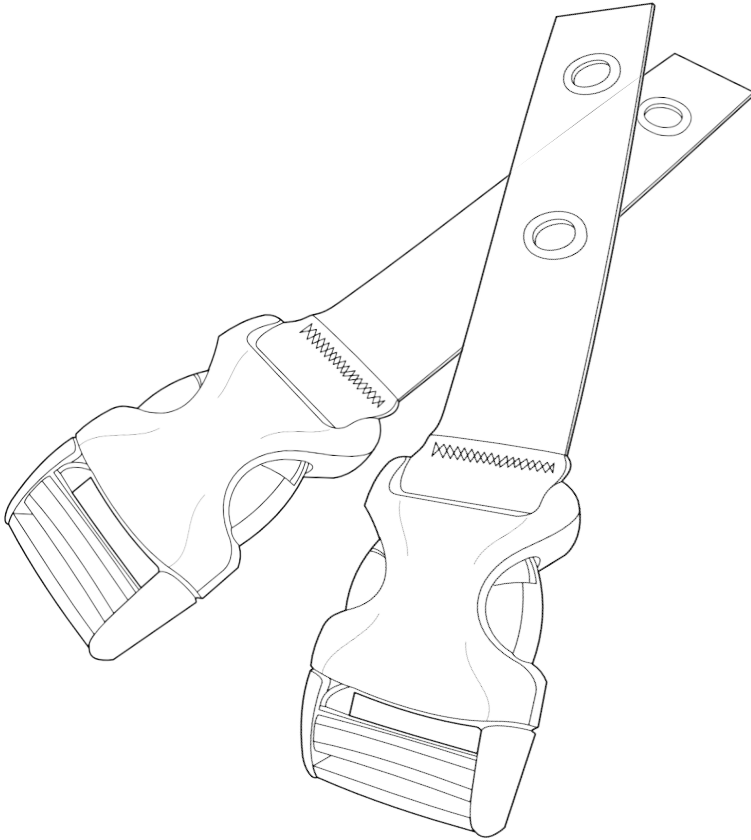

Accessories


Grommet Attachment Kit

Intended Use

The **Stealth Positioning™** Grommet Attachment Kit is designed provide an additional mounting option for customers using a Posture Support, Y-Strap, Backpack Strap, or H-Harness Package from the **Stealth Positioning** line of support straps and harnesses.

Installation and Use


Parts Included

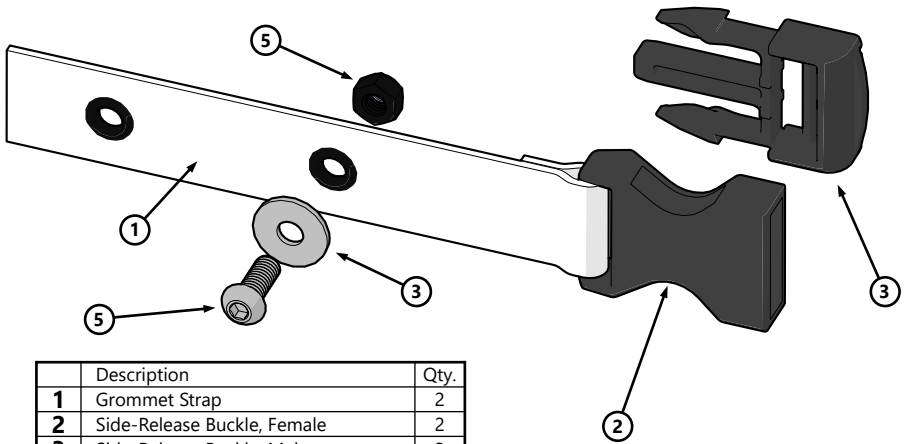

	Description	Qty.
1	Grommet Strap	2
2	Side-Release Buckle, Female	2
3	Side-Release Buckle, Male	2
4	M6 x 16mm Button Head Screw	2
5	M6 x 18mm Fender Washer	2
6	M6 x 1.0 Nylon Lock Nut	2

Installation

1. To ensure fit, the grommet straps come with male buckle sections that will be used in place of those that may have shipped with the straps or harness. Remove the original male buckles and replace with Part 3 from the list above.
2. Attach the male and female buckles of the grommet straps together.
3. Position the user in the chair and fit the strap or harness in the position in which it will be installed.
4. Adjust the length of the straps connected to the grommet strap and the angle of the grommet strap for best fit for the user. The buckle should be located such that it does not connect at the edge of a hard surface or make contact with the user while in use. If an edge

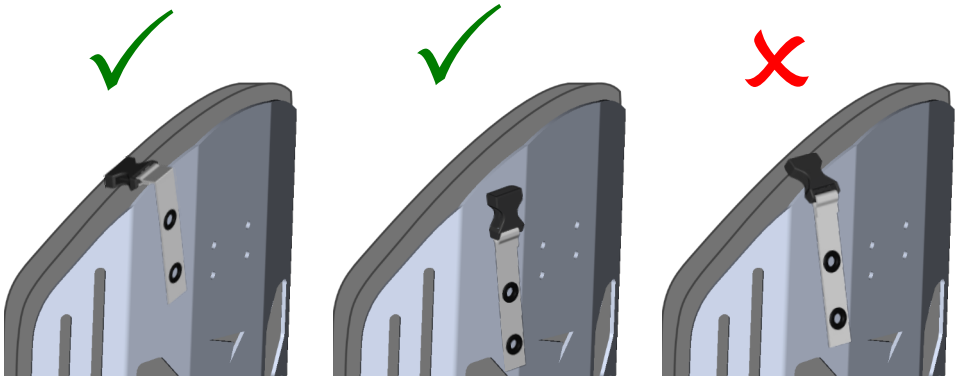

CAUTION

Ensure the side-release buckles of the Grommet Straps do not make contact with the user while in use.

needs to be traversed by the grommet strap, it should be crossed by the webbing of the grommet strap, or by the webbing of the strap or harness to which the grommet strap is connected.

5. When the desired position has been identified, mark the location of the grommet on the back shell. If tube mounting is required, a Stealth Positioning Cane Clamp #IFMDCC may be used to connect the strap to the tube.
6. If the location is not close to an existing accessory mounting slot, use a drill and #6 bit to drill a mounting hole in the proper location on the back shell.
7. Use only the supplied button head screws, washers, and nylon lock nuts to install the grommet straps to the back shell, cane, or tube. Insert the button head screw through the back shell from the side facing the user to minimize the chance of abrasion to seating components or soft tissue.

WARNING

This kit is designed to be used only with genuine Stealth Positioning straps and assemblies.

NOTICE

Inspect parts regularly for wear or damage. Have damaged parts repaired or replaced if worn or damaged. Periodically inspect fasteners and tighten, if necessary, to avoid damage or injury.

Care and Cleaning

We recommend that all products be examined on a regular basis for wear and function. Check the stitching on the webbing for stress or damage. If loose, frayed or broken parts are discovered, discontinue use and contact your provider. Repairs should only be performed by a wheelchair service professional and with genuine Stealth Products parts, fasteners or items appropriate for the service required.

Surfaces may be cleaned with a damp cloth and a mild, non-abrasive cleaner. Ensure all cleaners are approved for finished steel, aluminum, plastic and upholstered surfaces. Vests and straps that are removable and DO NOT contain metal components may be machine washed warm (40°C), permanent press cycle, mild process.

DO NOT bleach. DO NOT tumble dry. Dry flat. DO NOT iron.

Important

Never use a chest harness without a positioning belt. Positioning belts and supports should only be used to help support the user's posture. The belts, posture supports and straps must be snug, but must not interfere with the user's breathing. Check to make sure the user cannot slide down in the seating system, as they may suffer chest compression and may suffocate due to pressure from the belts or harnesses. A seat with angle or built-in contours may help prevent the user from sliding in the seat. Due to the possibility of a user sliding while in-chair, unsupervised use of anterior thoracic support devices could pose a serious choking hazard. Continually ensure that the user is sitting upright and that the thoracic support device does not obstruct the user's breathing in any way.

These devices do not conform to ANSI/RESNA WC20 or SAE J2249 and are not designed or approved to provide restraint or protection in the event of a crash. Wheelchair users should transfer to a vehicle seat and use the vehicle seat belt system or a child safety seat that complies with federal safety standards. The wheelchair should then be stored and secured in the vehicle. Children weighing less than 48 lbs. must always transfer to an approved child safety seat.

Before you install or begin using any product from **Stealth Products' Stealth Positioning™** line, it is important that you read and understand the content of these installation and operating instructions. Product misuse due to failure to adhere to instructions and warnings in this document will void your warranty and may result in property damage, injury or death. Always keep the operating instructions in a safe place so they may be referenced as necessary. If this document contains information you do not understand or if there are concerns about safety or operation, contact your provider.

All information, pictures, illustrations and specifications are based on the product information available at the time of printing, and are representative examples. They are not intended to be exact descriptions of the various parts of the product. Instructions are written with the expressed intent of use with standard configurations. They also contain important safety and maintenance information. For assistance with advanced configurations or options, contact your provider or **Stealth Products** at (512) 715-9995 or toll free at (800) 965-9229.

Immediately discontinue use if any function is compromised, if parts are missing or loose or if any component shows signs of excessive wear. Consult with your provider for repair, adjustment or replacement. You can download additional copies of this manual by visiting <https://stlpro.site/stealth-docs/> and searching for "P72D691" in the document manager.

Warranty

Our products are designed, manufactured, and produced to the highest of standards. If any defect in material or workmanship is found, **Stealth Products** will repair or replace the product at our discretion. Any implied warranty, including the implied warranties of merchantability and fitness for a particular purpose, shall not extend beyond the duration of this warranty. **Stealth Products, LLC** does not warrant damage due to, but not limited to, misuse, abuse, or misapplication of products and modification of product without written approval from **Stealth Products, LLC**.

Any alteration or lack of serial number, where applicable, will automatically void this warranty. **Stealth Products, LLC** is liable for replacement parts only. **Stealth Products, LLC** is not liable for any incurred labor costs. No person is authorized to alter, extend, or waive the warranties of **Stealth Products, LLC**.

To learn more about our warranty and returns policy, please visit <https://stlpro.site/returns-n-warranty>.

About Us

Stealth Products is committed to 100% customer satisfaction. Your complete satisfaction is important to us. Please contact us with suggestions or feedback to help us improve the quality and usability of our products.

You may reach us at:

 **Stealth Products, LLC**
104 John Kelly Drive,
Burnet, TX 78611

Phone: (512) 715-9995 Toll Free: (800) 965-9229

Fax: (512) 715-9954 Toll Free: (800) 806-1225

info@stealthproducts.com - www.stealthproducts.com

EU REP PRIDE MOBILITY PRODUCTS EUROPE BV. De Zwaan 3, 1601 MS Enhuizen, The Netherlands	CH REP SODIMED SA Chemin Praz Devant 12 1032 Romanel sur-Lausanne, Switzerland	UK Authorized Representative PRIDE MOBILITY PRODUCTS LTD. 32 Wedgwood Road Bicester, Oxfordshire OX26 4UL, United Kingdom
--	---	--