


Instructions

If present, remove the TruComfort 2 mounting hardware from the center spine of the TB3 back using a 4mm hex wrench before installing the **BHDP202** hardware.


- 1) Align the mounts with the appropriate holes on the back shell and TB3 center spine.
- 2) Install four rubber mounts to the TB3 center spine in the desired locations.
- 3) Using the four supplied M6 flathead bolts, install the interface bar to the rubber mounts. Tighten with a 4mm hex wrench.
- 4) Attach the shell to the interface bar using the four supplied M6 button head bolts. Tighten with a 4mm hex wrench.

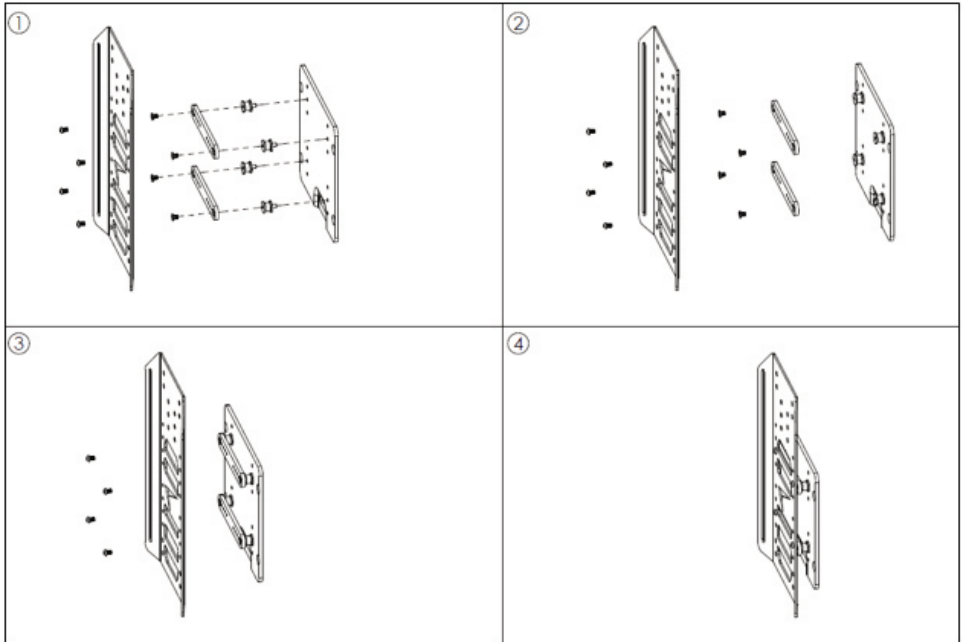

CAUTION

These products are designed to be fitted, applied, and installed exclusively by a healthcare professional trained for these purposes. The fitting, application and installation by a non-qualified individual could result in serious injury.



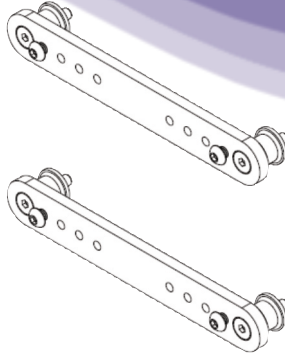

TB3 Interface Bar Back Mount

Intended Use

The **BHDP202 TB3 Interface Bar Back Mounting Package** installs directly into the **TruBalance 3 (TB3)** center spine, allowing the **True Forming Back (TFB)** or custom dimension **ADI Q-Back** to fit onto **TB3-compatible Quantum power chair bases**. Package includes two TB3 interface bars, four rubber mounts and attaching screws.

Note: Only required when mounting a 13-in. wide or narrower TFB or ADI Q-Back to a Quantum TB3 seat.

Our products are designed, manufactured and produced to the highest of standards. If any defect in material or workmanship is found, Stealth Products, LLC. will repair or replace the product at our discretion. Any implied warranty, including the implied warranties of merchantability and fitness for a particular purpose, shall not extend beyond the duration of this warranty.

No person is authorized to alter, extend, or waive the warranties of Stealth Products, LLC.



Stealth Products, LLC
104 John Kelly Drive,
Burnet, TX 78611

Phone: (512) 715-9995 Toll Free: (800) 965-9229
Fax: (512) 715-9954 Toll Free: (800) 806-1225
info@stealthproducts.com - www.stealthproducts.com



EU REP

EMERGO EUROPE B.V.
Westervoortsedijk 60,
6827 AT Arnhem, The Netherlands

CH REP

GELBART AG
Tribtschenstrasse 64
CH-6005 Luzern, Switzerland

UK Authorized Representative

EMERGO CONSULTING (UK) LIMITED
C/o Cr360 - UL International,
Compass House, Vision Park Histon,
Cambridge CB24 9BZ, United Kingdom