



Lower Extremity Support

Gel Footplate Cover

Intended Use


The Gel Footplate Cover is intended for use as a gel padding system for any Quantum footplate. Its soft gel interior and fitted cover provide users with increased comfort and aid in the prevention of pressure sores and abrasions.

Installation and Use



Step 1:

Remove one of the covers from the packaging and separate the hook and loop straps so they hang freely. Position the cover over the footplate so the straps align with their respective slots.

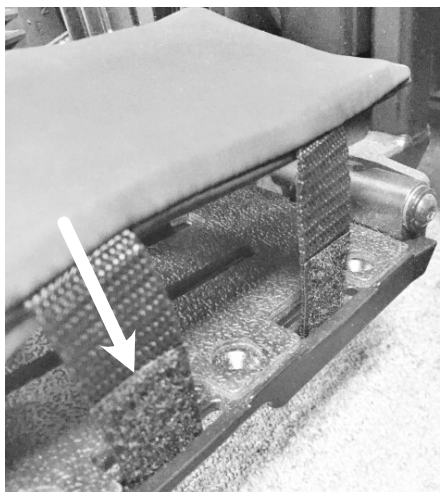


Step 2a:

Feed the cover's front straps through both the footplate's front slots. (Only one side shown in illustration.)



Step 2b:

Feed the cover's back straps through both the footplate's rear slots. (Only one side shown in illustration.)

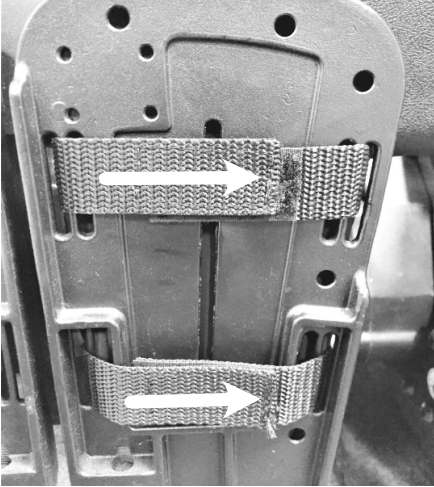

Step 3:

From underneath the footplate, pull the front straps tightly together and fasten them. Repeat for the rear straps. Make sure the cover is secure on the plate.

Repeat for the other cover.



WARNING

This product is designed to be fitted, applied, and installed exclusively by a healthcare professional trained for these purposes. The fitting, application, and/or installation by unqualified individuals could result in serious injury.

NOTICE

Remove gel from cover before washing. Failure to do so may result in damage to the product.

Care and Cleaning

We recommend that all products be examined on a regular basis for wear and function. Check the stitching on the cover and webbing for stress or damage. If loose, frayed or broken parts are discovered, discontinue use and contact your provider. Repairs should only be performed by a wheelchair service professional and with genuine Stealth Products parts, fasteners or items appropriate for the service required.

Cover should be cleaned with a damp cloth and a disinfectant. Ensure all cleaners are approved for finished steel, aluminum, plastic and upholstered surfaces.

DO NOT use bleach.

DO NOT tumble dry. Dry flat.

DO NOT iron.

About Us

Stealth Products is committed to 100% customer satisfaction. Your complete satisfaction is important to us. Please contact us with suggestions or feedback to help us improve the quality and usability of our products.

You may reach us at:



Stealth Products, LLC

104 John Kelly Drive,
Burnet, TX 78611

Phone: (512) 715-9995 Toll Free: (800) 965-9229
Fax: (512) 715-9954 Toll Free: (800) 806-1225
info@stealthproducts.com - www.stealthproducts.com



EU REP EMERGO EUROPE B.V. Westervoorstsedijk 60, 6827 AT Arnhem, The Netherlands	CH REP GELBART AG Tribtschenstrasse 64 CH-6005 Luzern, Switzerland	UK Authorized Representative EMERGO CONSULTING (UK) LIMITED C/o Cr360 - UL International, Compass House, Vision Park Histon, Cambridge CB24 9BZ, United Kingdom
---	---	---

Warranty

Our products are designed, manufactured, and produced to the highest of standards. If any defect in material or workmanship is found, **Stealth Products** will repair or replace the product at our discretion. Any implied warranty, including the implied warranties of merchantability and fitness for a particular purpose, shall not extend beyond the duration of this warranty. **Stealth Products, LLC** does not warrant damage due to, but not limited to, misuse, abuse, or misapplication of products and modification of product without written approval from **Stealth Products, LLC**.

Any alteration or lack of serial number, where applicable, will automatically void this warranty. **Stealth Products, LLC** is liable for replacement parts only. **Stealth Products, LLC** is not liable for any incurred labor costs. No person is authorized to alter, extend, or waive the warranties of **Stealth Products, LLC**.

Please visit <https://stlpro.site/returns-n-warranty> to learn more about our warranty and returns policy.

Important

Before you install or begin using any product, it is important that you read and understand the content of these installation and operating instructions. Product misuse due to failure to adhere to instructions and warnings in this document will void your warranty and may result in property damage, injury or death. Always keep the operating instructions in a safe place so they may be referenced as necessary. If this document contains information you do not understand or if there are concerns about safety or operation, contact your provider.

All information, pictures, illustrations and specifications are based on the product information available at the time of printing, and are representative examples. They are not intended to be exact descriptions of the various parts of the product. Instructions are written with the expressed intent of use with standard configurations. They also contain important safety and maintenance information. For assistance with advanced configurations or options, contact your provider or **Stealth Products** at (512) 715-9995 or toll free at (800) 965-9229.

Immediately discontinue use if any function is compromised, if parts are missing or loose or if any component shows signs of excessive wear. Consult with your provider for repair, adjustment or replacement. You can download additional copies of this manual by visiting <https://stlpro.site/stealth-docs> and searching for "Gel Footplate Pad" in the document manager.